




 Breezy EC[®]

PARTS MANUAL

MAY 2013


BREEZY EC	4
BREEZY EC (2000, 2000HD, 3000, & 4000)	4
BASE FRAME	4
FRAME	4
1000 FRAME	4
2000 FRAME (FIXED ARMREST)	5
2000 & 3000 FRAME (REMOVABLE ARMREST)	6
2000 HD FRAME	7
4000 FRAME	8
SUPER HEMI KIT (4000)	9
AXLE PLATES	9
AMPUTEE AXLE PLATE (1000-2000)	10
AMPUTEE AXLE PLATE (2000HD)	11
AMPUTEE AXLE PLATE (4000)	12
CASTERS & FORKS	12
STANDARD CASTER FORK ASSM (16"-20" WIDE)	13
2000HD (22"-24" WIDE) CASTER FORK ASSM	14
2000HD (26"-30" WIDE) CASTER FORK ASSM	15
ANTI-TIPS	15
ADJUSTABLE REAR ANTI-TIP (1000, 2000, 2000HD, 3000,4000)	16
STANDARD REAR ANTI TIP (1000,2000,2000HD,3000)	17
REAR ANTI TIP (4000)	18
WHEEL LOCKS	18
WHEEL LOCK FIXED ARMREST FRAME (1000,2000)	19
WHEEL LOCK REMOVABLE ARMREST (2000, 3000)	20
WHEEL LOCK (4000)	21
SUPER HEMI KIT (4000)	22
WHEELS	22
8 SPOKE MAG WHEEL	23
SUPER HEMI KIT (4000)	25
SEATING & POSITIONING	25
SEAT	25
2" SEAT EXTENSION WITH UPHOLSTERY	26
SEAT AND BACK UPHOLSTERY 1000,2000,3000	28
SEAT AND BACK UPHOLSTERY 4000	30
BACK	30
SEAT AND BACK UPHOLSTERY 1000,2000,3000	31
SEAT AND BACK UPHOLSTERY 4000	33
RECLINER BACKREST	34
HANGERS	36
SWING AWAY HANGER	36
COMPOSITE SWING-AWAY FOOTRESTS W/ HEEL LOOP	37
EASY CARE ELEVATING LEG REST	38
LOW PROFILE COMPOSITE ELEVATING LEG REST (2000, 4000)	39
STANDARD FOOTPLATE AND EXTENSION (FIXED)	40
EASY CARE ARTICULATING LEG REST	41
ARMRESTS	41
ARMPADS	41
FULL ARMREST ARMPADS	42
DESK ARMREST ARMPADS	43
ARMREST	43
ARMREST DESK PADDED (2000, 2000HD)	44
ARMREST FULL PADDED (2000, 2000HD)	45
ARMREST DESK PADDED (3000)	46
ARMREST DESK PADDED (4000)	47
ARMREST FULL PADDED (4000)	48
ARMREST RECIEVER (4000)	49
HEIGHT ADJUSTABLE ARMREST	49
HEIGHT-ADJUSTABLE ARMREST FULL PADDED (2000, 3000, 4000 RECLINER)	50
HEIGHT-ADJUSTABLE ARMREST DESK PADDED (2000, 3000, 4000 RECLINER)	51
HEIGHT-ADJUSTABLE ARMREST FULL PADDED (4000)	52
HEIGHT-ADJUSTABLE ARMREST DESK PADDED (4000)	53
ACCESSORIES	53
OVERHEAD ANTI-THEFT DEVICE	54
ANTI-FOLD/ANTI-THEFT DEVICE	55
OXYGEN TANK HOLDER	56
COMBINATION O2 IV HOLDER	57
TELESCOPING IV HOLDER	58
TOOL KIT	59
WHEEL LOCK EXTENSION HANDLE	60

SEAT BELTS.....	61
TIRE PUMP.....	62
----- BREEZY EC TRANSPORT.....	62
TRANSPORT CHAIR (STEEL & ALUMIUM)	63
EC TRANSPORT CHAIR DISCONTINUED.....	63
FRAME (STEEL/ALUMINUM TRANSPORT).....	64
CASTER FORK ASSM (STEEL TRANSPORT).....	65
CASTER FORK ASSM (ALUMINUM TRANSPORT)	66
REAR WHEEL-EASY	67
WHEEL LOCK (STEEL TRANSPORT).....	68
WHEEL LOCK (ALUMINUM TRANSPORT)	69
FOLDING TRANSPORT UPHOLSTERY (STEEL/ALUMINUM)	70
SWING-AWAY FRAME HANGER (ALUMINUM/STEEL TRANSPORT)	71
back.....	72

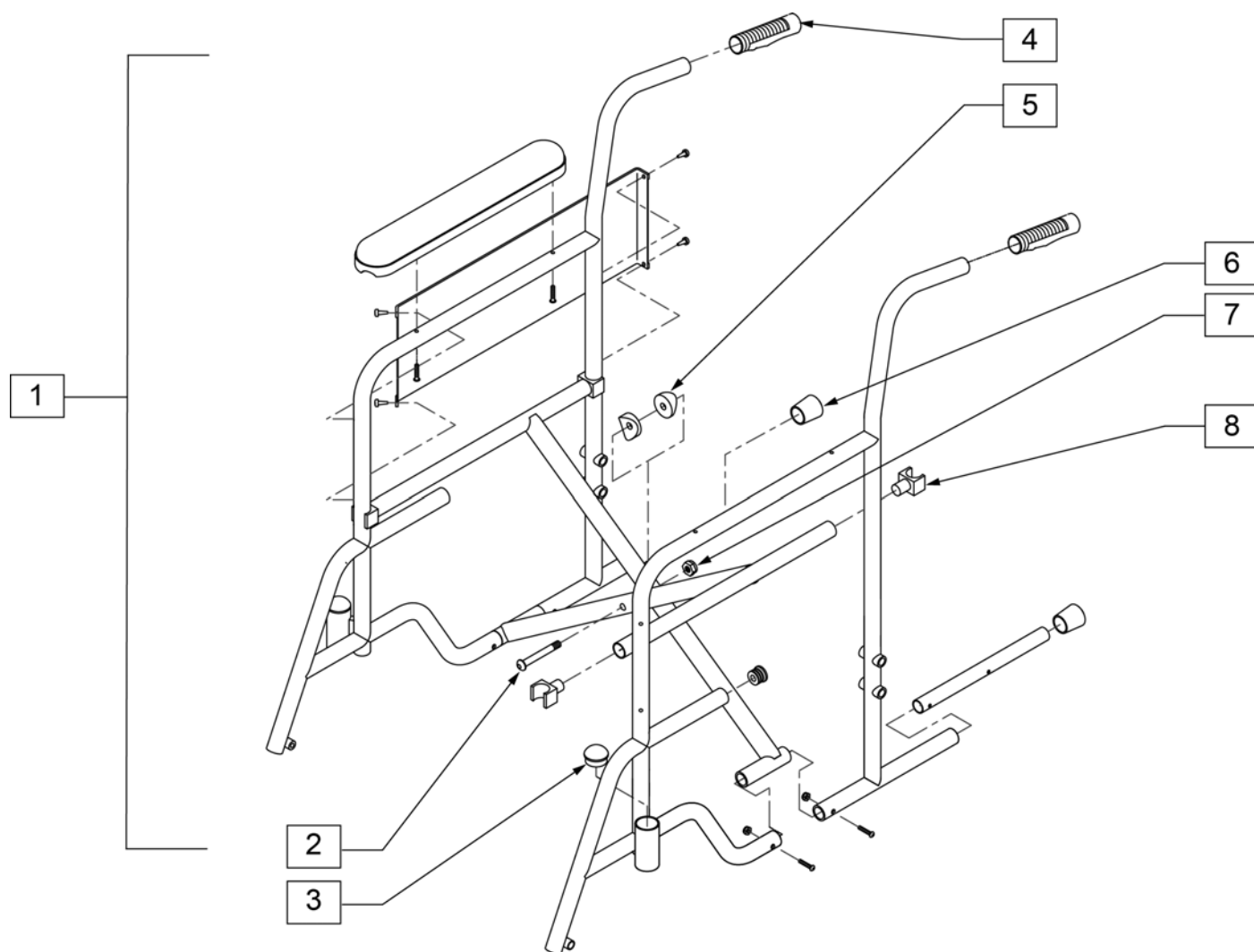


Pos.	Item Number	Description
2, 5, 7	GEY-CBOLT	1" CROSSBOLT PLUS SPACERS
3	GEY-CASC	CASTER HOUSING CAPS
4	GEY-HAND	BACK CANE HANDLES
6	GEY-STPC	STEPPER TUBE CAPS
8	GEY-SRGUIDE	SEAT RAIL GUIDE

Note: Replacement frame for the EC1000 is not available as a replacement part.

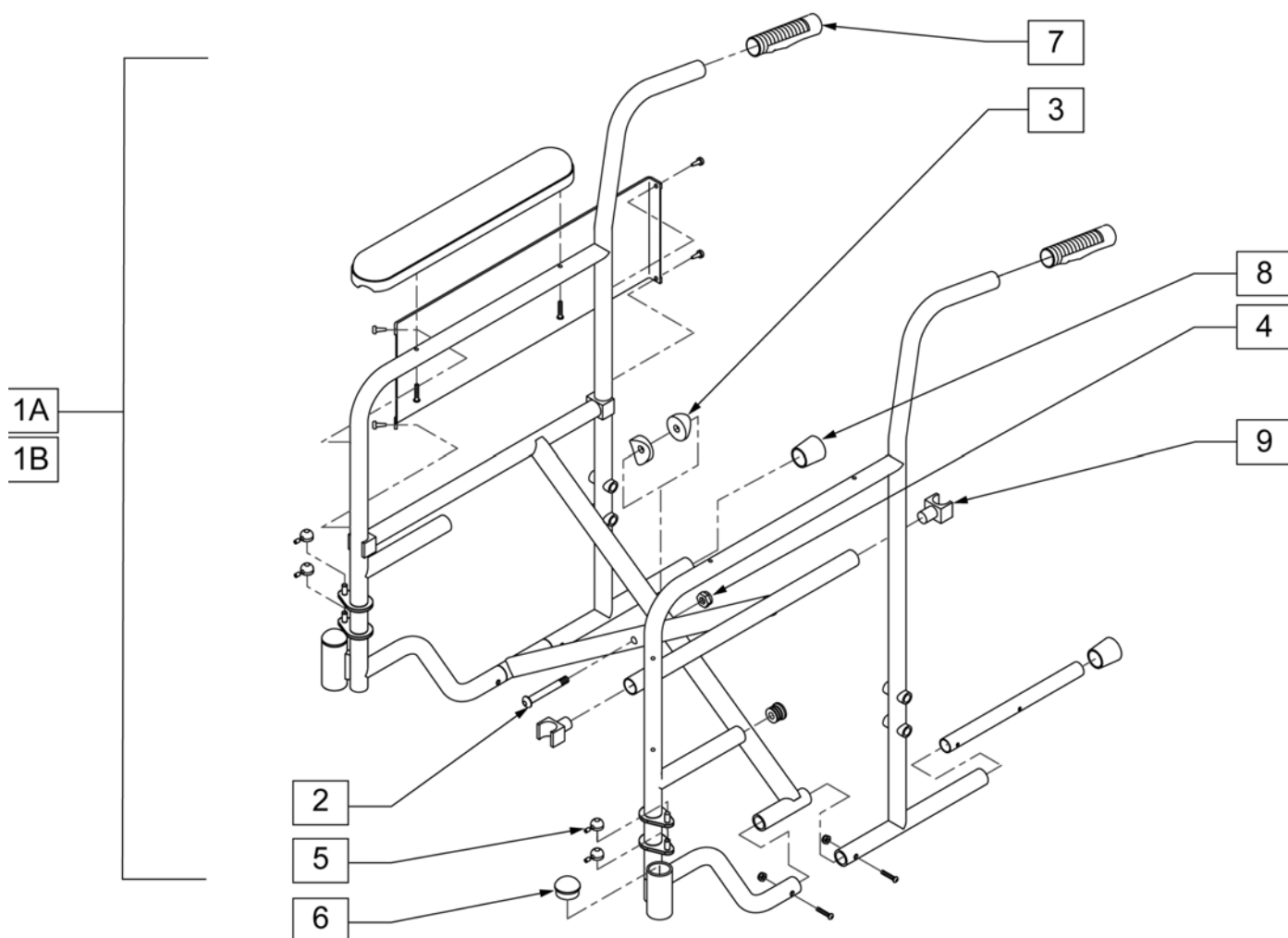


Pos.	Item Number	Description	Remarks
1A	GEY-216FRFR	2K FRAME ASSM 16"	
1B	GEY-218FRFR	2K FRAME ASSM 18"	
2, 3, 4	GEY-CBOLT	1" CROSSBOLT PLUS SPACERS	
5	GEY-TIN	ONE PAIR TINNEMAN NUTS	PAIR
6	GEY-CASC	CASTER HOUSING CAPS	
7	GEY-HAND	BACK CANE HANDLES	
8	GEY-STPC	STEPPER TUBE CAPS	
9	GEY-SRGUIDE	SEAT RAIL GUIDE	

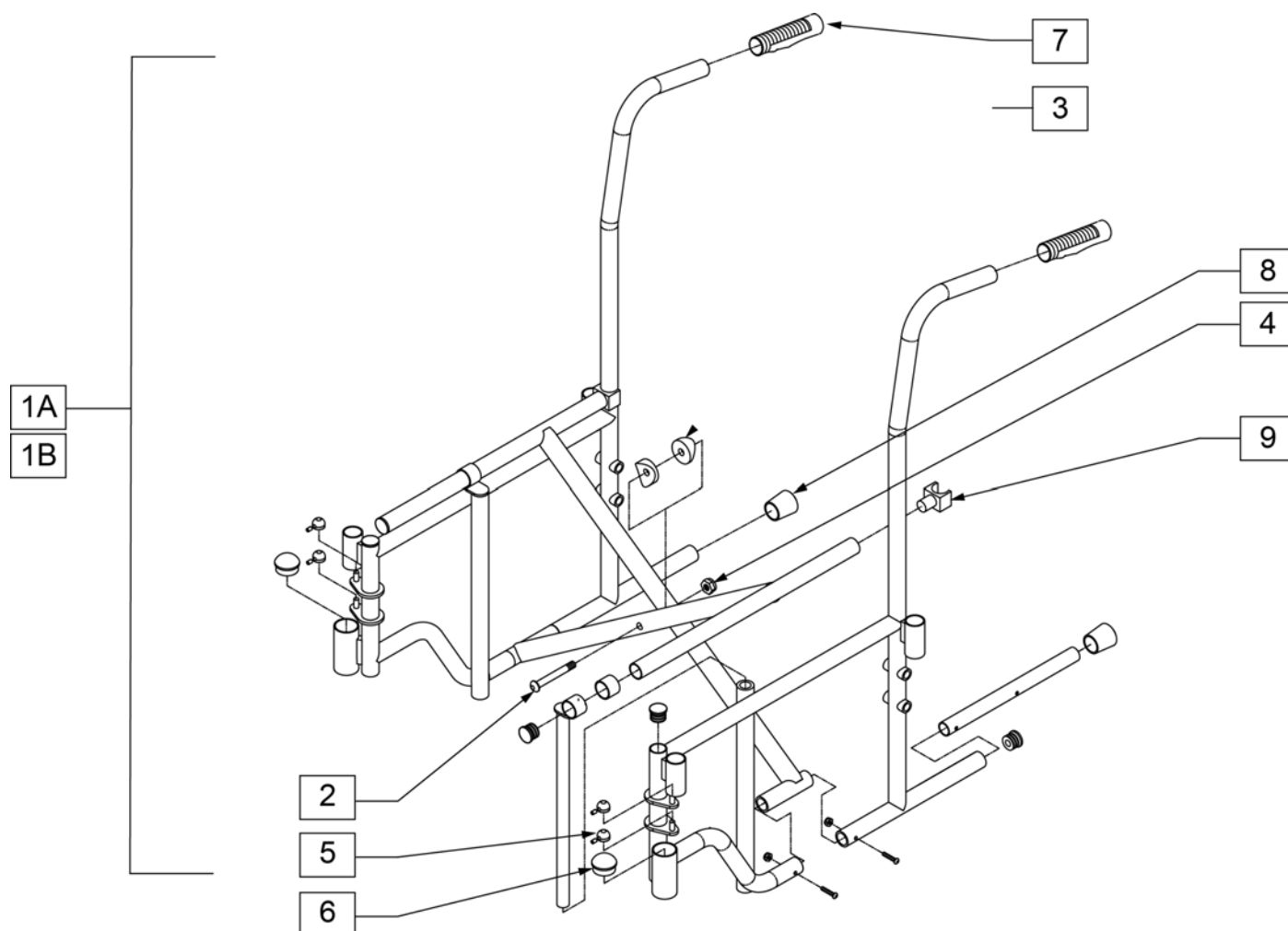


Pos.	Item Number	Description	Remarks
1A	GEY-216RRFR	2K FRAME ASSM 16"	2000 (16 WIDE)
1A	GEY-218RRFR	2K FRAME ASSM 18"	2000 (18 WIDE)
1A	GEY-220RRFR	2K FRAME ASSM 20"	2000 (20 WIDE)
1B	GEY-318FR	18"3000 FRAME ASSEMBLY	3000 (18 WIDE)
1B	GEY-320FR	20"3000 FRAME ASSEMBLY	3000 (20 WIDE)
5	GEY-TIN	ONE PAIR TINNEMAN NUTS	PAIR
6	GEY-CASC	CASTER HOUSING CAPS	
7	GEY-HAND	BACK CANE HANDLES	
8	GEY-STPC	STEPPER TUBE CAPS	
9	GEY-SRGUIDE	SEAT RAIL GUIDE	

Note: Replacement frames for the 26", 28", & 30" wide EC2000 are not available as a replacement part.

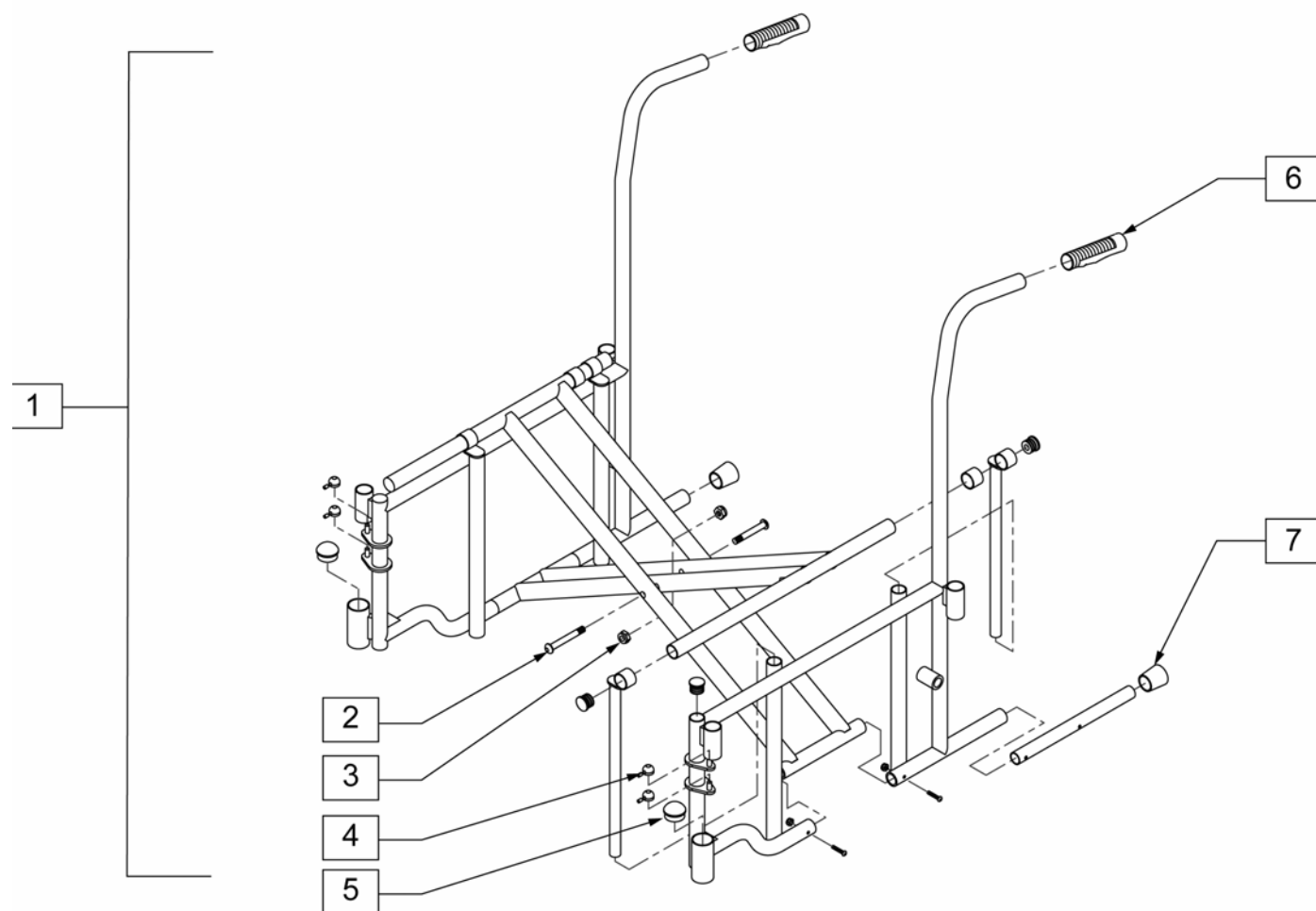


Pos.	Item Number	Description	Remarks
1	GEY-222HDFR	22"2000 FRAME ASSEMBLY	FOR 22" WIDE CHAIRS
1	GEY-224HDFR	24"2000 FRAME ASSEMBLY	FOR 24" WIDE CHAIRS
4	GEY-TIN	ONE PAIR TINNEMAN NUTS	PAIR
5	GEY-CASC	CASTER HOUSING CAPS	
6	GEY-HAND	BACK CANE HANDLES	
7	GEY-STPC	STEPPER TUBE CAPS	

Note: Replacement frame and Push Handles for the EC 4000 Recliner are not available as a replacement parts.

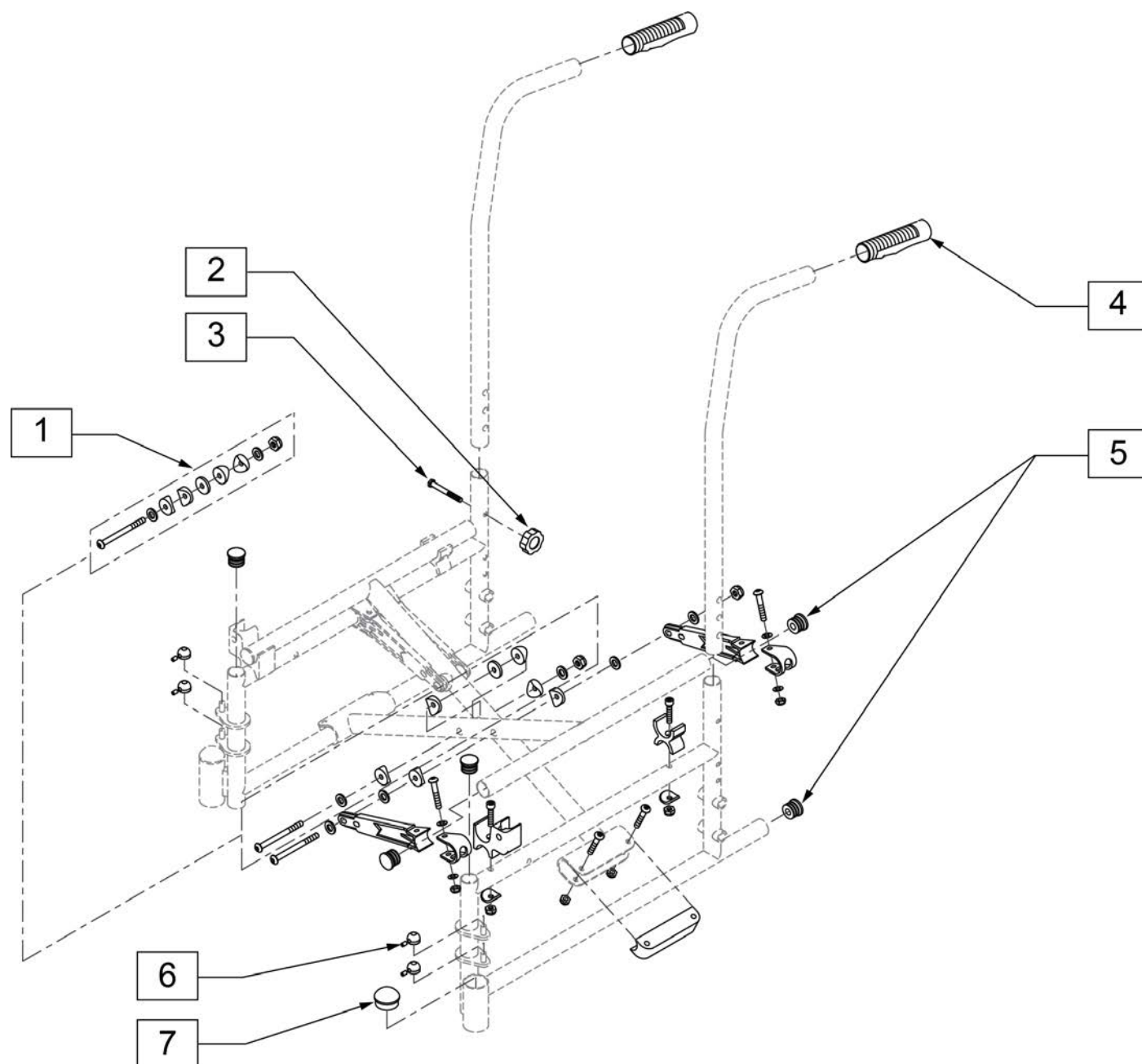


Pos.	Item Number	Description	Remarks
1	GEY-4CBOLT	1" CROSSBOLT PLUS SPACERS	4000
2,3	GEY-4HAB1	4000 ADJ KNOBS W/ BOLTS	
4	GEY-HAND	BACK CANE HANDLES	
5	GEY-STPC	STEPPER TUBE CAPS	
6	GEY-TIN	ONE PAIR TINNEMAN NUTS	PAIR
7	GEY-CASC	CASTER HOUSING CAPS	


NOTE: Super Hemi Kit includes QTY (2) 20" Wheel Assemblies; QTY (2) 6" Casters; QTY (2) Forks; & QTY (2) Amputee Wheel Lock Extensions.

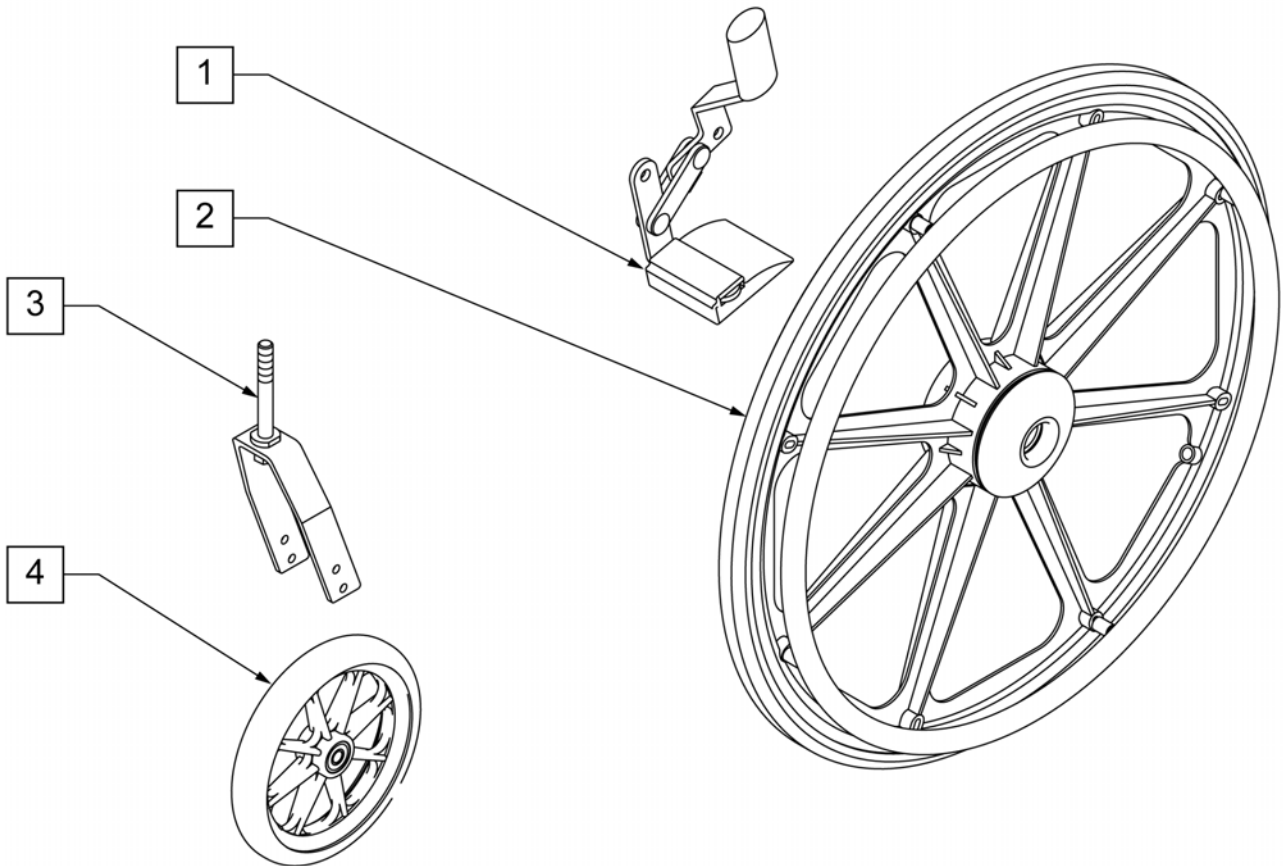


Pos.	Item Number	Description	Remarks
1	GEY-WDG	AMPUTEE WHEEL LOCK EXTENSION	
1-4	GEY-SHEMI	SUPER HEMI KIT	LOWERS TO 15 3/4" SEAT HEIGHT.

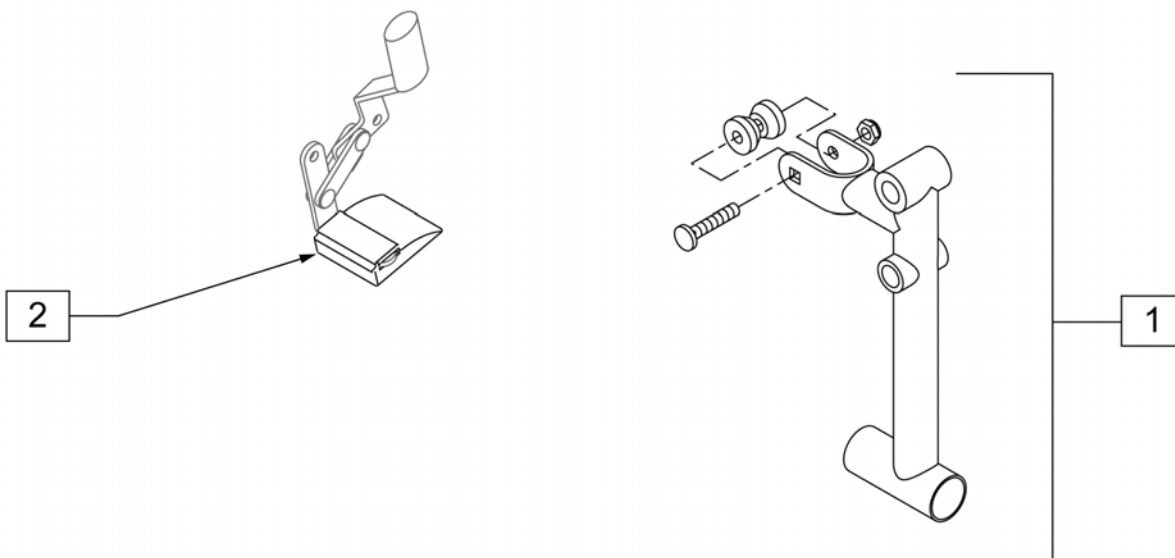

Pos.	Item Number	Description	Remarks
1	GEY-AMP	EC AMPUTEE AXLE PLATE	
2	GEY-WDG	AMPUTEE WHEEL LOCK EXTENSION	MUST BE USED W/ AMPUTEE AXLE PLATE

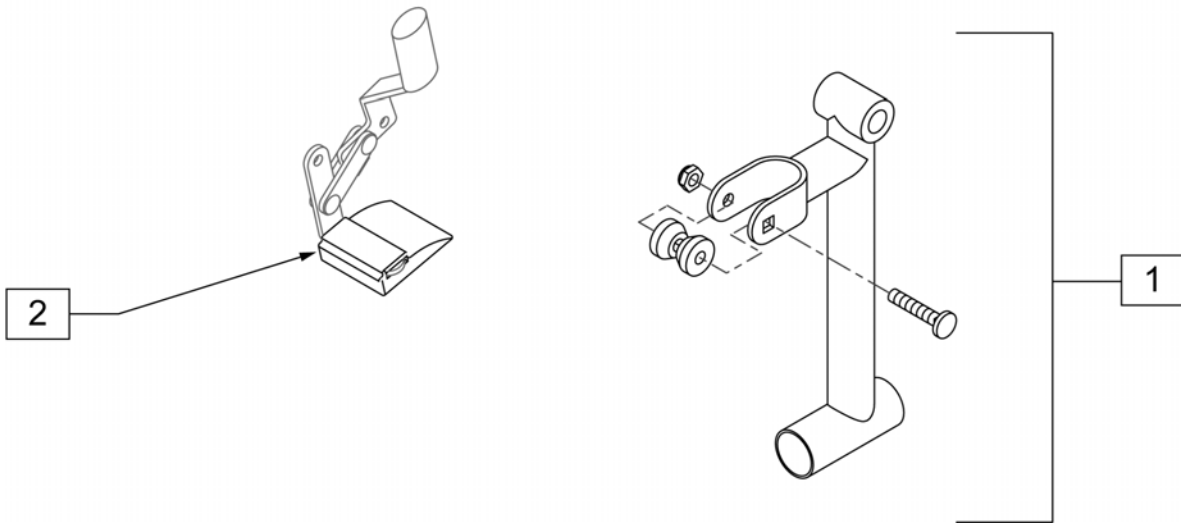


Pos.	Item Number	Description	Remarks
1	GEY-HDAMP	HEAVY DUTY AMPUTEE AXLE PLATE	WORKS WITH 22" & 24" WIDE CHAIRS ONLY
2	GEY-WDG	AMPUTEE WHEEL LOCK EXTENSION	MUST BE USED W/ AMPUTEE AXLE PLATE

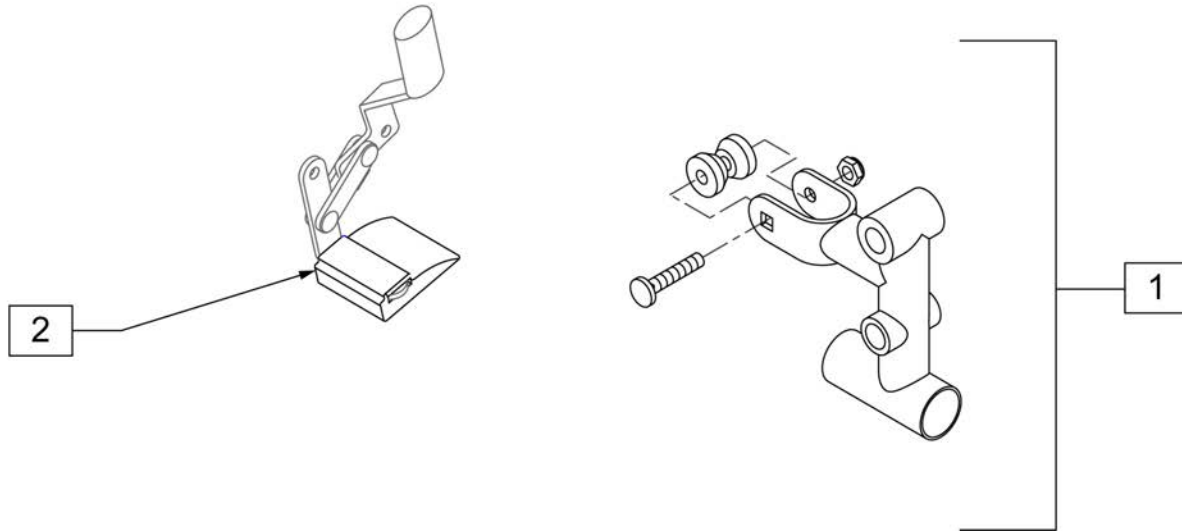

Pos.	Item Number	Description	Remarks
1	GEY-4AMP	AMPUTEE AXLE PLATE	
2	GEY-WDG	AMPUTEE WHEEL LOCK EXTENSION	MUST BE USED W/ AMPUTEE AXLE PLATE


NOTE: ITEM 2 REQUIRES A 19mm SOCKET TO REMOVE

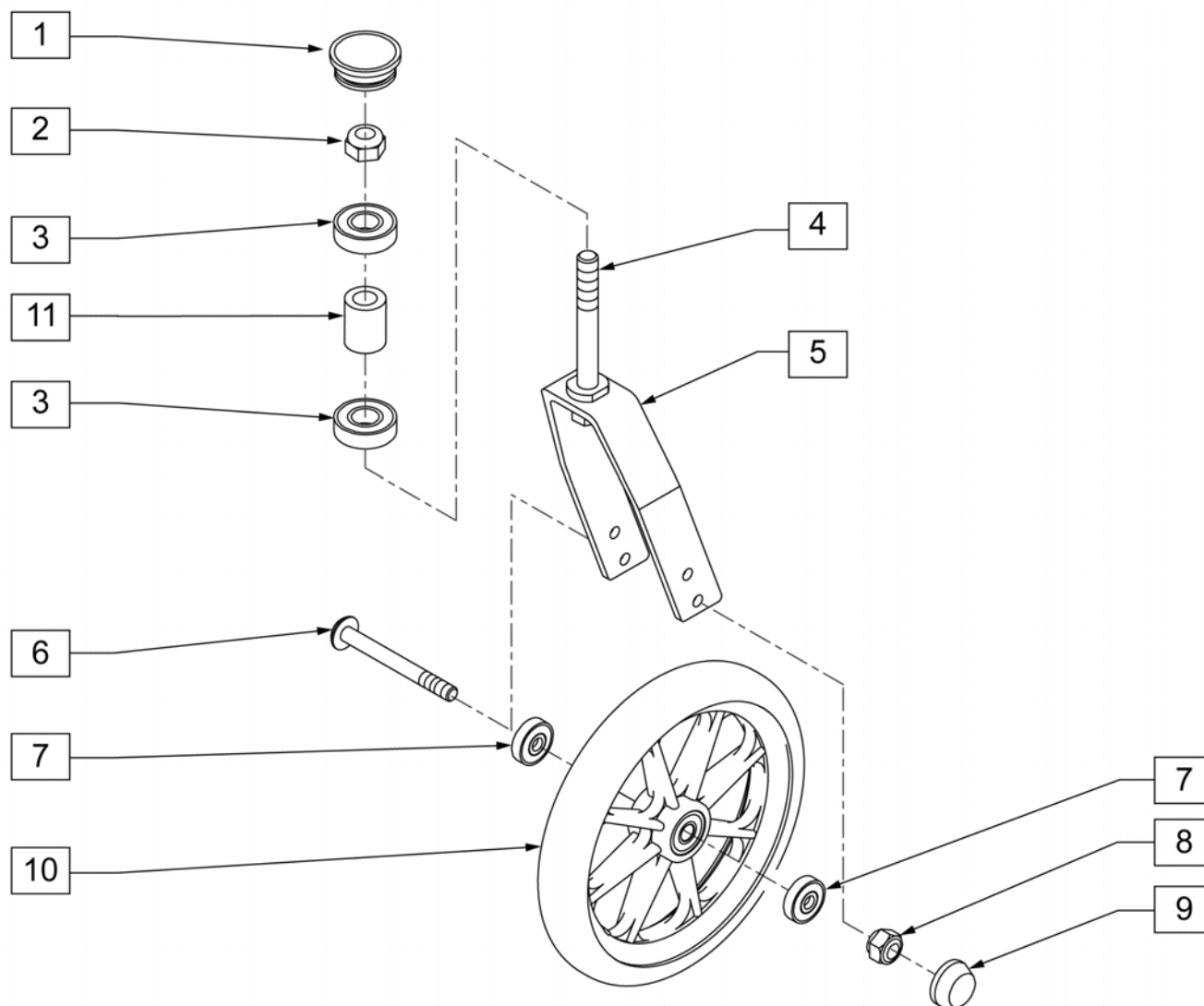

Pos.	Item Number	Description	Remarks
1	GEY-CASC	CASTER HOUSING CAPS	20 PER BAG
2	N/A	CASTER STEM LOCK NUT	M8 X 1.25 LOCKING NUT
3, 11	GEY-CASTJR	CAST JOURNAL BEAR & SLEEVE	PAIR INCLUDES 4 BEARINGS AND 2 SLEEVES - BEARING SIZE = 28mm X 12mm X 36.25mm
6, 7	GEY-CASTBR	CASTER BEARINGS	PAIR - SIZE=22mm X 8mm X 31.5mm
9	GEY-CASTCAP	CASTER TIR BOLT CAP	20 PER BAG
1-11	GEY-2CA	CASTER FORK ASSM	PAIR


NOTE: ITEM 10 MEASURES 8" X 1.83".

NOTE: ITEM 2 REQUIRES A 19mm SOCKET TO REMOVE.

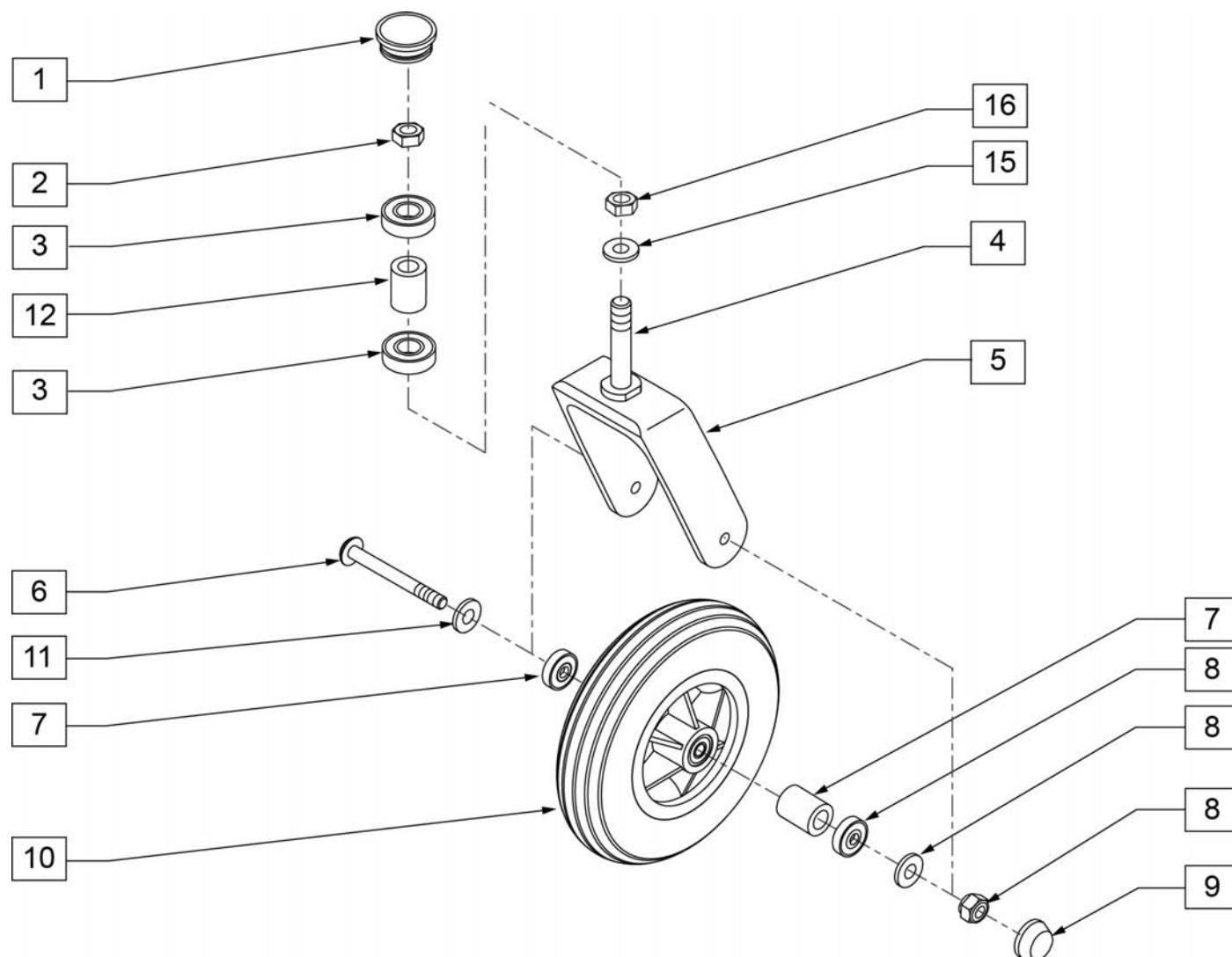


Pos.	Item Number	Description	Remarks
1	GEY-CASC	CASTER HOUSING CAPS	20 Per Bag
2	N/A	CASTER STEM LOCK NUT	M8 X 1.25 LOCKING NUT
3,12	GEY-CASTJR	CAST JOURNAL BEAR & SLEEVE	PAIR INCLUDES 4 BEARINGS AND 2 SLEEVES - BEARING SIZE = 28mm X 12mm X 36.25mm
9	GEY-CASTCAP	CASTER TIR BOLT CAP	PAIR
1-16	GEY-2HDCA	CASTER ASSM	PAIR


NOTE: ITEM 2 REQUIRES A 19mm SOCKET TO REMOVE

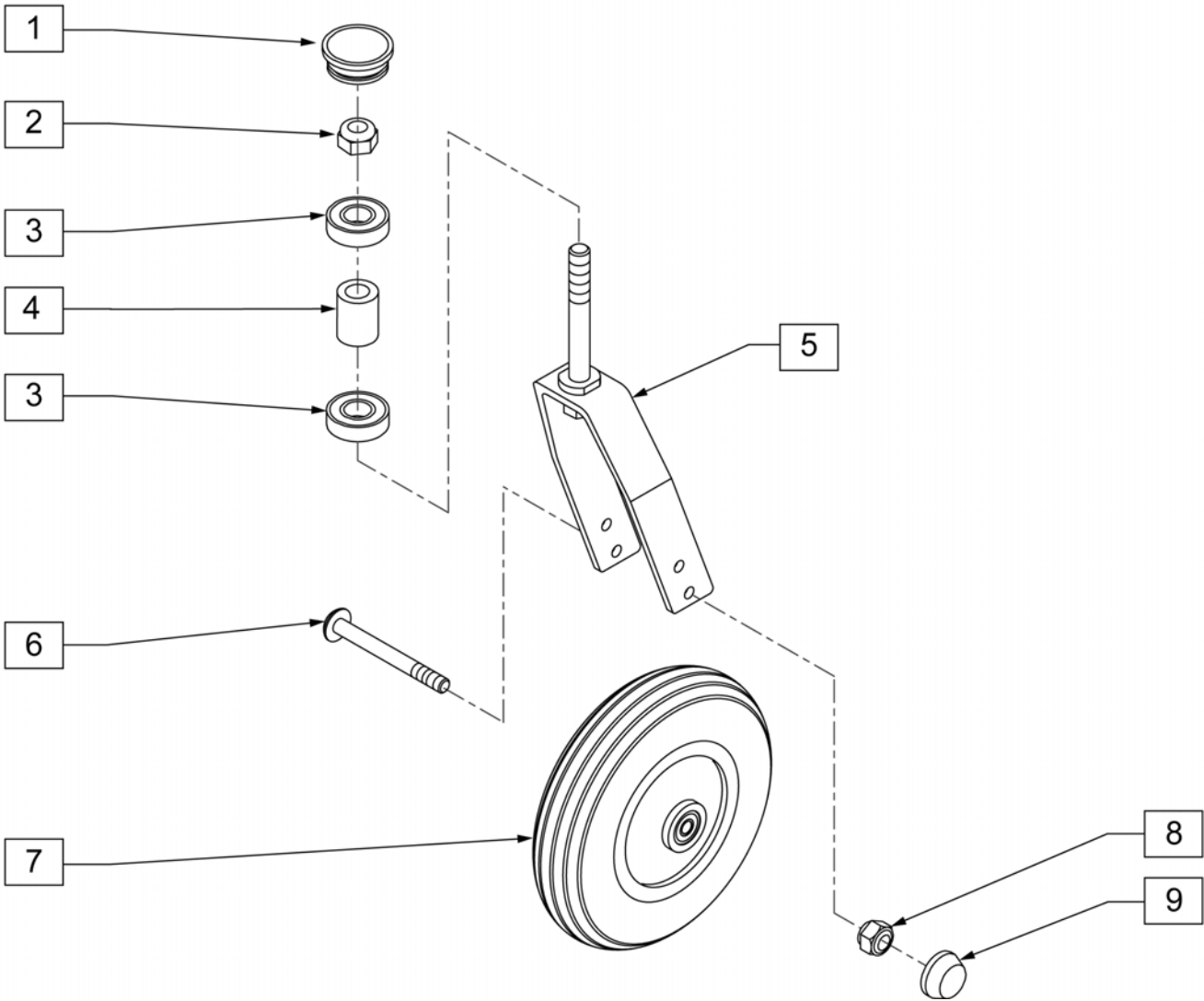

Pos.	Item Number	Description	Remarks
1	GEY-CASC	CASTER HOUSING CAPS	20 PER BAG
2-9	GEYB-CA	BARIATRIC CASTER ASSEMBLY KIT	

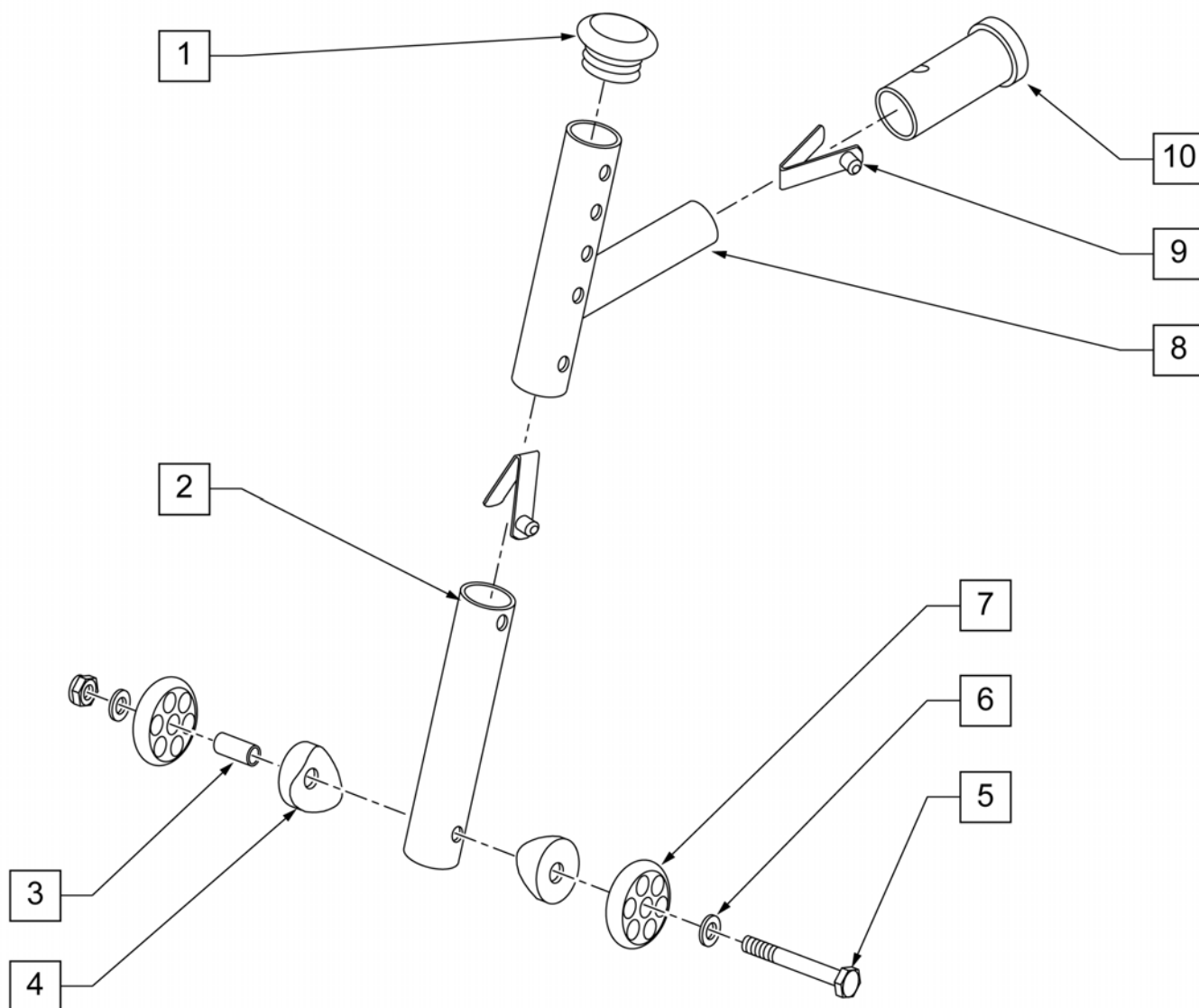


Pos.	Item Number	Description	Remarks
1-9	GEY-ATL	REAR ANTI TIPPER ADJUST	FITS 2000, 2000HD (22" & 24"), & 3000
1-10	GEY-4ATL	REAR ANTI TIPPER SUPER HEMI	FITS 4000 W/ SUPER HEMI KIT
1-10	GEYB-AT	BARIATRIC ANTI TIPPERS	FITS 2000HD (26", 28", & 30")

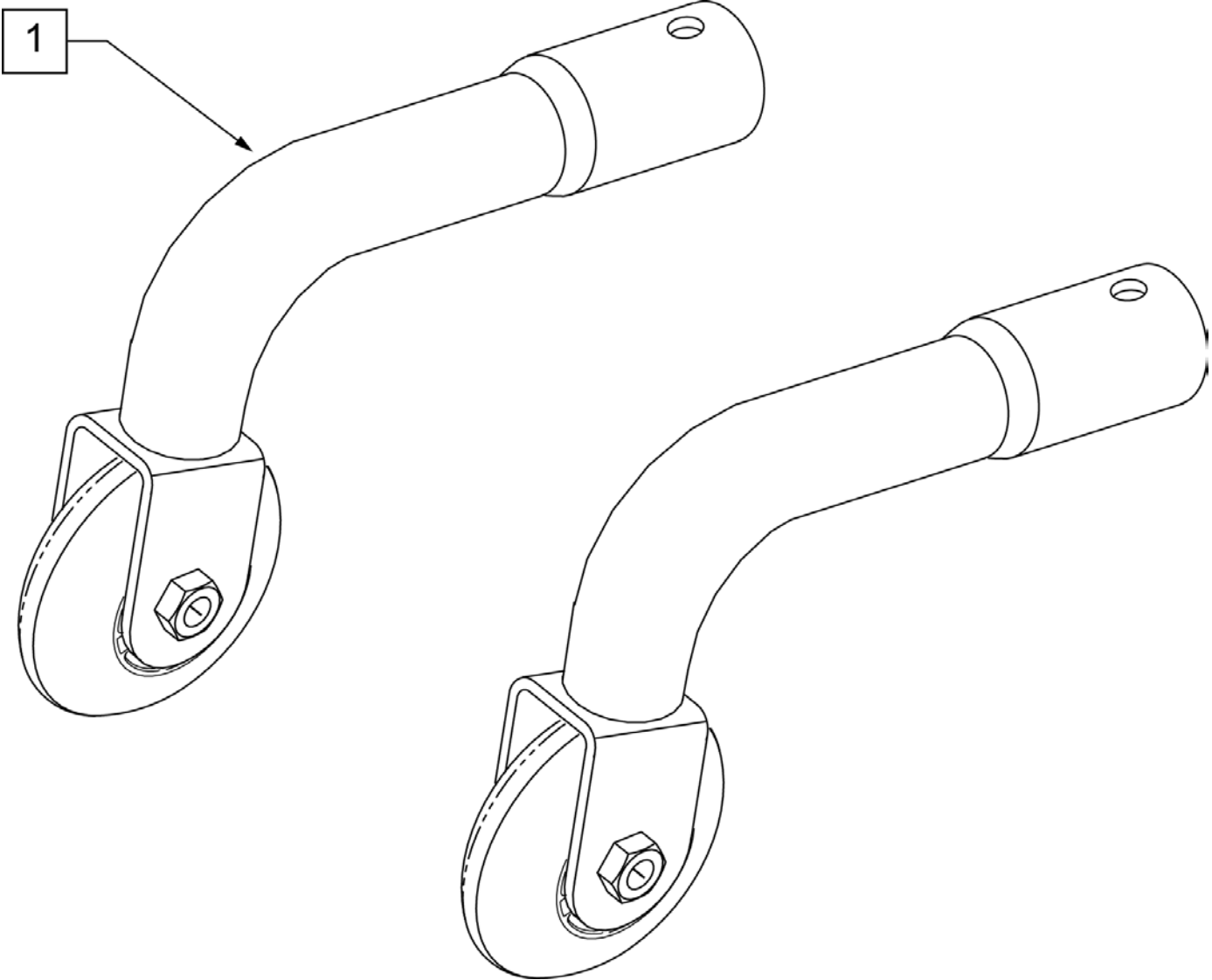


Pos.	Item Number	Description	Remarks
1	GEY-AT	EC REAR ANTI TIPPERS	PAIR - FITS (1000, 2000, 2000HD (22" & 24"), & 3000


NOTE: IF CHAIR IS EQUIPPED WITH SUPER HEMI KIT REFER TO "ADJUSTABLE REAR ANTI-TIP (1000,2000,2000HD,3000,4000)" PARTS MANUAL PAGE FOR REPLACEMENT PART.

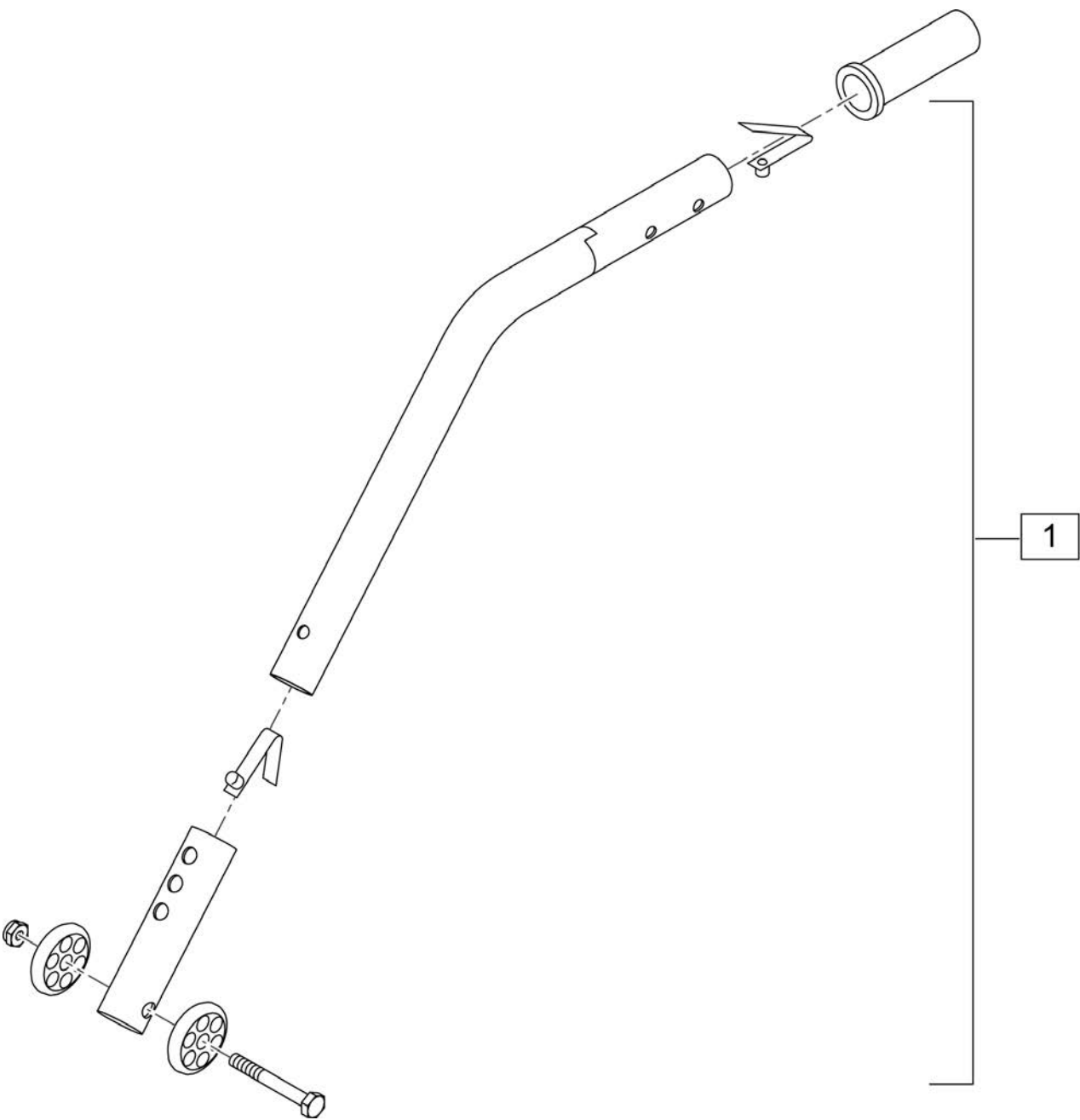


Pos.	Item Number	Description	Remarks
1	GEY-4AT	REAR ANTI-TIPS	FITS 4000 ONLY W/O SUPER HEMI KIT

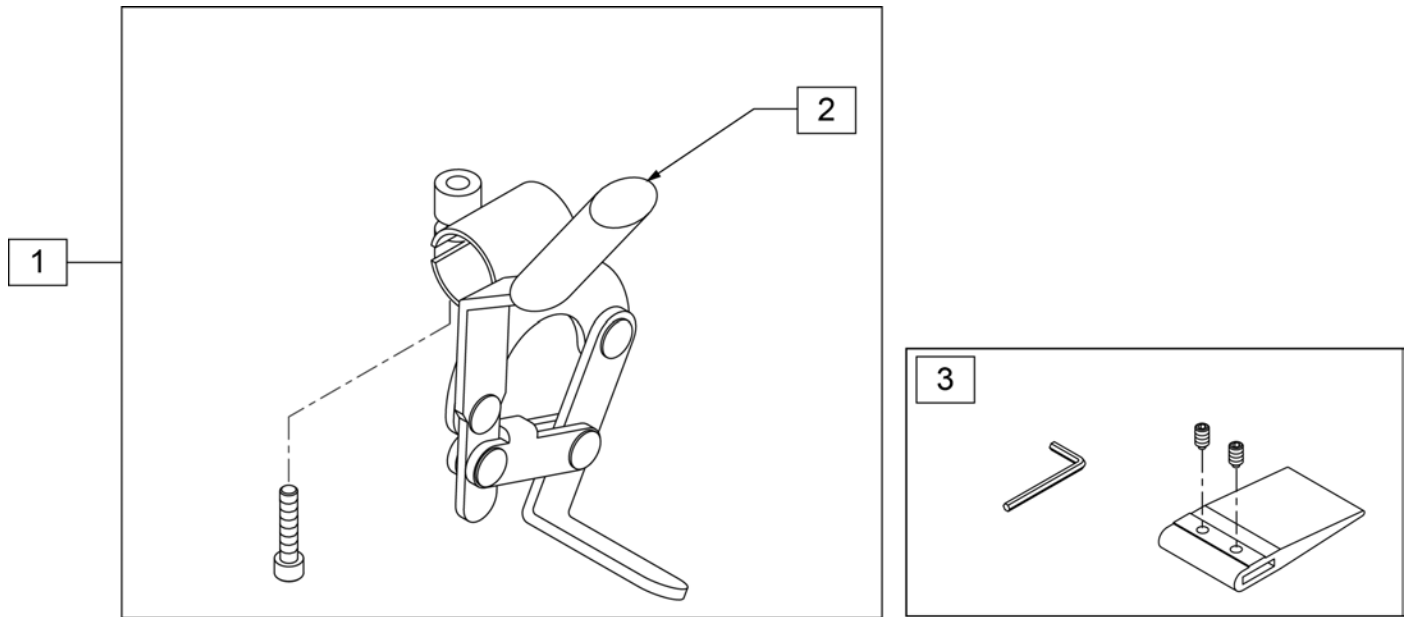

Pos.	Item Number	Description	Remarks
	GEY-1WL	WHEEL LOCK	PAIR
2	GEY-BRAKEHD	BRAKE HANDLE	
3	GEY-WDG	AMPUTEE WHEEL LOCK EXTENSION	MUST BE USED W/ AMPUTEE AXLE PLATE

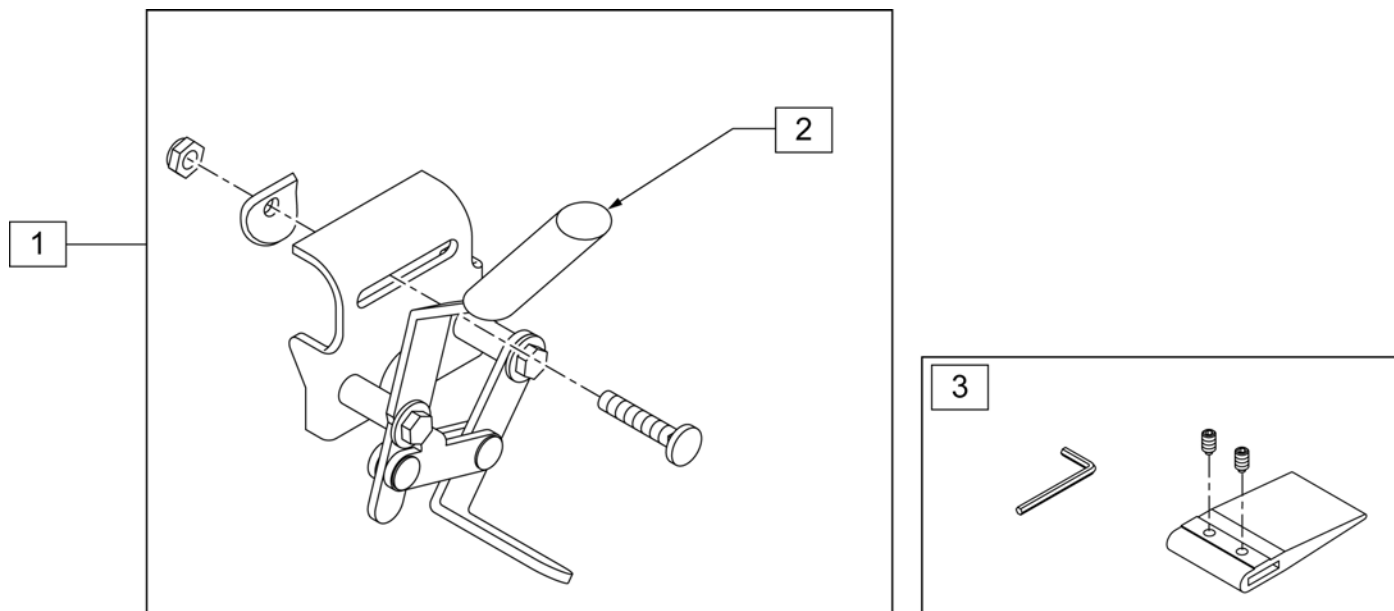


Pos.	Item Number	Description	Remarks
	GEY-2WL	WHEEL LOCK PUSH-TO-LOCK	PAIR PUSH-TO-LOCK COMES STANDARD FOR CHAIRS W/O ELR'S.
	GEY-2WL1	WHEEL LOCK PULL-TO-LOCK	PAIR PULL-TO-LOCK USE W/ ELR'S
2	GEY-BRAKEHD	BRAKE HANDLE	
3	GEY-WDG	AMPUTEE WHEEL LOCK EXTENSION	MUST BE USED W/ AMPUTEE AXLE PLATE

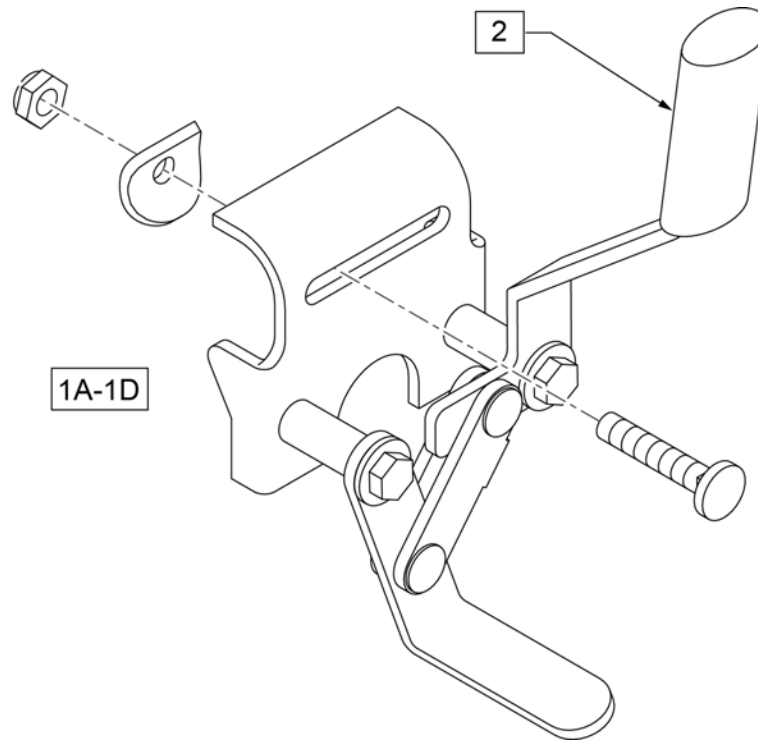


Pos.	Item Number	Description	Remarks
1A	GEY-4WL	WHEEL LOCK PULL-TO-LOCK	Pair- use with ELRs
1B	GEY-4WPLUSH	WHEEL LOCK PUSH-TO-LOCK	Pair-comes std for chairs w/o ELRs
1C	GEY-4RWL	4000R PULL TO LOCK WHEELLOCK	Pair- Use with ELRs & Recline option
1D	GEY-4RWLPUSH	4000R PUSH TO LOCK WHEELLOCK	Pair-Std for chairs w/o ELRs & Recline option
2	GEY-BRAKEHD	BRAKE HANDLE	

NOTE: Super Hemi Kit includes QTY (2) 20" Wheel Assemblies; QTY (2) 6" Casters; QTY (2) Forks; & QTY (2) Amputee Wheel Lock Extensions.

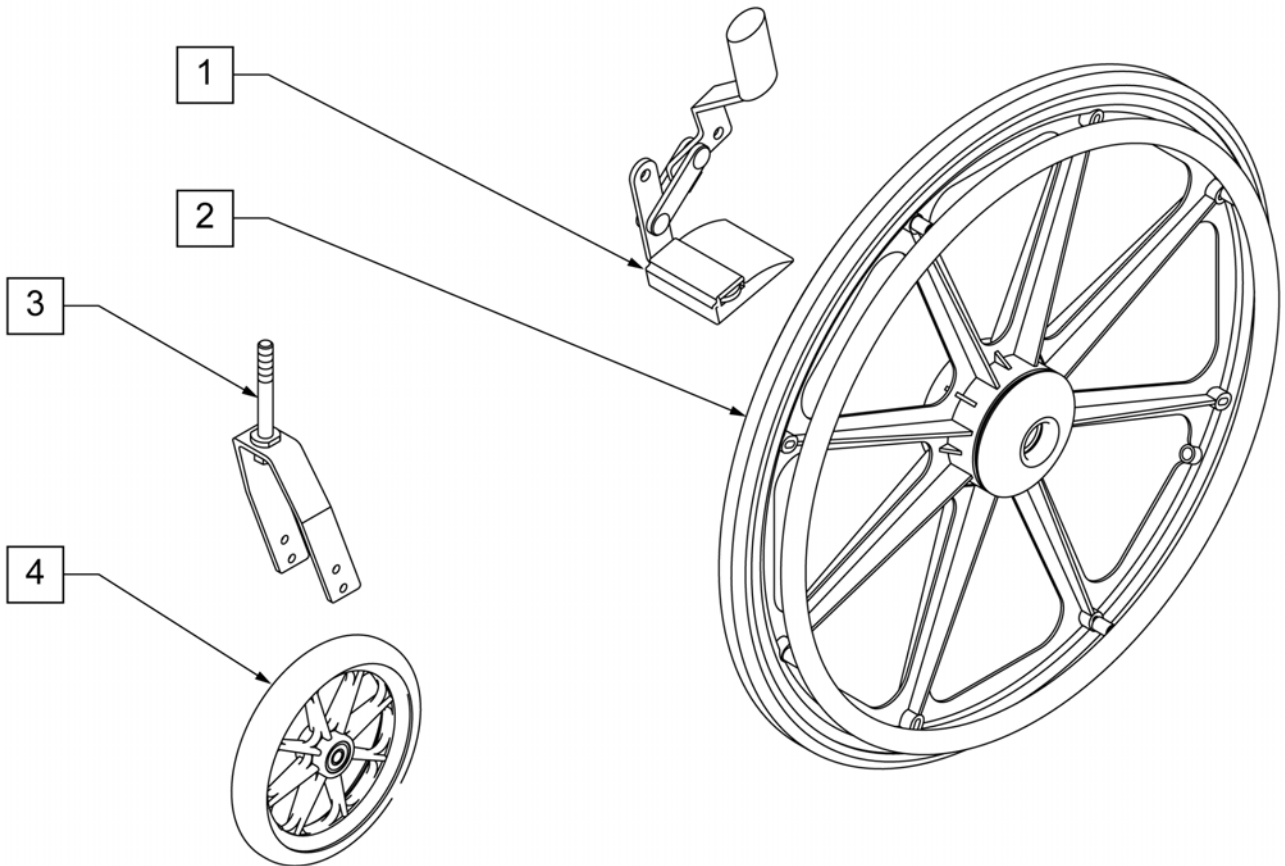

Pos.	Item Number	Description	Remarks
1	GEY-WDG	AMPUTEE WHEEL LOCK EXTENSION	
1-4	GEY-SHEMI	SUPER HEMI KIT	LOWERS TO 15 3/4" SEAT HEIGHT.

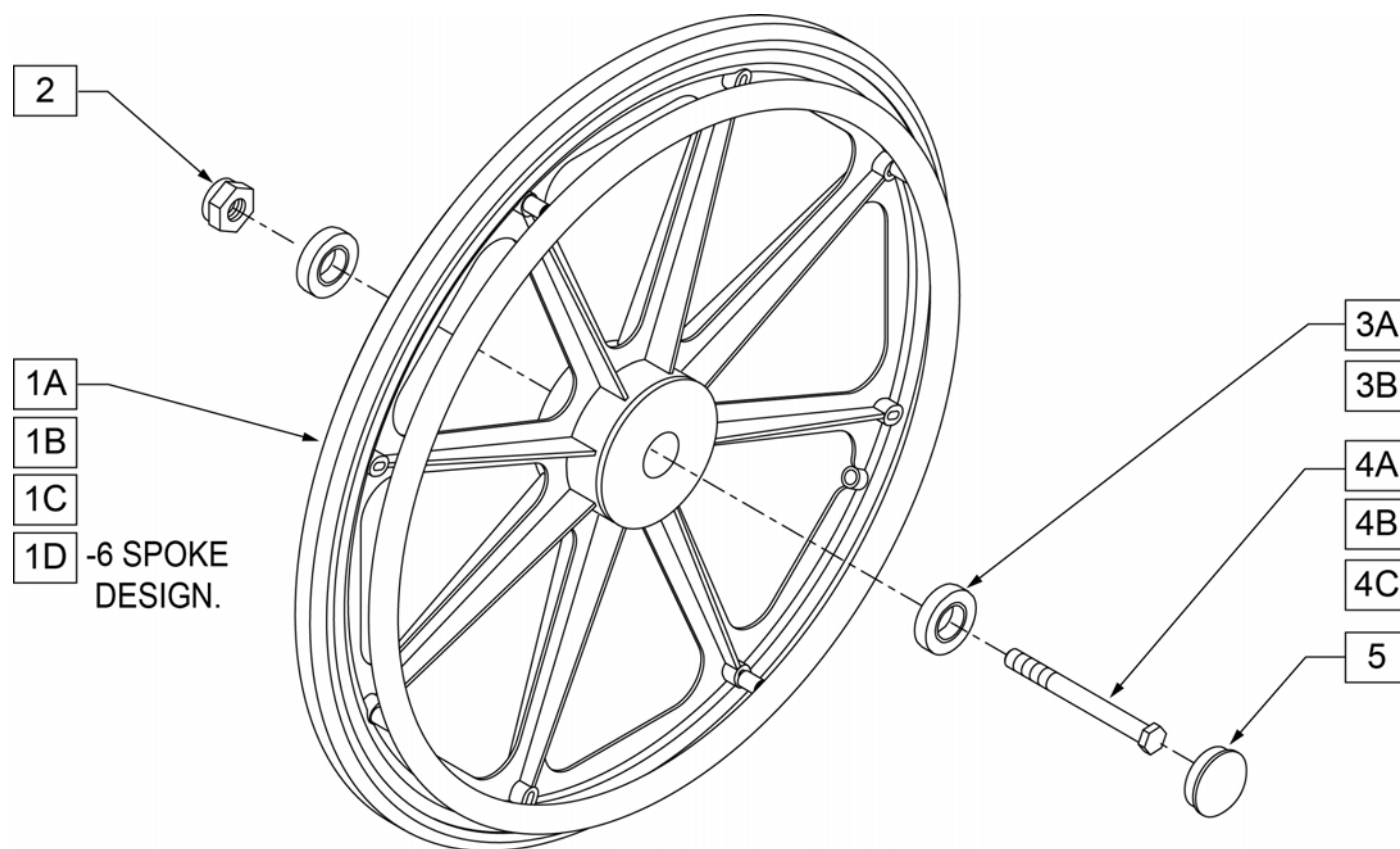


Pos.	Item Number	Description	Remarks
1A	GEY-1WA	1 x 24 REAR WHEEL ASSM, COMP RIM, BLK TIRE	EACH - 1000 2000 3000 - LOW POLY TIRE
1B	GEY-2HDWA	1 x 24 REAR WHEEL ASSM, CHROME RIM, BLK TIRE	EACH - 2000HD EXTRA WIDE (22" & 24" WIDE) - FULL POLY TIRE
1C	GEY-4WA	1 x 24 REAR WHEEL ASSM, ALUM RIM, BLK TIRE	EACH - 4000 - FULL POLY TIRE
1D-5	GEYB-WA	BARIATRIC WHEEL ASSM KIT, CHROME RIM, BLK TIRE	PAIR - 2000HD BARIATRIC (26", 28" & 30" WIDE) - FULL POLY TIRE
3A	GEY-AXLEB	AXLE BEARINGS	PAIR - 1000, 2000, 3000, 4000 - BEARING SIZE 37mm X 12mm
3B	GEY-AXLEBHD	AXLE BEARINGS	PAIR - 2000HD - BEARING SIZE 36mm X 15mm
2, 4A	GEY-AXLE	AXLE AND NUT	PAIR - 1000, 2000, 3000, 4000 - 12mm Diameter, 128 mm long
2, 4B	GEY-AXLEHD	AXLE AND NUT	PAIR - 2000HD - 138.5mm Long - 15mm Diameter
2, 4C	GEY-AXLE4	EC 4000 REAR AXLE KIT	Used w/ 20W EC4000 Only. Includes 2 Axles and Nuts
5	GEY-CAP	AXLE DUST CAPS-ALL SERIES	PAIR

[10/2011]

NOTE: TOOLS REQUIRED TO REMOVE AXLE ARE 22MM WRENCH AND 19MM WRENCH.

NOTE: THERE IS A 1.3" GAP BETWEEN THE WHEEL AND HANDRIM.

NOTE: Super Hemi Kit includes QTY (2) 20" Wheel Assemblies; QTY (2) 6" Casters; QTY (2) Forks; & QTY (2) Amputee Wheel Lock Extensions.

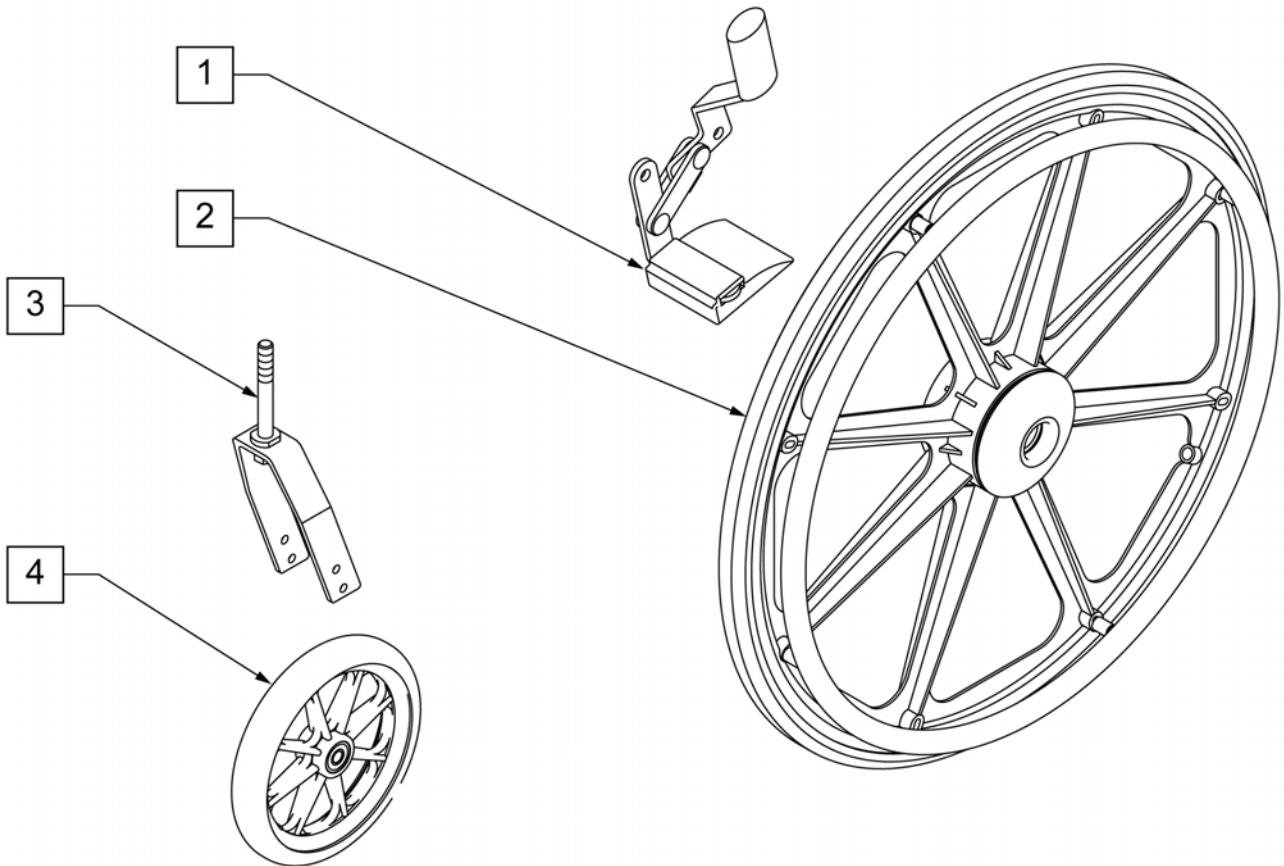


Pos.	Item Number	Description	Remarks
1	GEY-WDG	AMPUTEE WHEEL LOCK EXTENSION	
1-4	GEY-SHEMI	SUPER HEMI KIT	LOWERS TO 15 3/4" SEAT HEIGHT.

NOTE: PART NUMBERS BELOW INCLUDE 2" SEAT EXTENSION AND UPHOLSTERY.

NOTE: THE 2" SEAT EXTENSION IS NO LONGER OFFERED ON 4000 MODEL CHAIRS THAT ARE 20" WIDE.

NOTE: THE 2" SEAT EXTENSION DOES NOT WORK WITH CHAIRS EQUIPPED WITH FIXED ARMREST.


NOTE: IN ORDER TO ADD THE 2" SEAT EXTENSION TO A CHAIR EQUIPPED WITH ELR'S THEN LOW PROFILE ELR'S MUST BE UTILIZED.

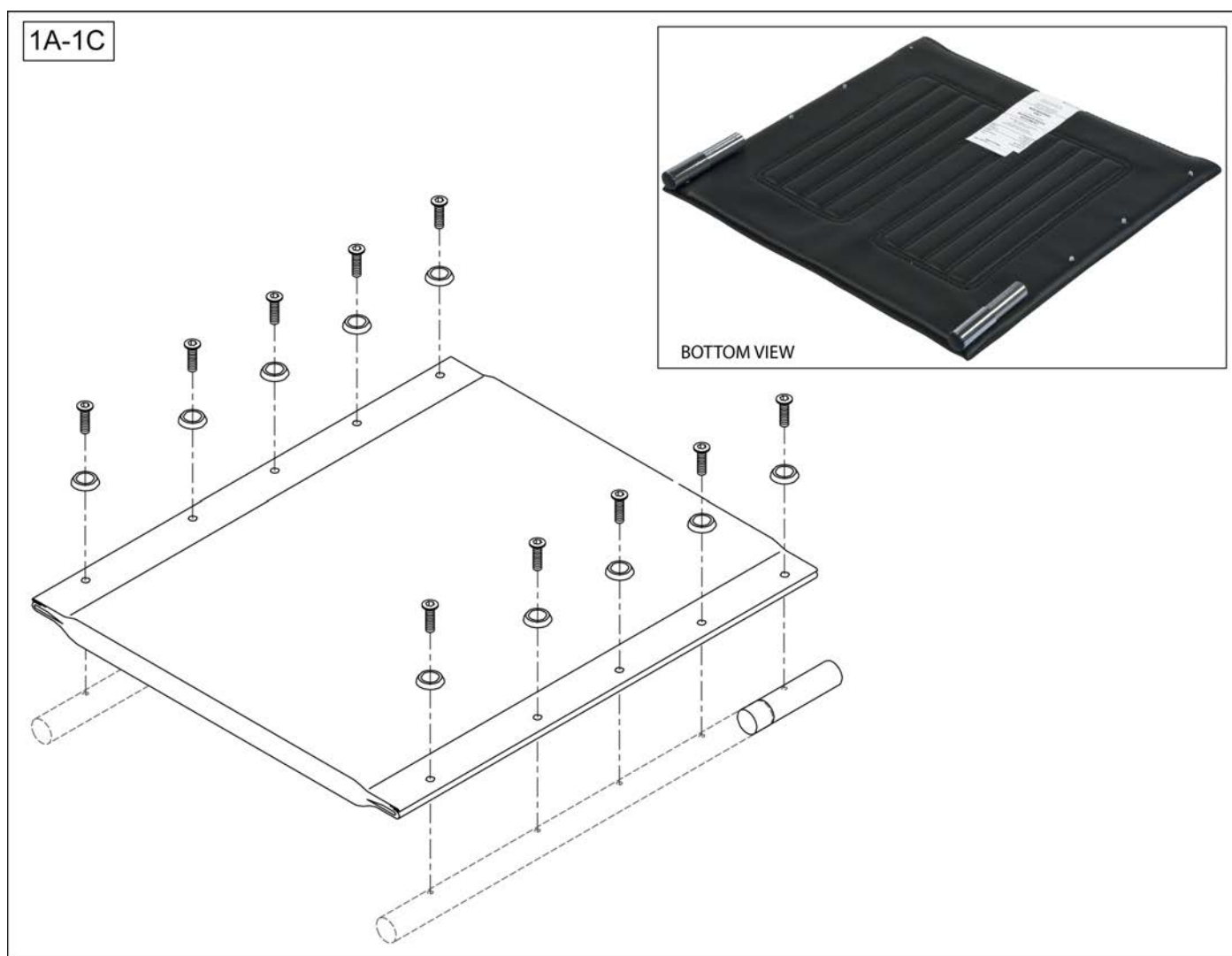


Pos.	Item Number	Description	Remarks
>>>>		4000 SERIES	
	GEY-4SREXT16	2" SEAT EXTENSION ASSM FOR 16" WIDE CHAIRS	4000 SERIES NYLON
	GEY-4SREXT18	2" SEAT EXTENSION ASSM FOR 18" WIDE CHAIRS	4000 SERIES NYLON
>>>>		2000 SERIES	
	GEY-2SREXT16	2" SEAT EXTENSION ASSM FOR 16" WIDE CHAIRS	2000 SERIES VINYL
	GEY-2SREXT18	2" SEAT EXTENSION ASSM FOR 18" WIDE CHAIRS	2000 SERIES VINYL
	GEY-2SREXT20	2" SEAT EXTENSION ASSM FOR 20" WIDE CHAIRS	2000 SERIES VINYL

Note: Burgundy and Blue upholstery is no longer available and must be replaced with Black upholstery.

Note: Prior to August 2010, chairs used M5 screws. Chairs now use M6 screws. All upholstery ships with M6 screws assembled to seat, and M5 screws included with back hardware.

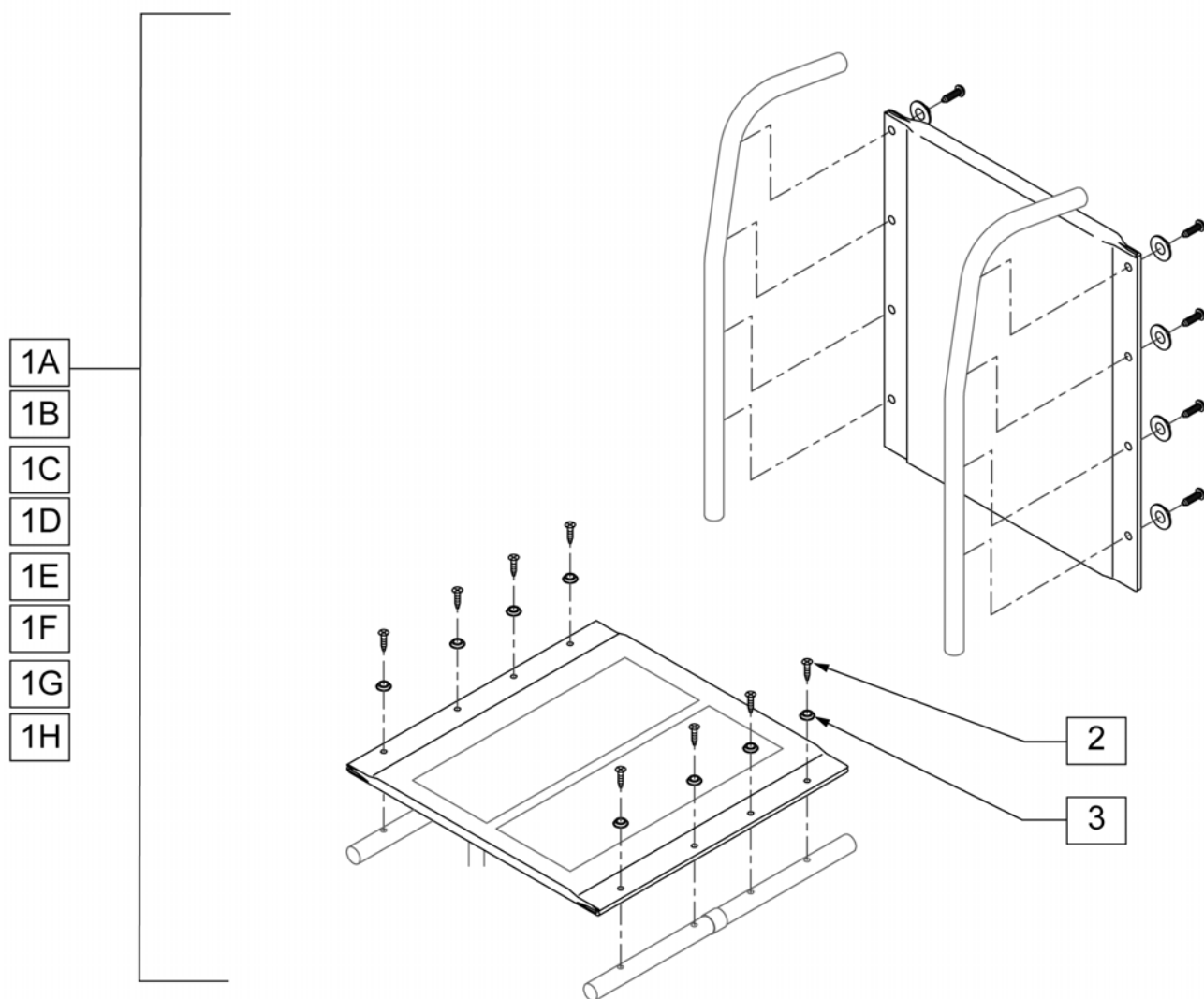


Pos.	Item Number	Description	Remarks
1A	GEY-116UP	16 1000 BLACK VINYL UPHOLSTERY	1000 - 16 WIDE
1A	GEY-118UP	18 1000 BLACK VINYL UPHOLSTERY	1000 - 16 WIDE
1B	GEY-216UP	16 2000 BLACK VINYL UPHOLSTERY	2000 - 16 WIDE BLACK
1C	GEY-218UP	18 2000 BLACK VINYL UPHOLSTERY	2000 - 18 WIDE BLACK
1D	GEY-20RRUP	20 2000 BLACK VINYL UPHOLSTERY	2000 - 20 WIDE BLACK
1E	GEY-22HDUP	22 2000 BLACK VINYL UPHOLSTERY	2000HD - 22 WIDE BLACK
1F	GEY-24HDUP	24 2000 BLACK VINYL UPHOLSTERY	2000HD - 24 WIDE BLACK
1G	GEYB-126UP	26 BLACK UPHOLSTERY	2000HD BARIATRIC - 26 WIDE
1G	GEYB-128UP	28 BLACK UPHOLSTERY	2000HD BARIATRIC - 28 WIDE
1G	GEYB-130UP	30 BLACK UPHOLSTERY	2000HD BARIATRIC - 30 WIDE
1H	GEY-316UP	16 3000 BLACK NYLON UPHOLSTERY	3000 - 16 WIDE
1H	GEY-318UP	18 3000 BLACK NYLON UPHOLSTERY	3000 - 18 WIDE
1H	GEY-320UP	20 3000 BLACK NYLON UPHOLSTERY	3000 - 20 WIDE
2, 3	GEY-UPHHW	UPHOLSTERY HARDWARE	

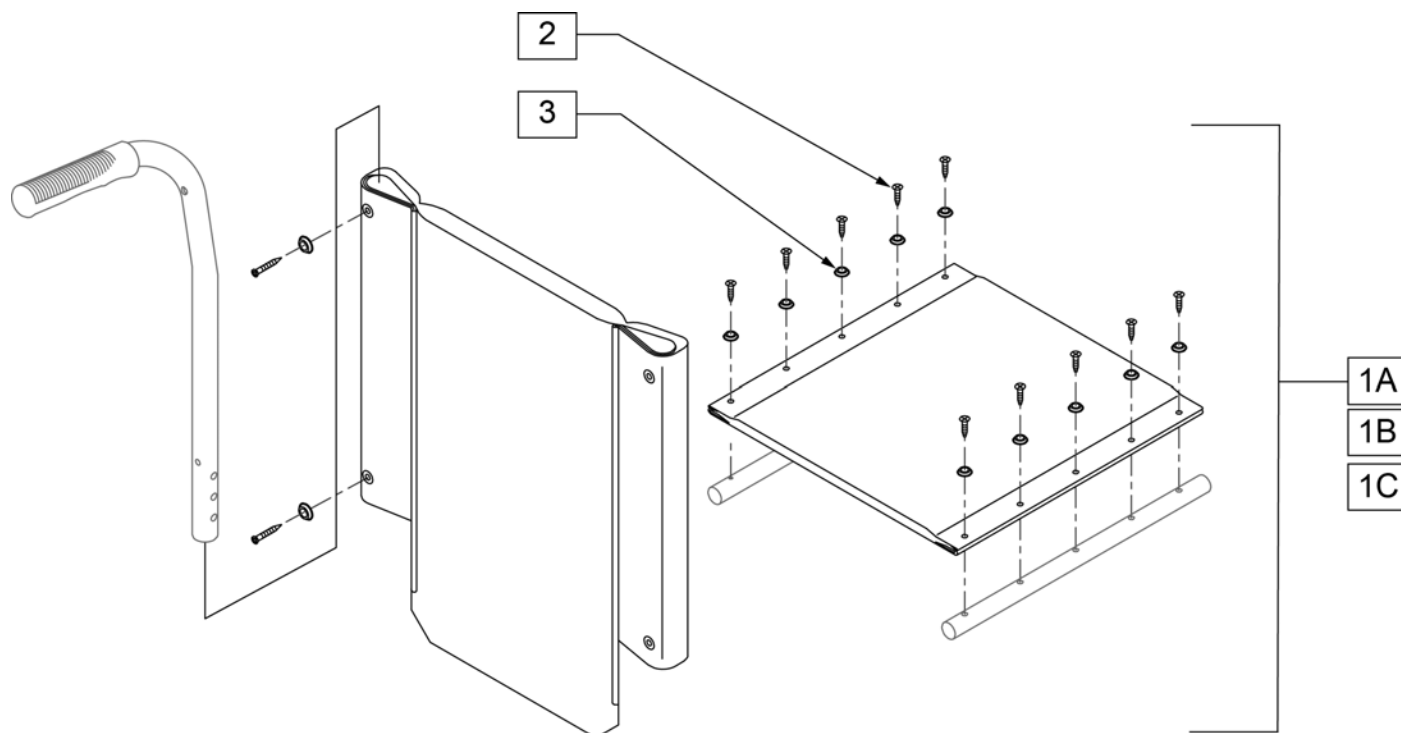


Pos.	Item Number	Description
1A	GEY-416UP	16" BLACK NYLON UPHOLSTERY
1B	GEY-418UP	18" BLACK NYLON UPHOLSTERY
1C	GEY-420UP	20" BLACK NYLON UPHOLSTERY
2, 3	GEY-UPHHW	UPHOLSTERY HARDWARE

Note: Burgundy and Blue upholstery is no longer available and must be replaced with Black upholstery.

Note: Prior to August 2010, chairs used M5 screws. Chairs now use M6 screws. All upholstery ships with M6 screws assembled to seat, and M5 screws included with back hardware.

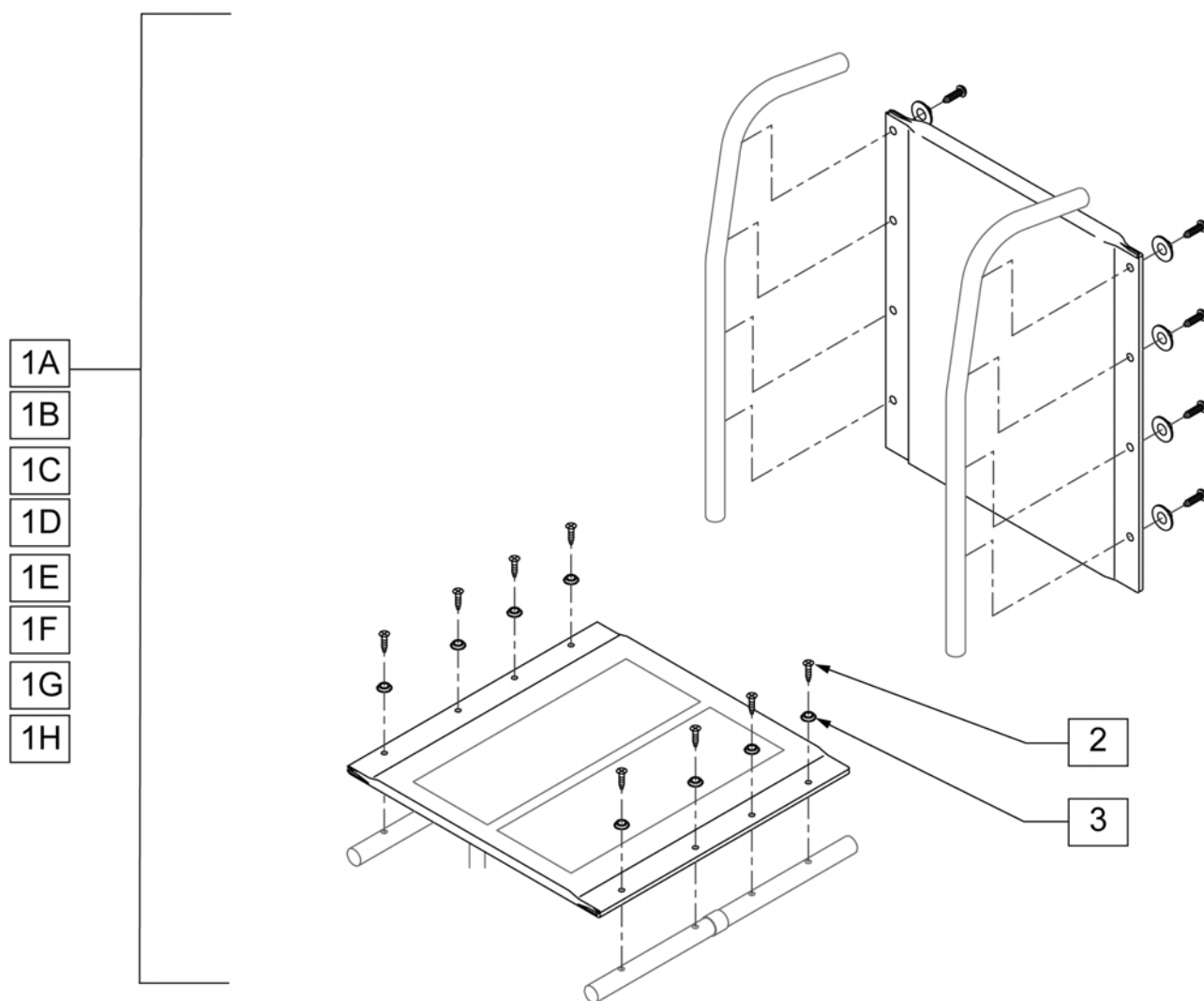


Pos.	Item Number	Description	Remarks
1A	GEY-116UP	16 1000 BLACK VINYL UPHOLSTERY	1000 - 16 WIDE
1A	GEY-118UP	18 1000 BLACK VINYL UPHOLSTERY	1000 - 16 WIDE
1B	GEY-216UP	16 2000 BLACK VINYL UPHOLSTERY	2000 - 16 WIDE BLACK
1C	GEY-218UP	18 2000 BLACK VINYL UPHOLSTERY	2000 - 18 WIDE BLACK
1D	GEY-20RRUP	20 2000 BLACK VINYL UPHOLSTERY	2000 - 20 WIDE BLACK
1E	GEY-22HDUP	22 2000 BLACK VINYL UPHOLSTERY	2000HD - 22 WIDE BLACK
1F	GEY-24HDUP	24 2000 BLACK VINYL UPHOLSTERY	2000HD - 24 WIDE BLACK
1G	GEYB-126UP	26 BLACK UPHOLSTERY	2000HD BARIATRIC - 26 WIDE
1G	GEYB-128UP	28 BLACK UPHOLSTERY	2000HD BARIATRIC - 28 WIDE
1G	GEYB-130UP	30 BLACK UPHOLSTERY	2000HD BARIATRIC - 30 WIDE
1H	GEY-316UP	16 3000 BLACK NYLON UPHOLSTERY	3000 - 16 WIDE
1H	GEY-318UP	18 3000 BLACK NYLON UPHOLSTERY	3000 - 18 WIDE
1H	GEY-320UP	20 3000 BLACK NYLON UPHOLSTERY	3000 - 20 WIDE
2, 3	GEY-UPHHW	UPHOLSTERY HARDWARE	

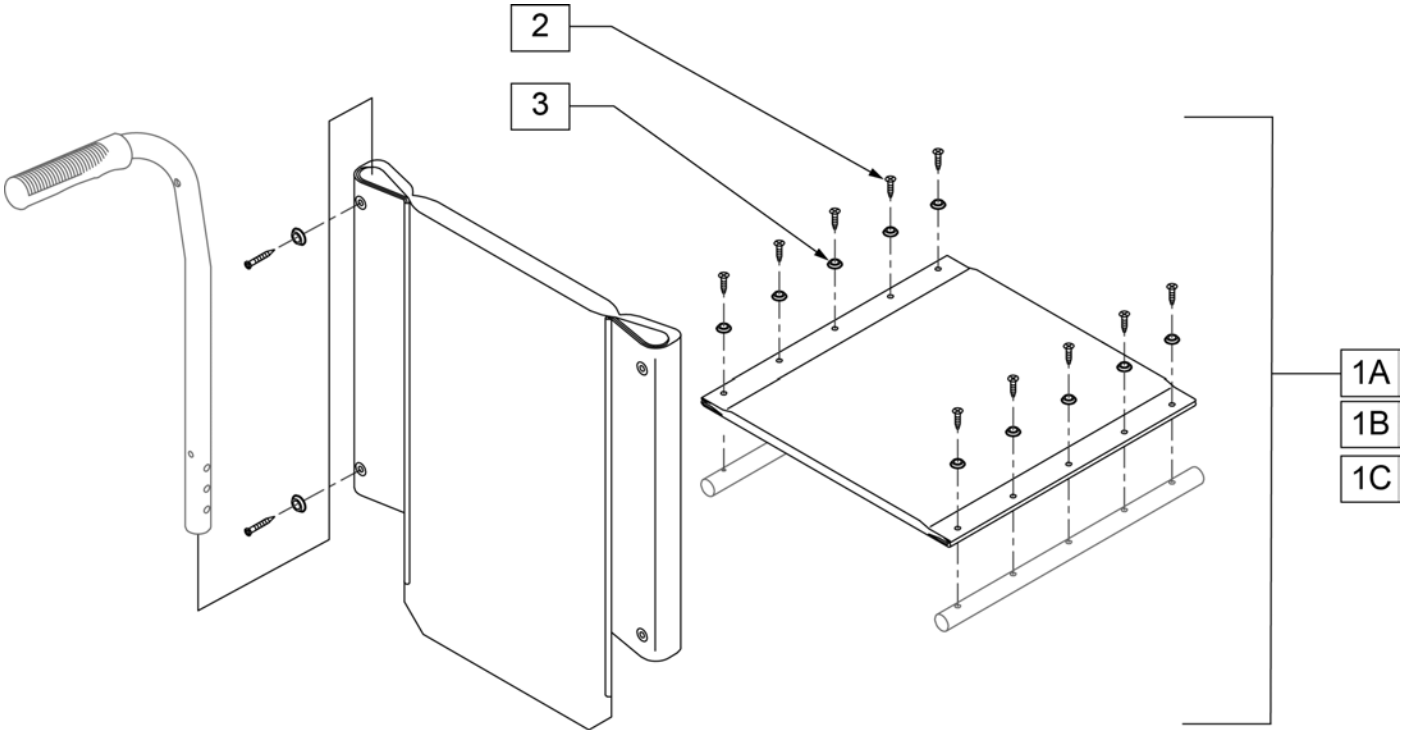


Pos.	Item Number	Description
1A	GEY-416UP	16" BLACK NYLON UPHOLSTERY
1B	GEY-418UP	18" BLACK NYLON UPHOLSTERY
1C	GEY-420UP	20" BLACK NYLON UPHOLSTERY
2, 3	GEY-UPHHW	UPHOLSTERY HARDWARE


NOTE: REPLACEMENT BACK UPHOLSTERY'S ARE NOT AVAILABLE FOR CHAIRS BUILT PRIOR TO S/N A60413860. CONTACT CUSTOMER SERVICE FOR REPLACEMENT INFORMATION.

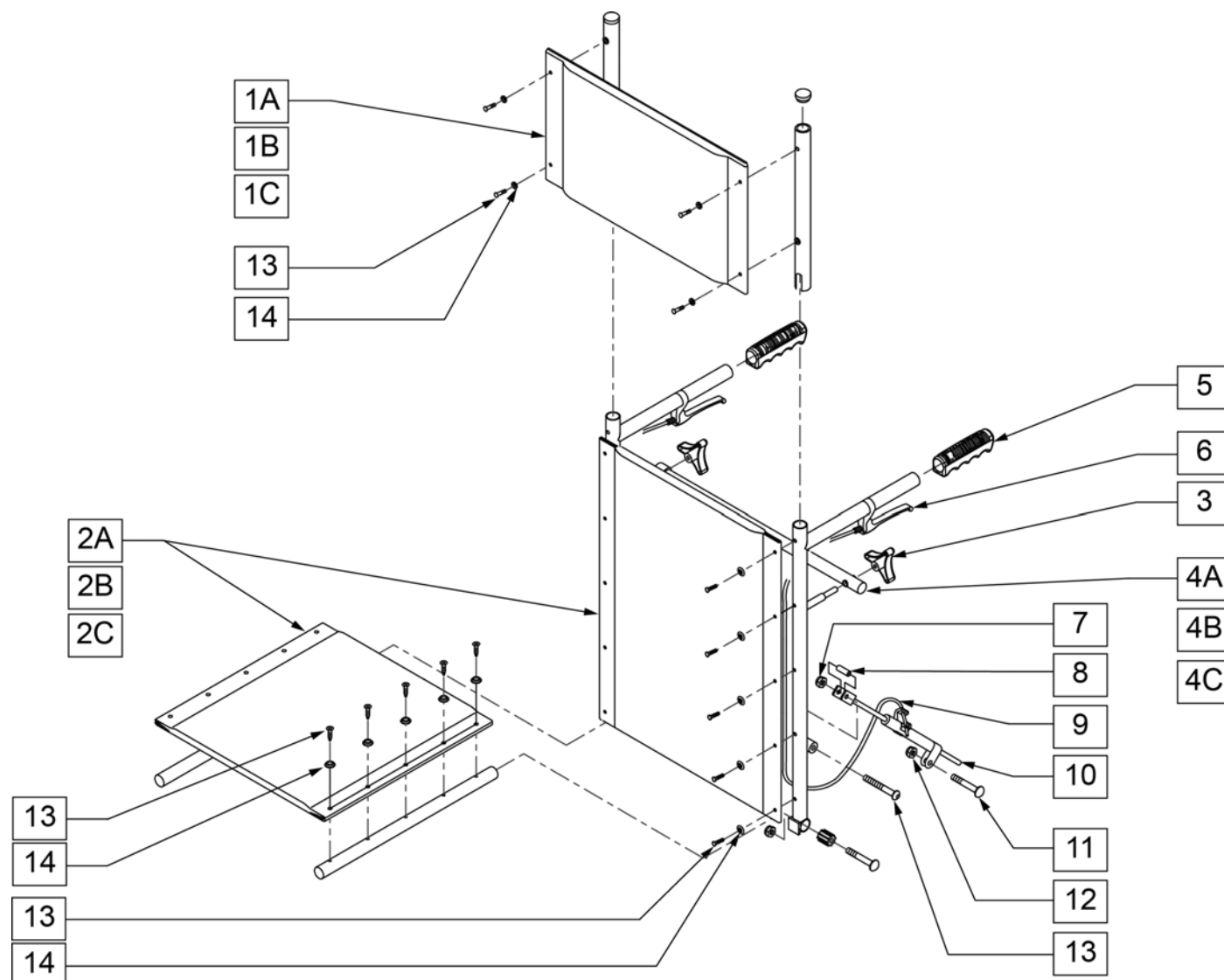

Pos.	Item Number	Description	Remarks
1A, 13-14	GEY-4RHR16	4000 RECLINER HEADREST 16	INCLUDES HEADREST PAD AND MOUNTING TUBES
1B, 13-14	GEY-4RHR18	4000 RECLINER HEADREST 18	INCLUDES HEADREST PAD AND MOUNTING TUBES
1C, 13-14	GEY-4RHR20	4000 RECLINER HEADREST 20	INCLUDES HEADREST PAD AND MOUNTING TUBES
2A, 13-14	GEY-4R16UP	4000 RECLINER UPHOLSTERY 16	
2B, 13-14	GEY-4R18UP	4000 RECLINER UPHOLSTERY 18	
2C, 13-14	GEY-4R20UP	4000 RECLINER UPHOLSTERY 20	
3	GEY-4RKNOB	4000 RECLINER REPLACE KNOB	
4A	GEY-4RSBAR16	4000 RECLINER SPREADER BAR 16	
4B	GEY-4RSBAR	4000 RECLINER SPREADER BAR 18	
4C	GEY-4RSBAR20	4000 RECLINER SPREADER BAR 20	
5	GEY-HAND	BACK CANE HANDLES	
13-14	GEY-UPHHW	UPHOLSTERY HARDWARE	
6-13	GEY-4RAK1	4000 RECLINER ASSM	

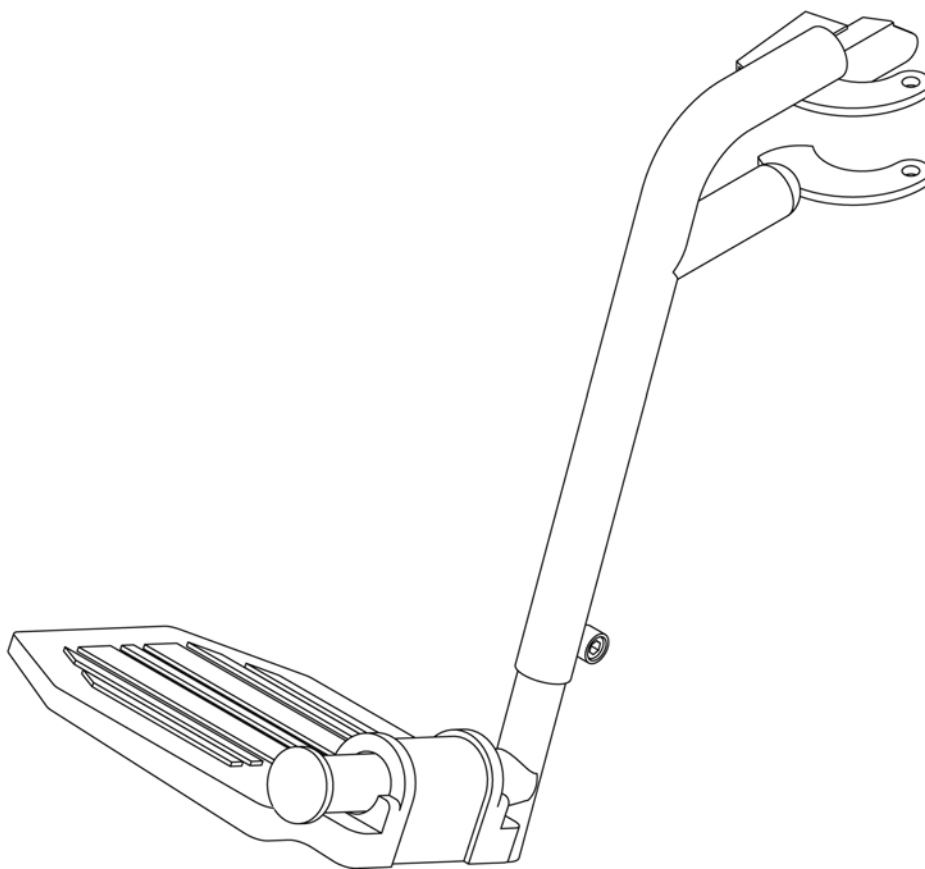


Pos.	Item Number	Description	Remarks
1	GEY-FRC	SWINGAWAY HANGER W/ COMPOSITE FOOTPLATE	USED WITH 2000, 2000HD, 3000, & 4000 - 16", 18" & 20"
1	GEY-FRA	SWINGAWAY HANGER W/ ALUMINUM FOOTPLATE	USED WITH 2000, 2000HD, 3000, & 4000 - 22" & 24"
1	GEYB-FRA	BARIATRIC SWINGAWAY HANGER W/ ALUMINUM FOOTPLATE	USED WITH 2000 HD BARIATRIC ONLY - 26", 28" & 30"

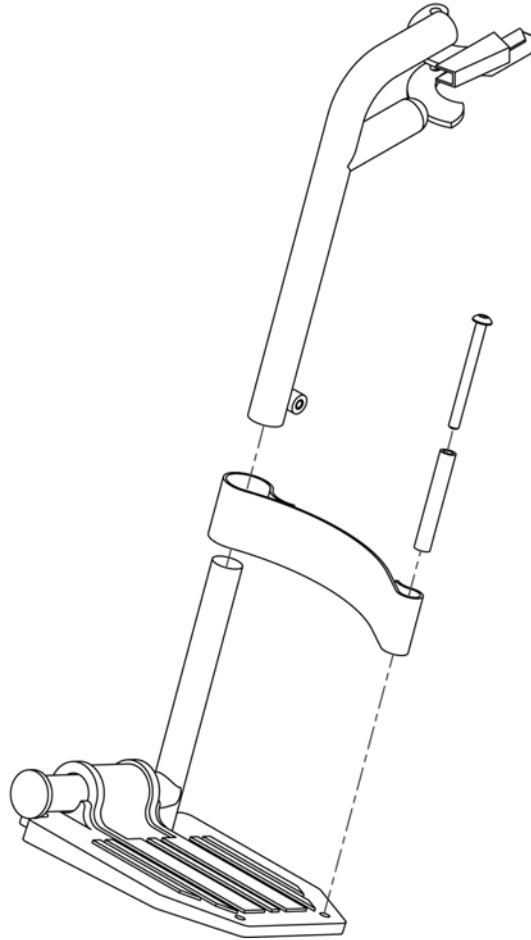


Pos.	Item Number	Description
1	GEY-FRCHL	COMPOSITE FOOTREST W/ HEEL LOOP


NOTE: IN ORDER TO ADD ELR'S TO A CHAIR EQUIPPED WITH A 2" SEAT EXTENSION THEN LOW PROFILE ELR'S MUST BE UTILIZED.

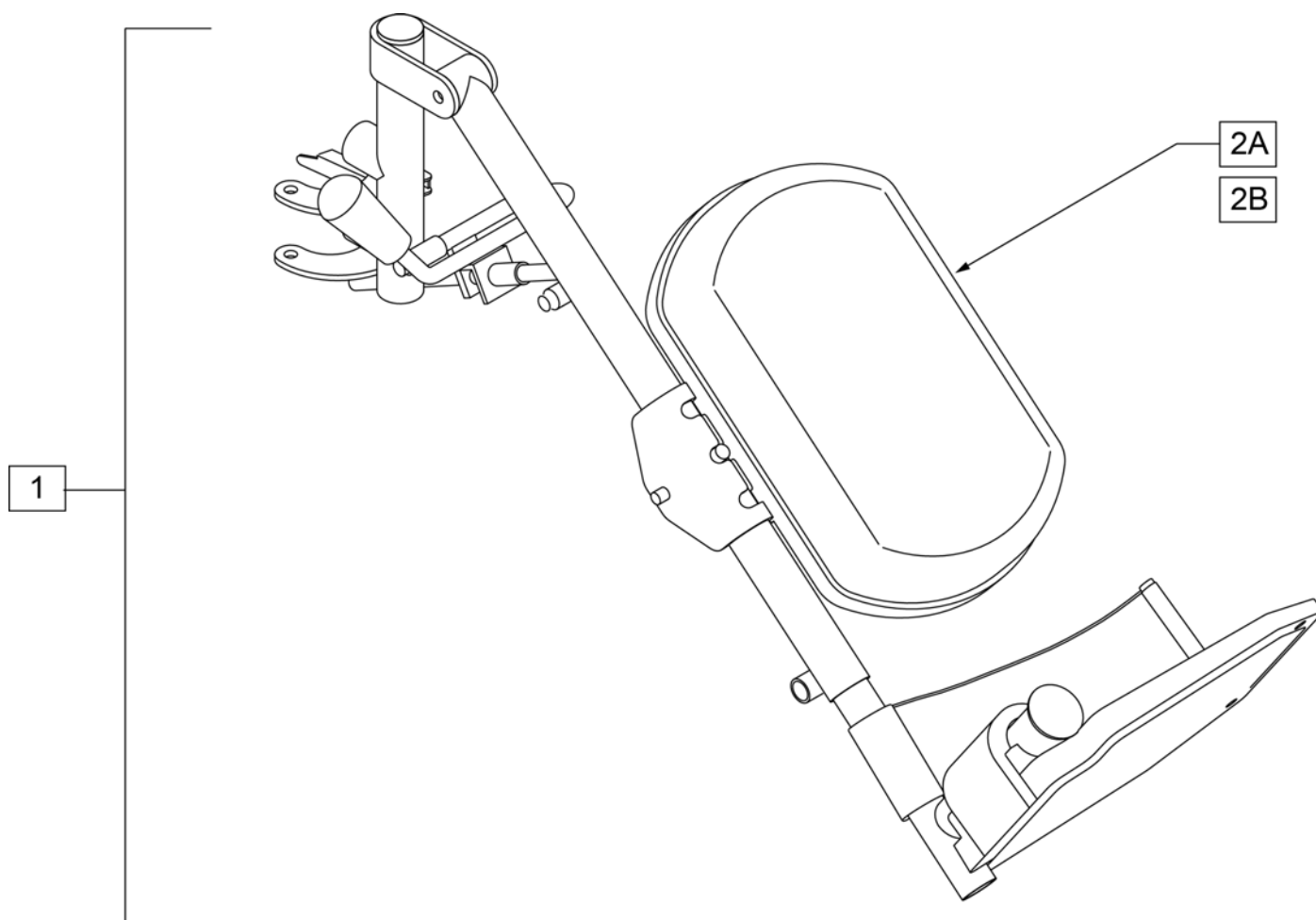

Pos.	Item Number	Description	Remarks
1, 2B	GEY-ELRC	ELEVATING LEGRESTS	PAIR
1, 2A	GEY-ELRCN	K4 COMPOSITE ELRS'S	PAIR
1, 2B	GEYB-ELRA	BARIATRIC ELEVATING LEG RESTS	PAIR - ALUMINUM FOOTPLATE
2A	GEY-CALFN	EC NYLON CALF PAD	
2B	GEY-CALFV	EC VINYL CALF PAD	

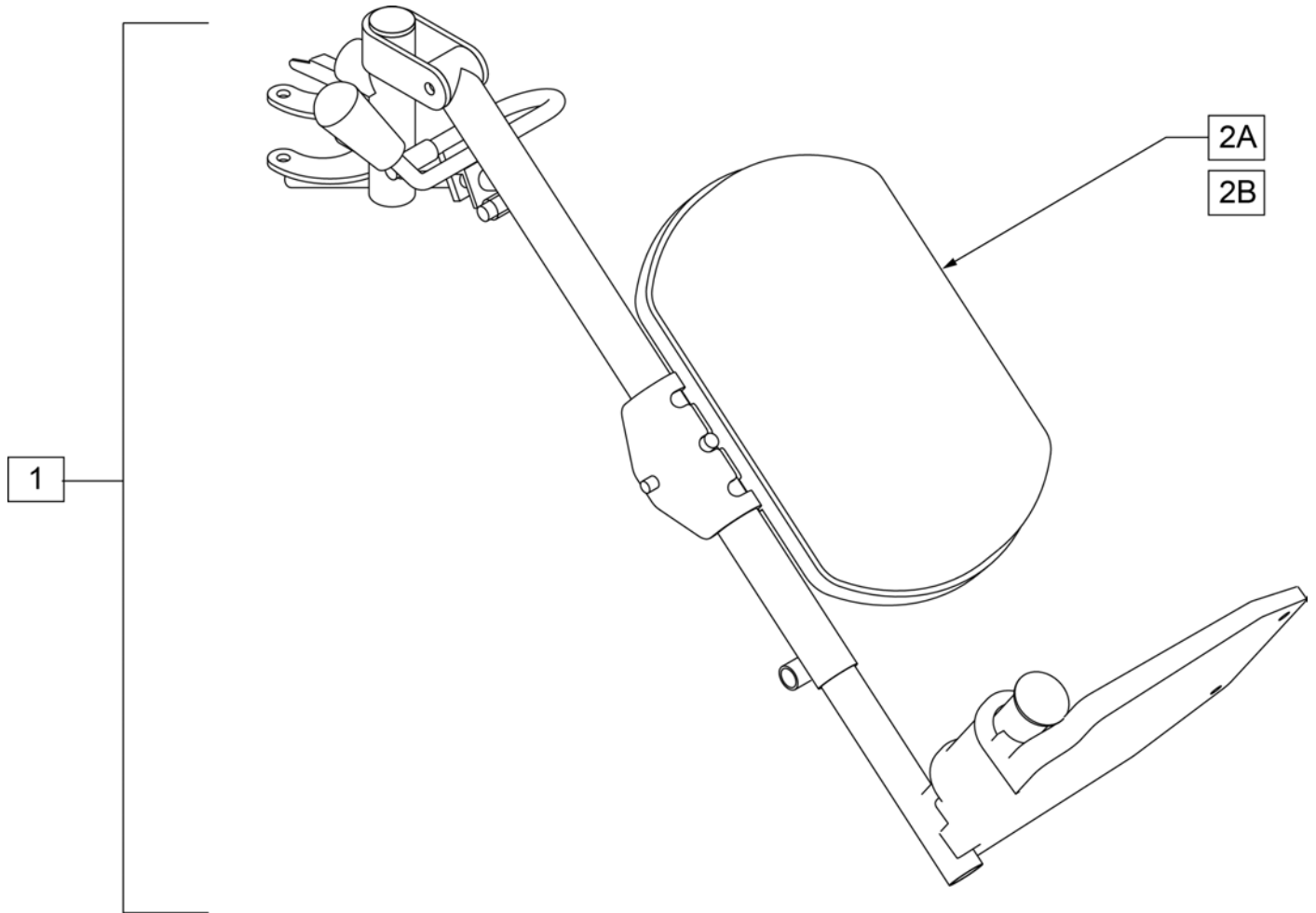

Pos.	Item Number	Description	Remarks
1	GEY-ELRCLP	LOW PROFILE ELR	PAIR TO BE USED W/ 2" SEAT EXT.
2A	GEY-CALFN	EC NYLON CALF PAD	
2B	GEY-CALFV	EC VINYL CALF PAD	

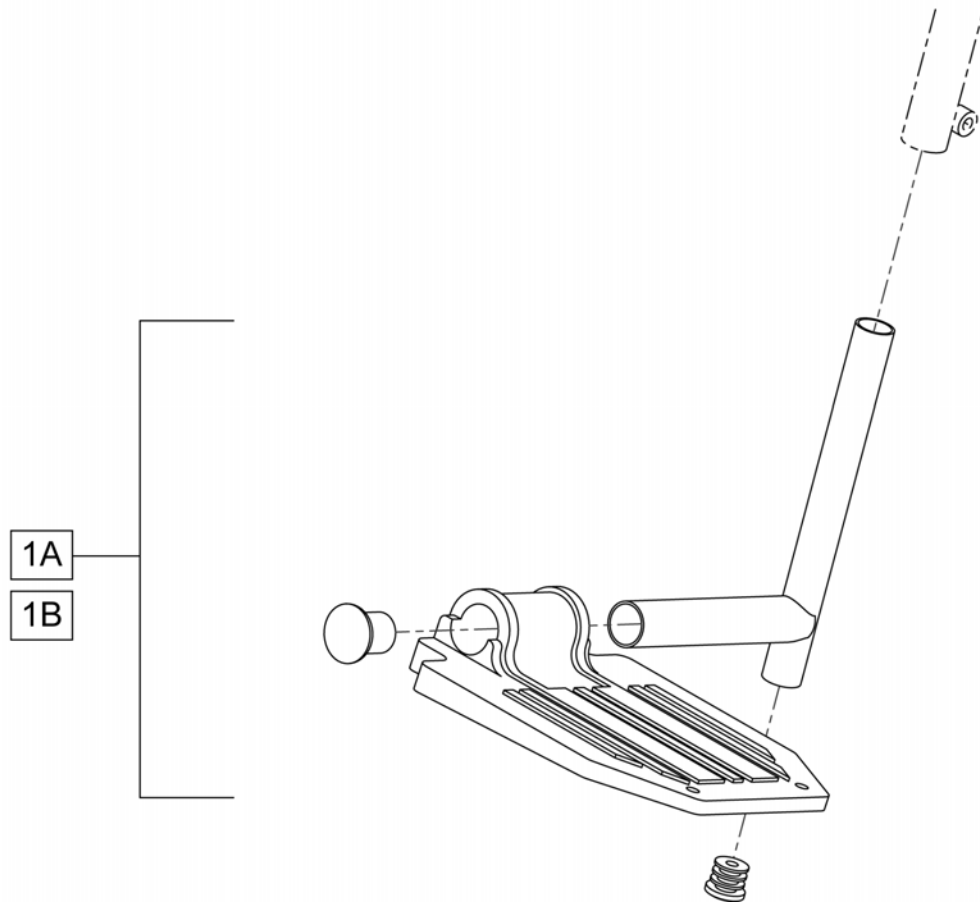


Pos.	Item Number	Description	Remarks
1A	GEY-1FPC	FOOTPLATE COMPOSITE	PAIR
1B	GEY-1FPA	FOOTPLATE ALUMINUM	PAIR




1

Pos.	Item Number	Description
1	GEY-ALR	ARTICULATING LEGREST

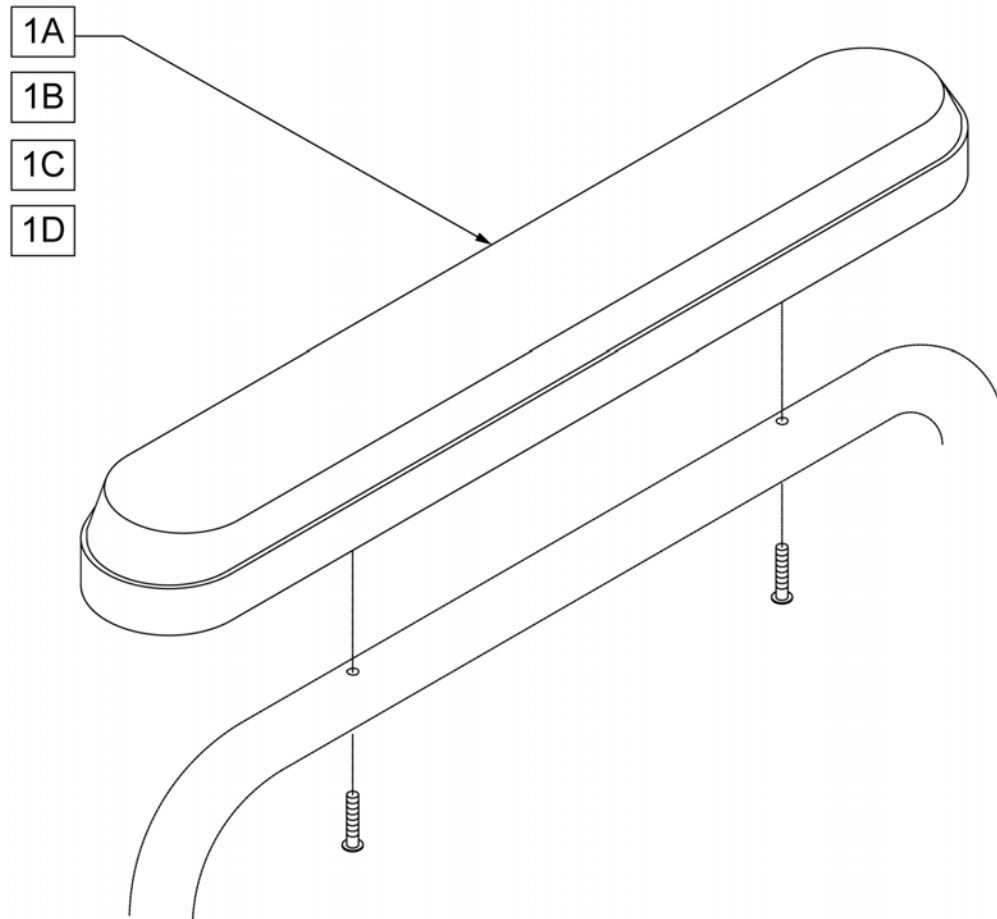


Pos.	Item Number	Description	Remarks
1A	GEY-1TAP	TRANSPORT FULL NYLON ARM PAD	Mounting Hole Spacing 10"-Single Pad
1B	GEY-1AP	1000 FULL PLASTIC ARM PAD	Mounting Hole Spacing 10"-Single Pad
1C	GEY-2AP	2000 SERIES FULL PADDED VINYL	Mounting Hole Spacing 10"-Single Pad
1D	GEY-3AP	3000,4000 SERIES FULL PADDED	Mounting Hole Spacing 10"-Single Pad

NOTE: DISTANCE BETWEEN MOUNTING HOLES IS 5".

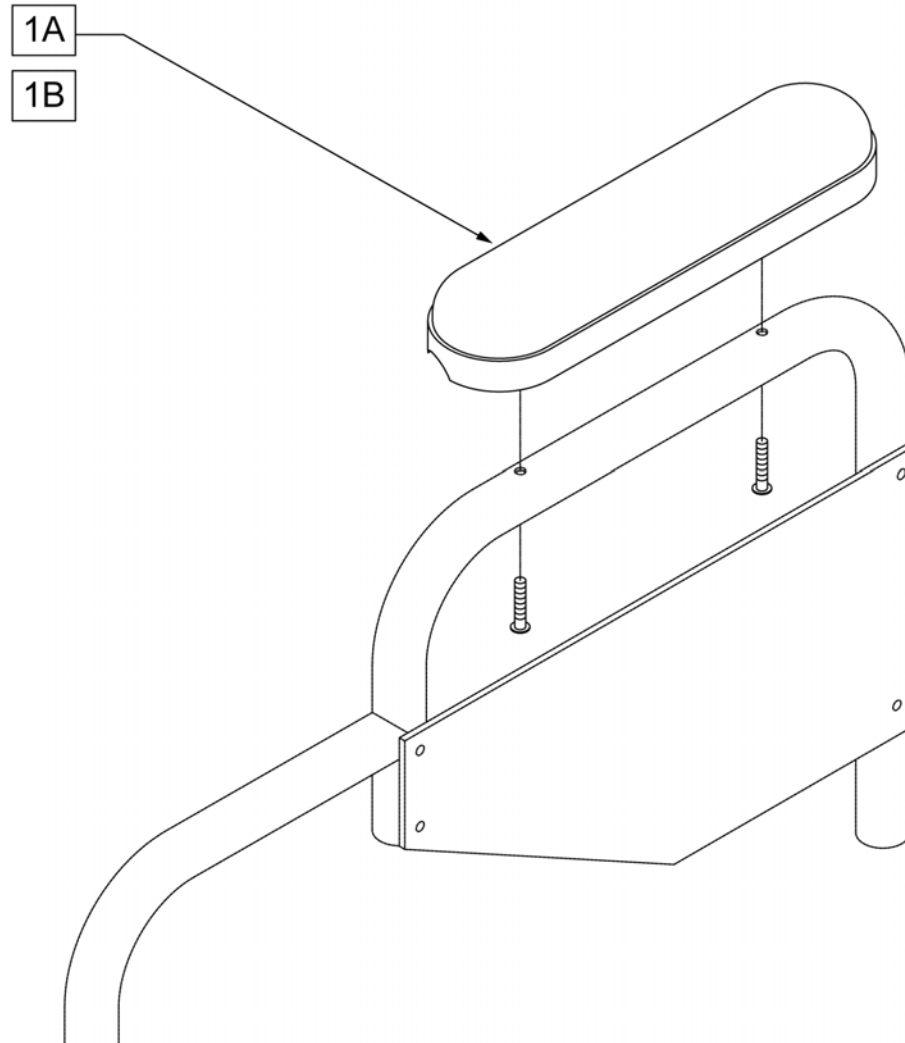


Pos.	Item Number	Description	Remarks
1A	GEY-2APD	DESK PADDED VINYL ARMPAD W/ HDWR 2000, 2000HD, RECLINER	Mounting Hole Spacing 5 1/2"- single pad
1B	GEY-3APD	DESK PADDED NYLON ARMPAD W/ HDWR 3000, 4000	Mounting Hole Spacing 5 1/2"- single pad

NOTE: OVERALL LENGTH OF ARMREST IS 18" AND OVERALL HEIGHT IS 12".

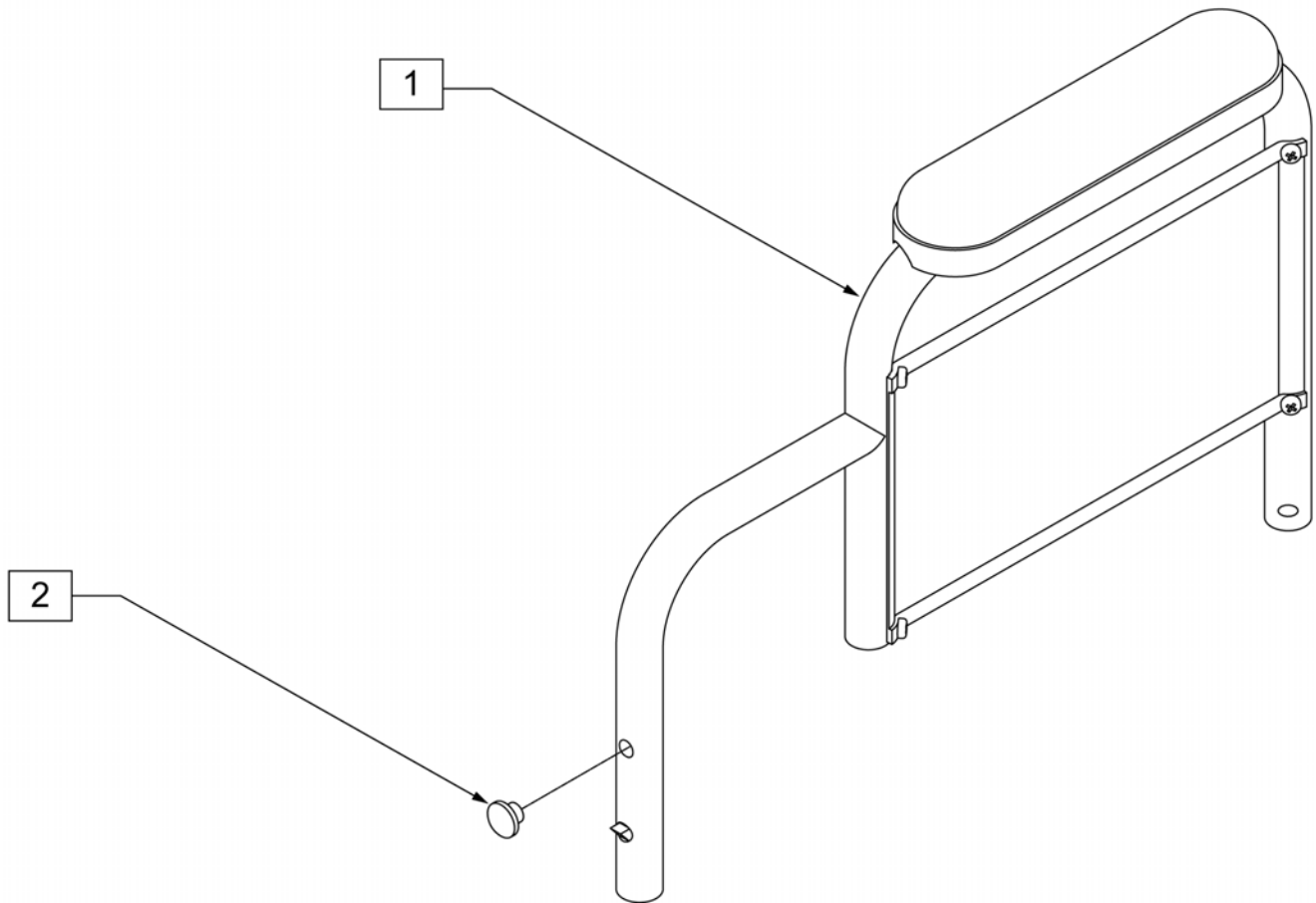


Pos.	Item Number	Description	Remarks
1	GEY-RADP	EC ARMREST DESK PADDED	USE WITH 2000 & 2000HD
1	GEYB-APD	BARIATRIC DESK ARM REST ARMPAD	Pair Armpads 2000HD Bariatric
2	GEY-ARMRL	RELEASE ARM BUTTON	PACK OF 20


NOTE: FULL LENGTH PADDED ARMS WILL NOT WORK WITH 2000HD BARIATIRC MODELS (26", 28", & 30" WIDE CHAIRS).

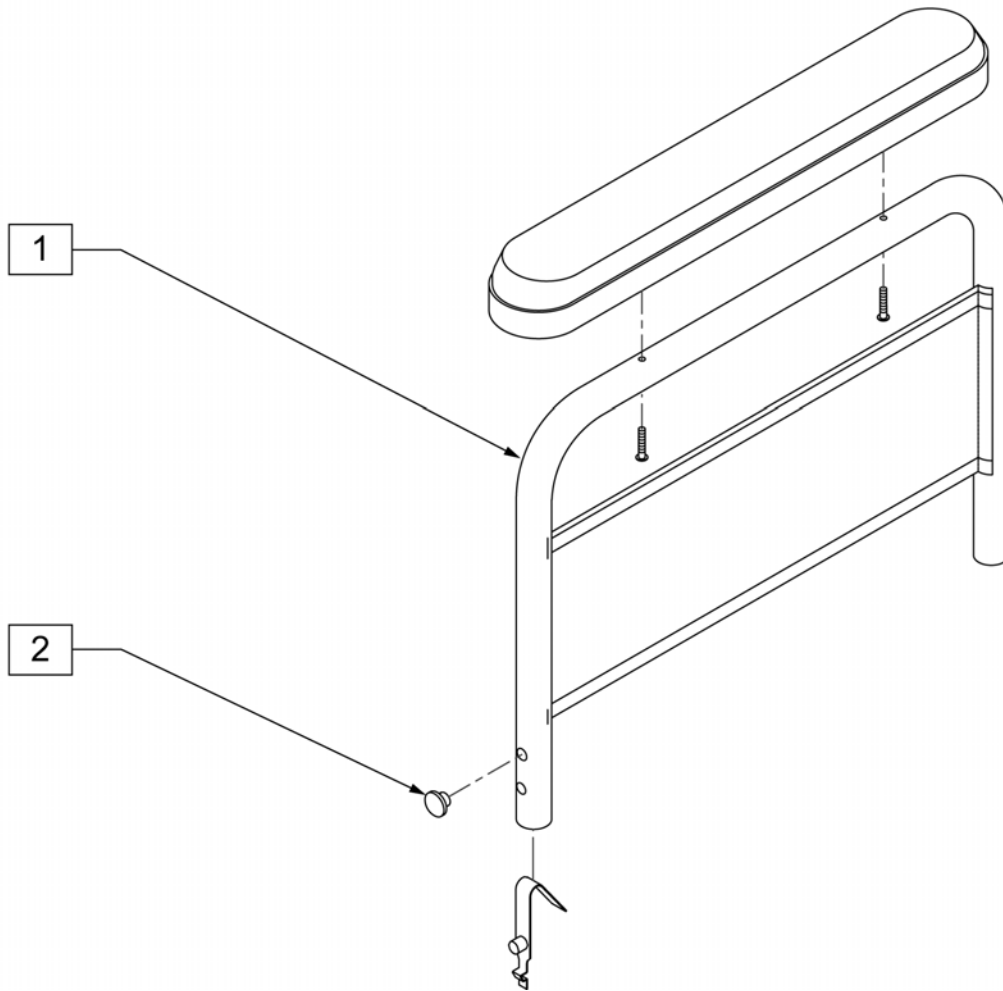


Pos.	Item Number	Description	Remarks
1	GEY-RAFP	EC ARMREST FULL PADDED	USE WITH 2000 AND 2000HD
2	GEY-ARMRL	RELEASE ARM BUTTON	PACK OF 20

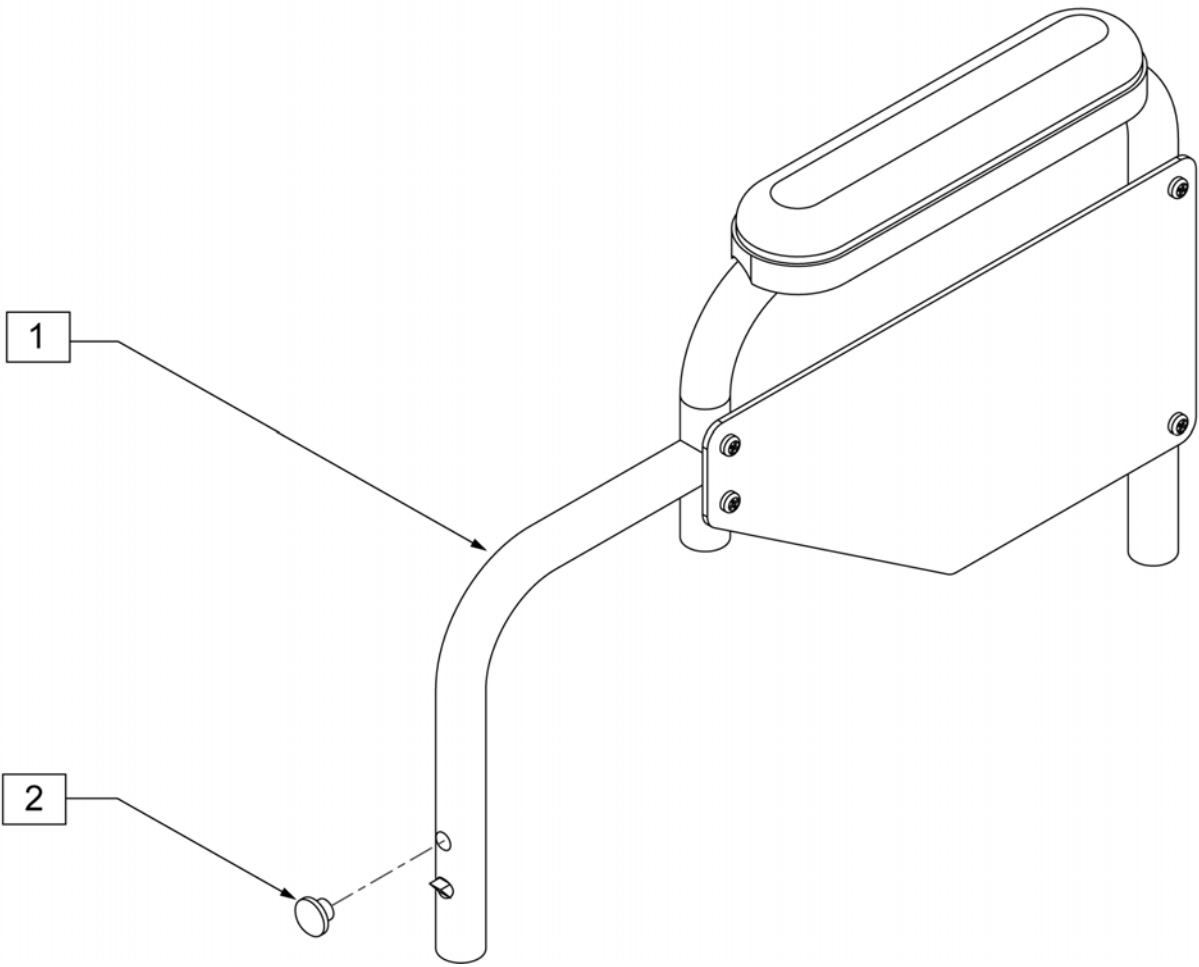


Pos.	Item Number	Description	Remarks
1	GEY-3RADP	ARMREST DESK PADDED NYLON	NYLON
2	GEY-ARMRL	RELEASE ARM BUTTON	PACK OF 20

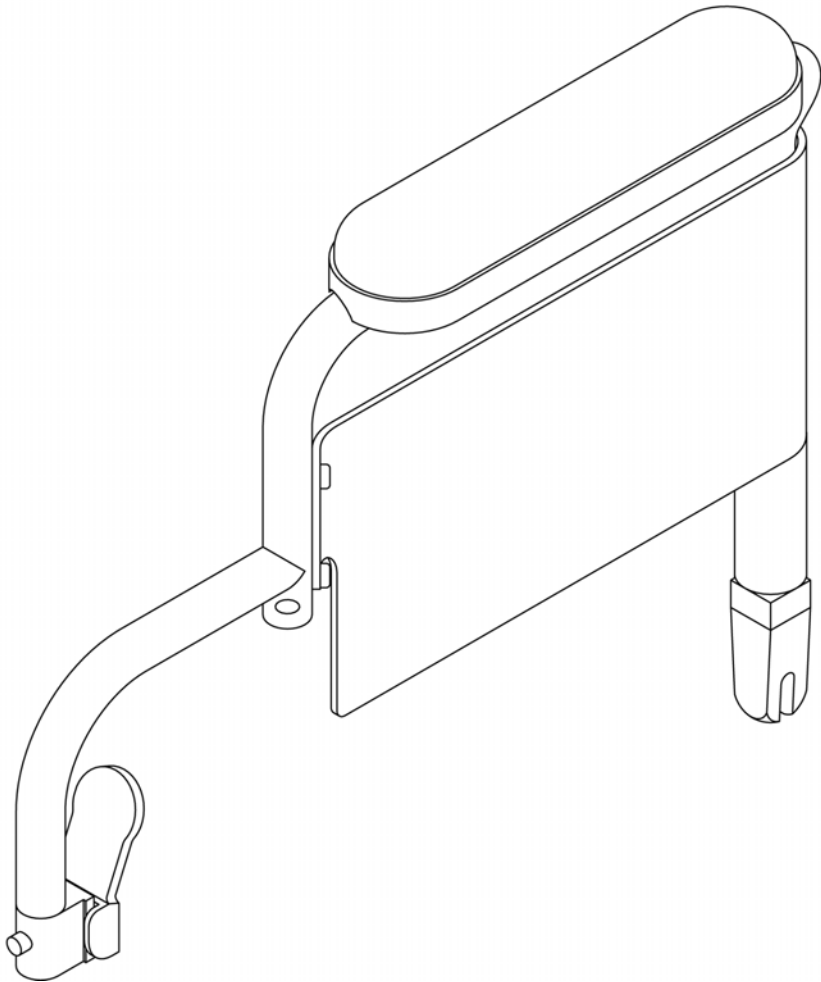


Pos.	Item Number	Description	Remarks
	GEY-4FADP	ARMREST DESK PADDED	NYLON

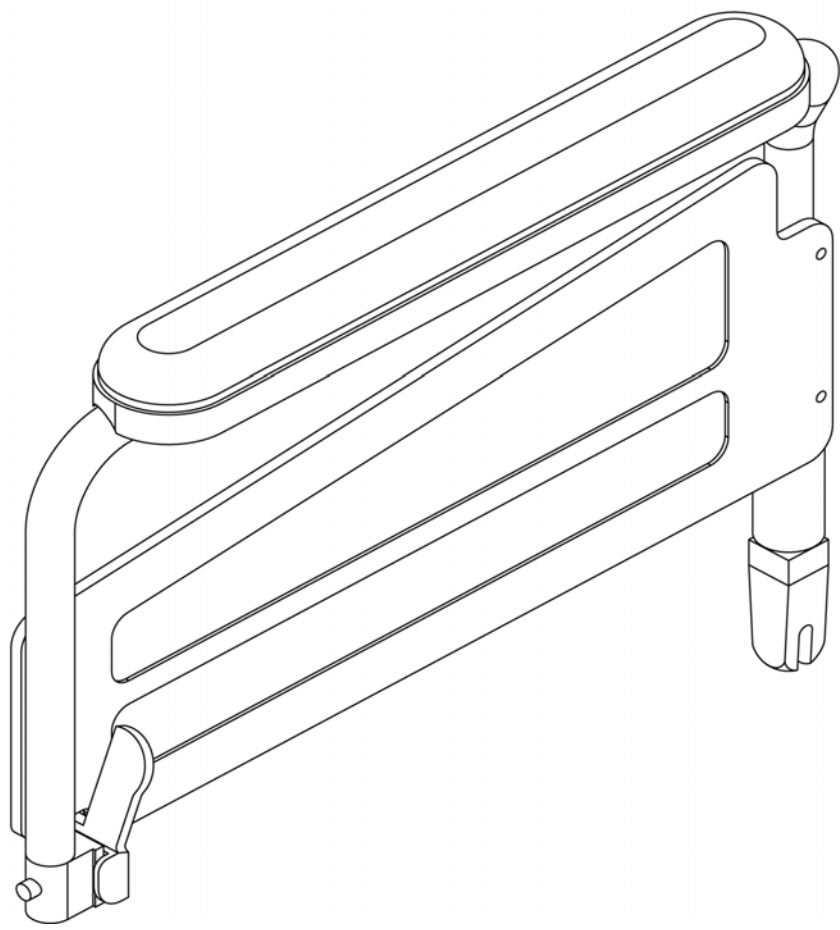


Pos.	Item Number	Description	Remarks
	GEY-4FAFP	ARMREST FULL PADDED	NYLON

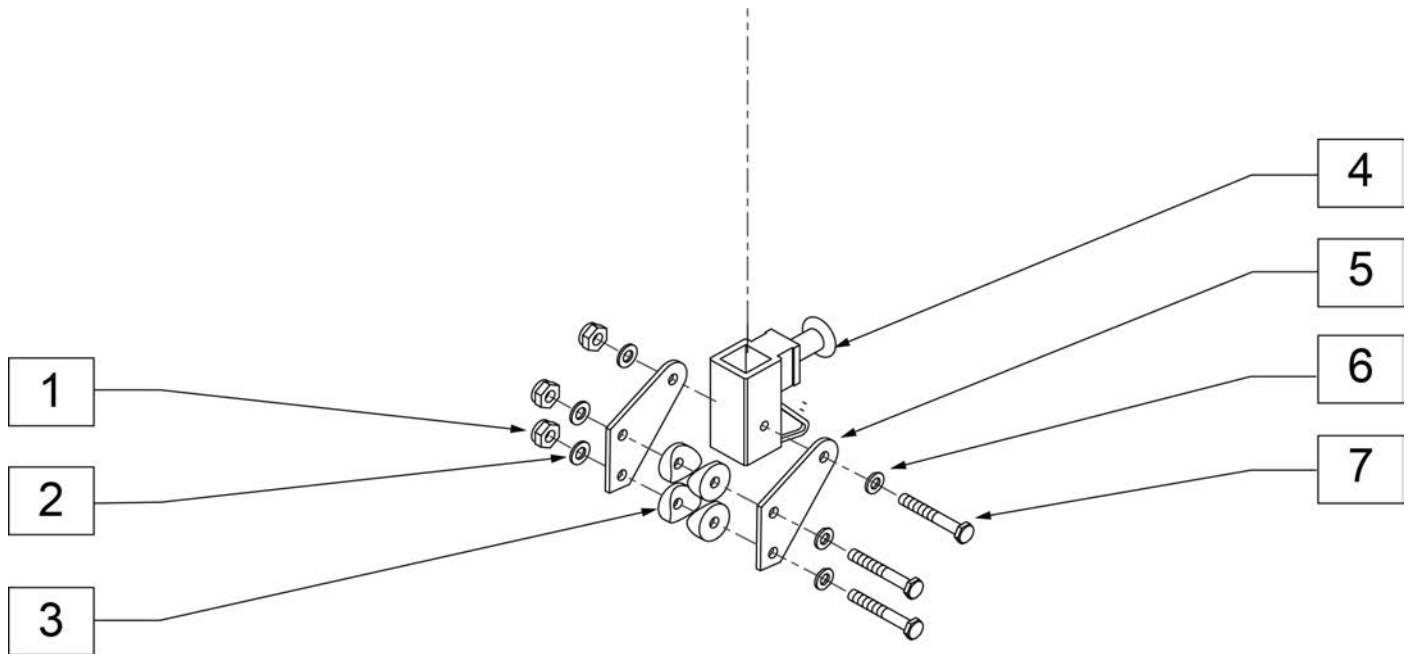


Pos.	Item Number	Description	Remarks
1-7	GEY-4ARR	4000 ARM RECEIVER W/ MOUNTS	PAIR

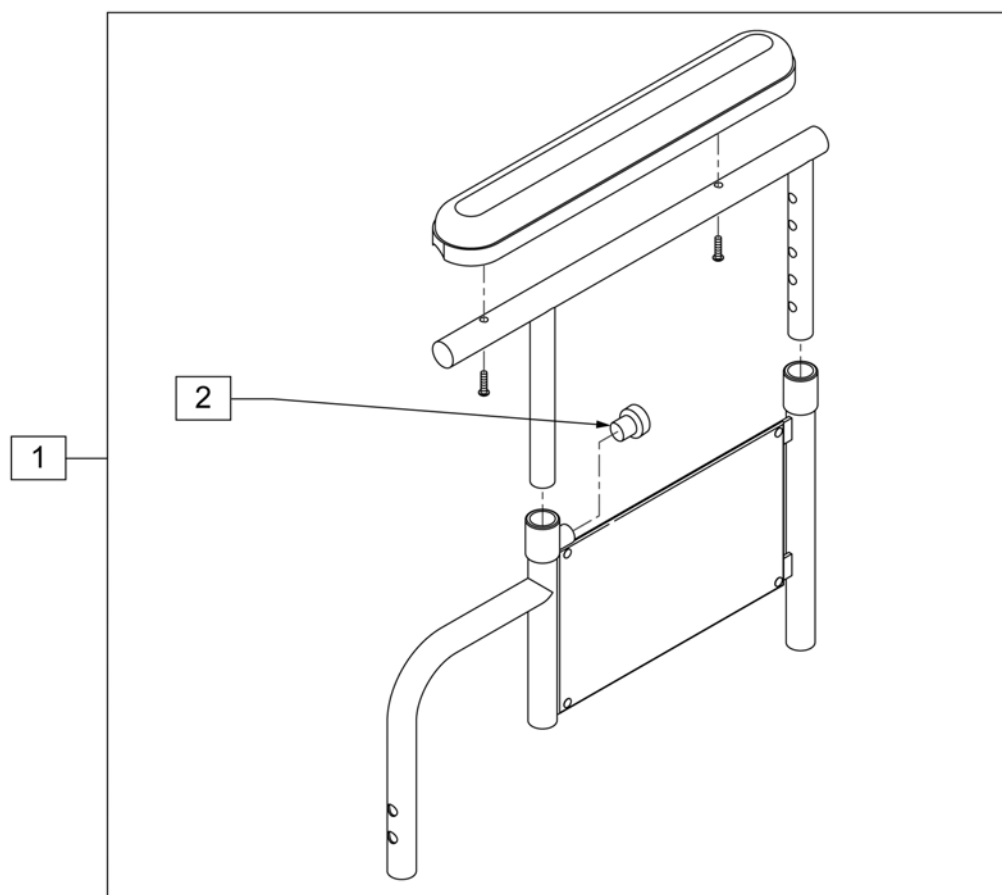


Pos.	Item Number	Description
1,2	GEY-HAFP	HEIGHT ADJ ARMREST FULL PADDED
2	GEY-4HAKNOB	HEIGHT ADJ RELEASE KNOB

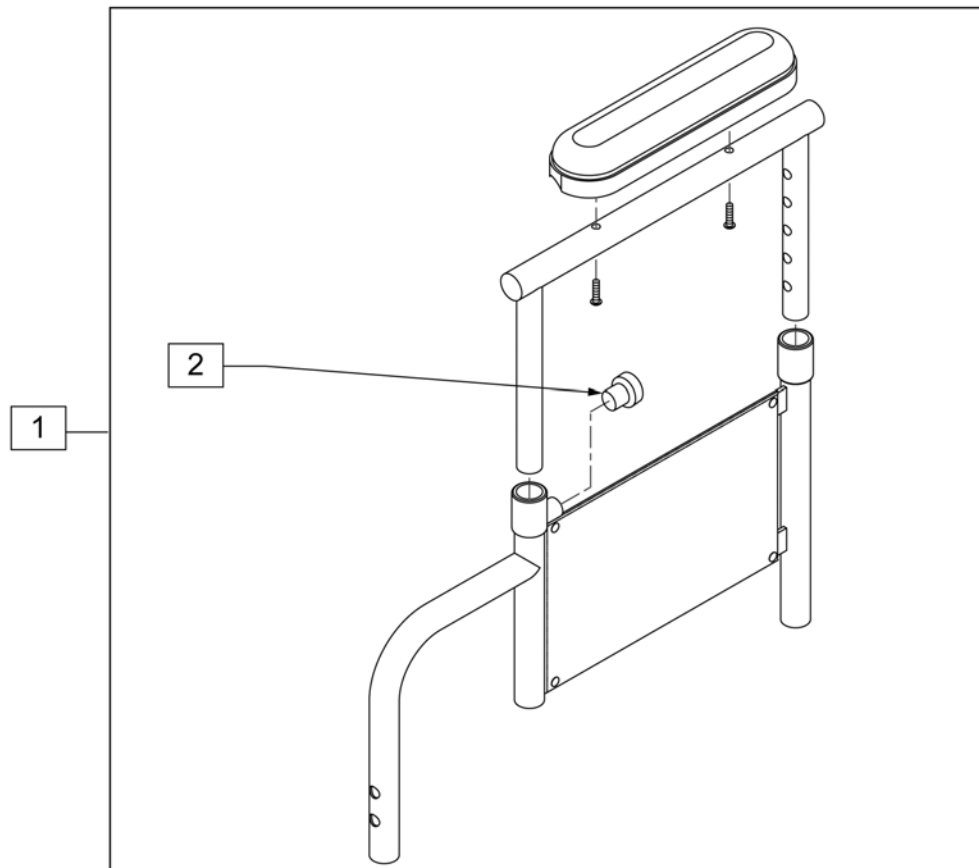



Pos.	Item Number	Description
1,2	GEY-HADP	HEIGHT ADJ ARMREST DESK PADDED
2	GEY-4HAKNOB	HEIGHT ADJ RELEASE KNOB

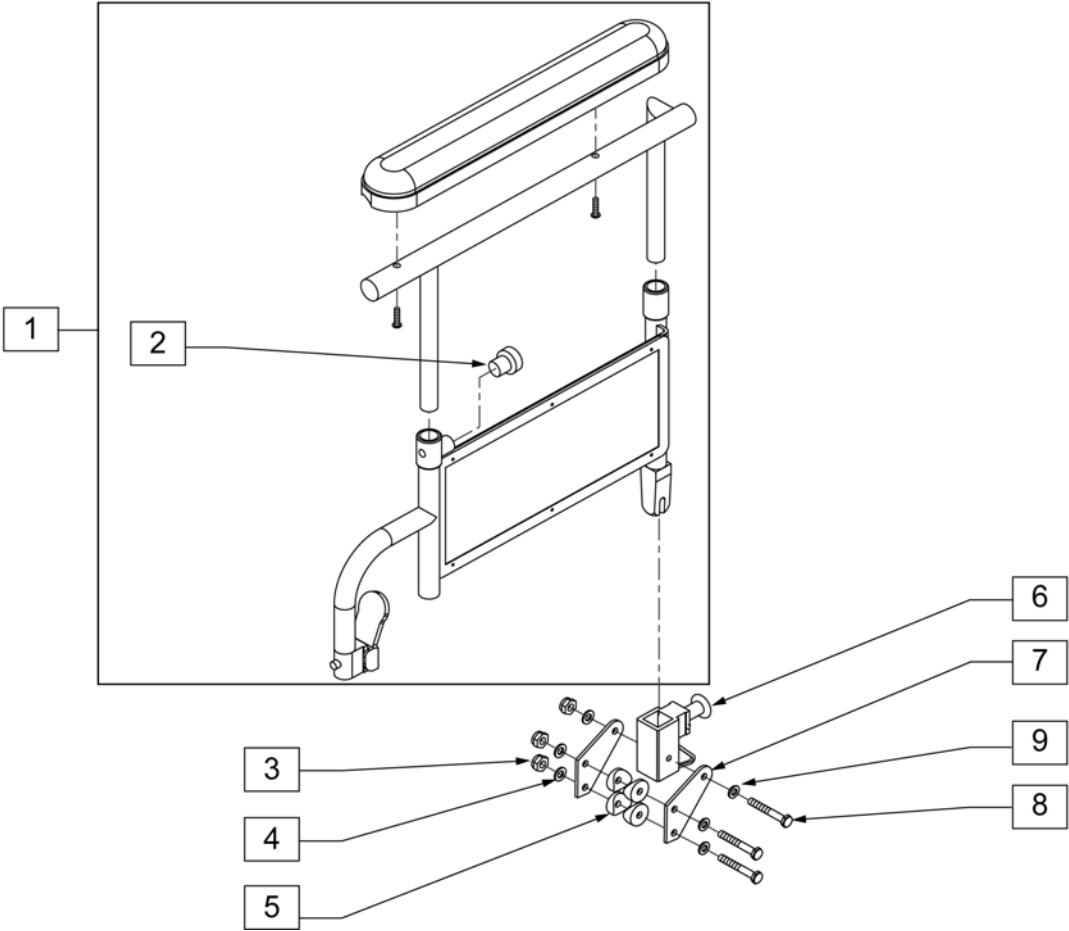


Pos.	Item Number	Description
1	GEY-4HAFP	HEIGHT ADJUSTABLE ARMREST FULL PADDED
2	GEY-4HAKNOB	HEIGHT ADJUSTABLE RELEASE KNOB
6-9	GEY-4ARR	4000 ANGLE ADJUSTABLE ARM RECEIVER

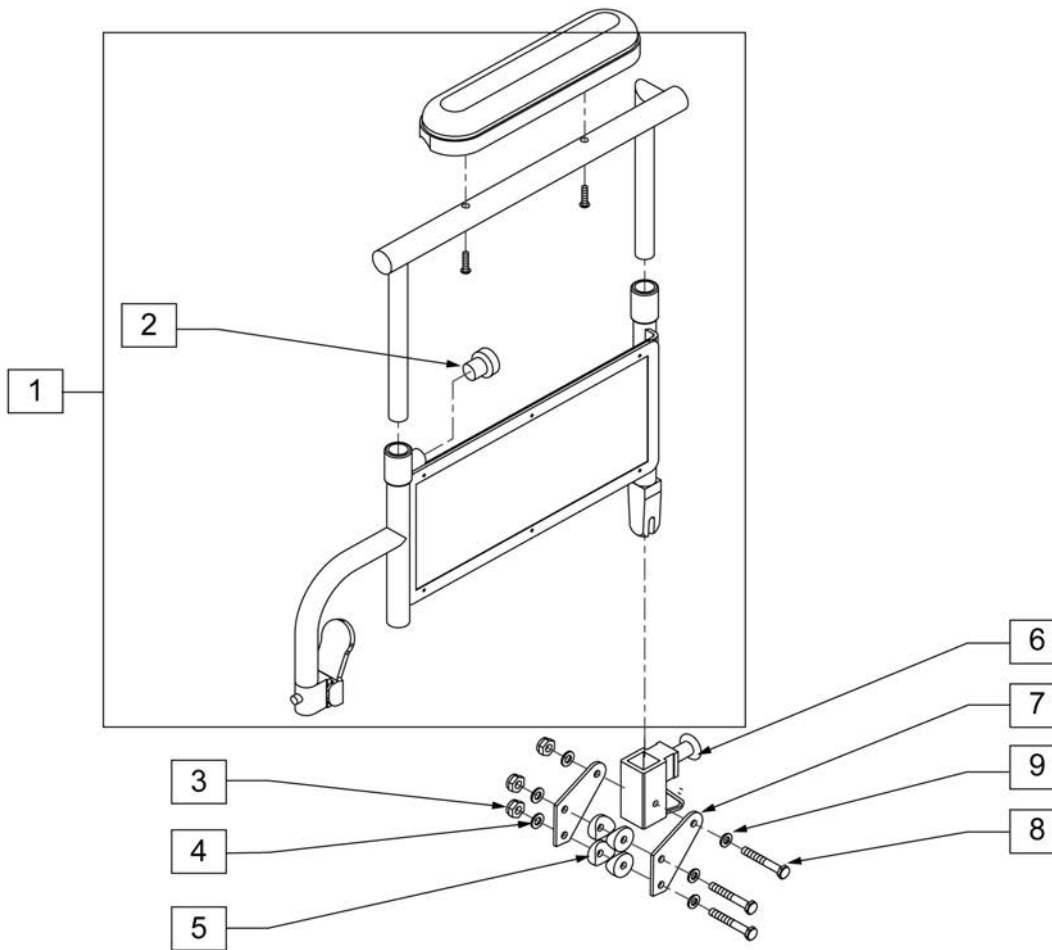

Pos.	Item Number	Description
1	GEY-4HADP	HEIGHT ADJUSTABLE ARMREST DESK PADDED
2	GEY-4HAKNOB	HEIGHT ADJUSTABLE RELEASE KNOB
6-9	GEY-4ARR	4000 ANGLE ADJUSTABLE ARM RECEIVER

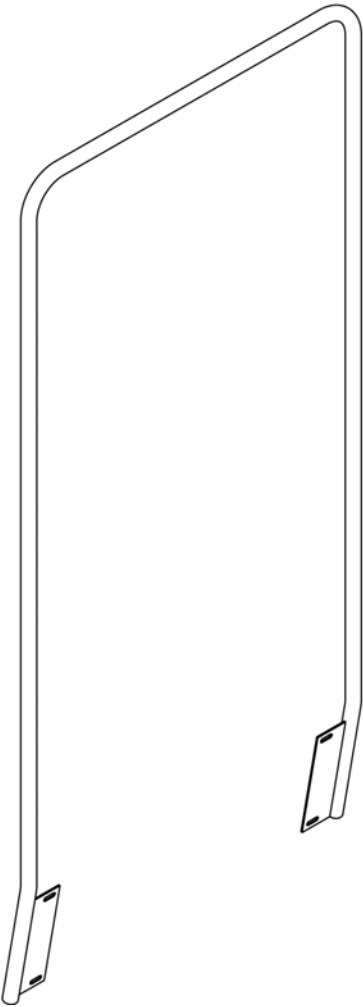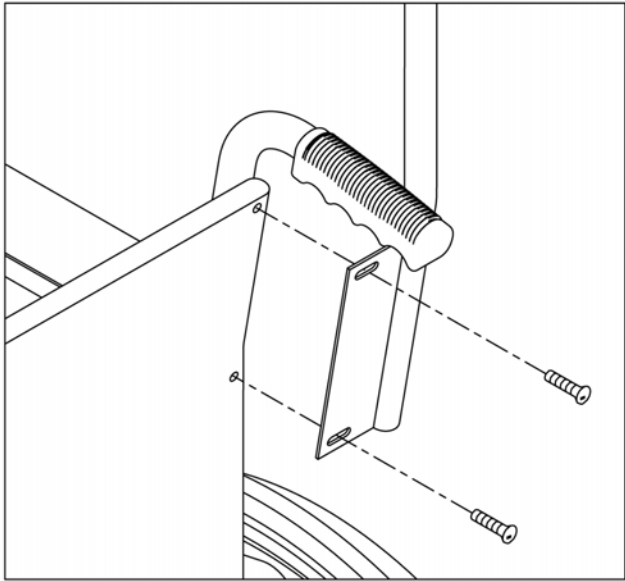


Pos.	Item Number	Description
1	GEY-OATH	EC OVERHEAD ANTI-THEFT DEVICE

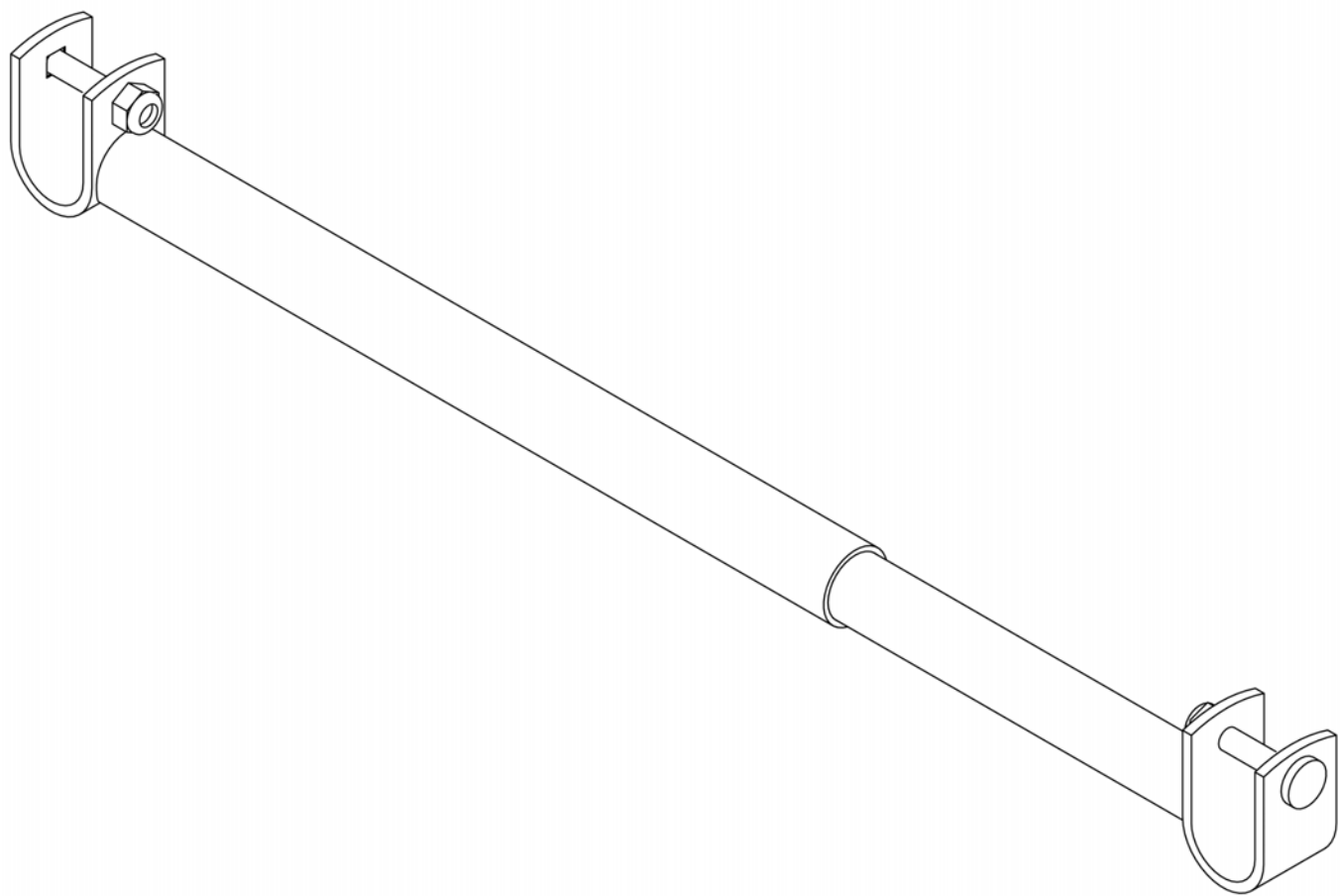


Pos.	Item Number	Description	Remarks
1	GEY-ATH	EC ANTI-FOLD/ ANTI-THEFT DEVICE	Adjustable from 16" to 24"

Note: top attachment point utilizes existing back upholstery hardware.

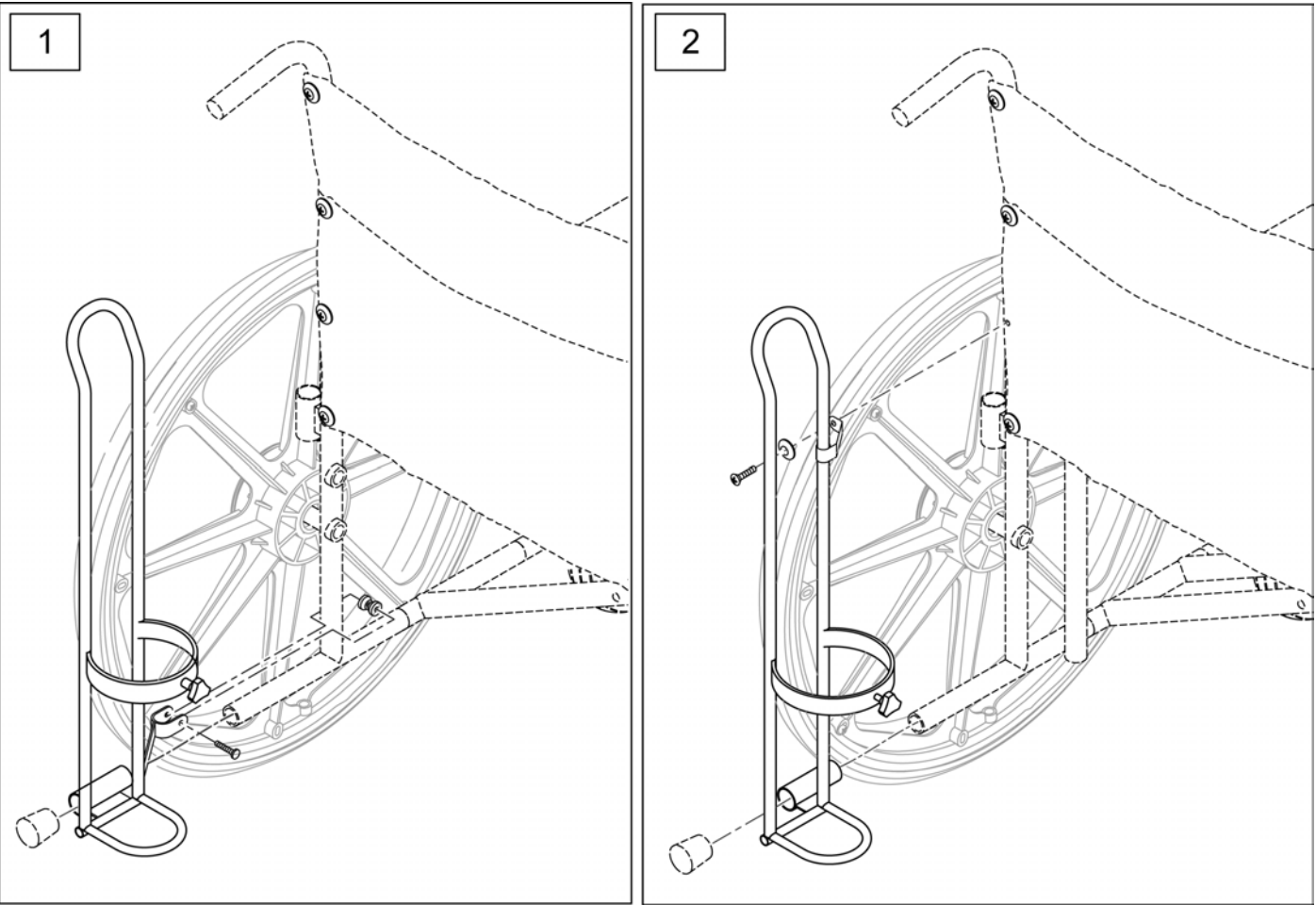

Pos.	Item Number	Description
1	GEY-O2	EC OXYGEN TANK HOLDER
1	GEYB-O2	BARIATRIC OXYGEN TANK HOLDER

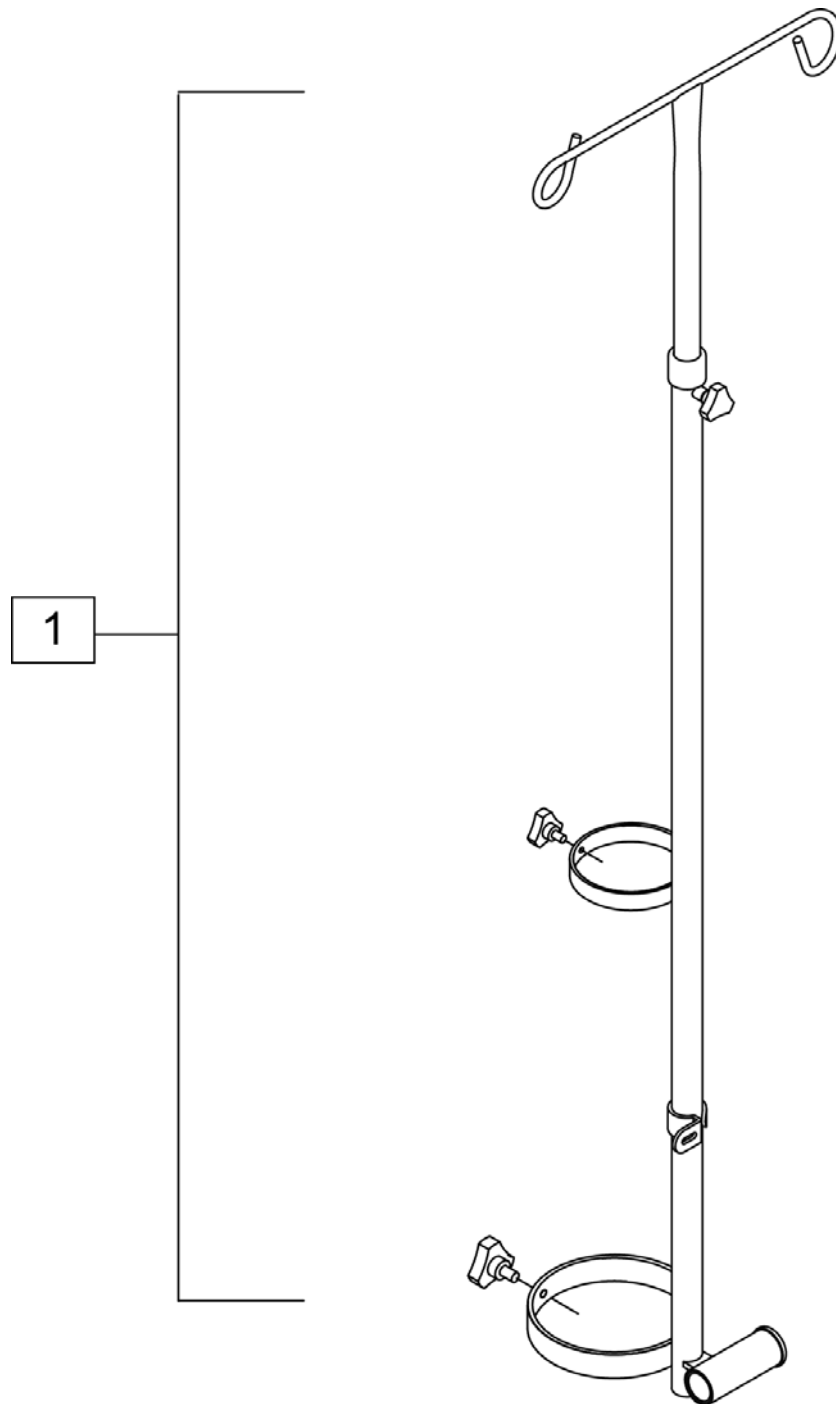


Pos.	Item Number	Description
1	GEY-O2IV	O2 IV HOLDER

Note: top attachment point utilizes existing back upholstery hardware.



Pos.	Item Number	Description
1	GEY-IVP	EC TELESCOPING IV POLE
1	GEYB-IVP	BARIATRIC TELESCOPING IV POLE

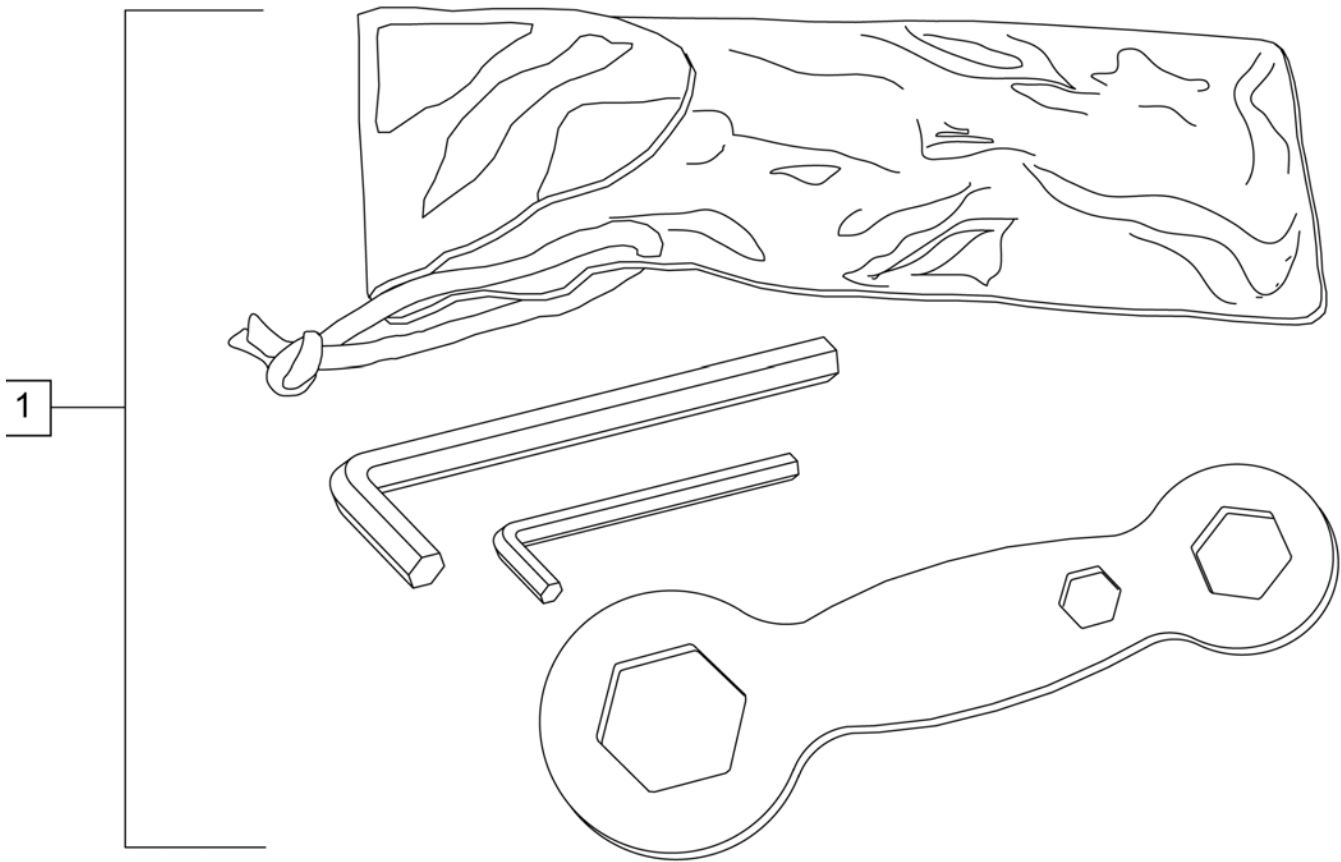

Pos.	Item Number	Description
1	GEY-1TK	TOOL KIT




Pos.	Item Number	Description
1	GEY-EXT	EC WHEEL LOCK EXTENSION HANDLE

Note: Seat Belt utilizes existing back upholstery hardware.



Pos.	Item Number	Description	Remarks
1	GEY-SBA	SEAT BELT AUTOMOTIVE STYLE	48.5" LONG AND 2" WIDE
2	GEY-SBV	SEAT BELT VELCRO STYLE	OVERALL LENGTH CAN BE ADJUSTED FROM 24" TO 35"

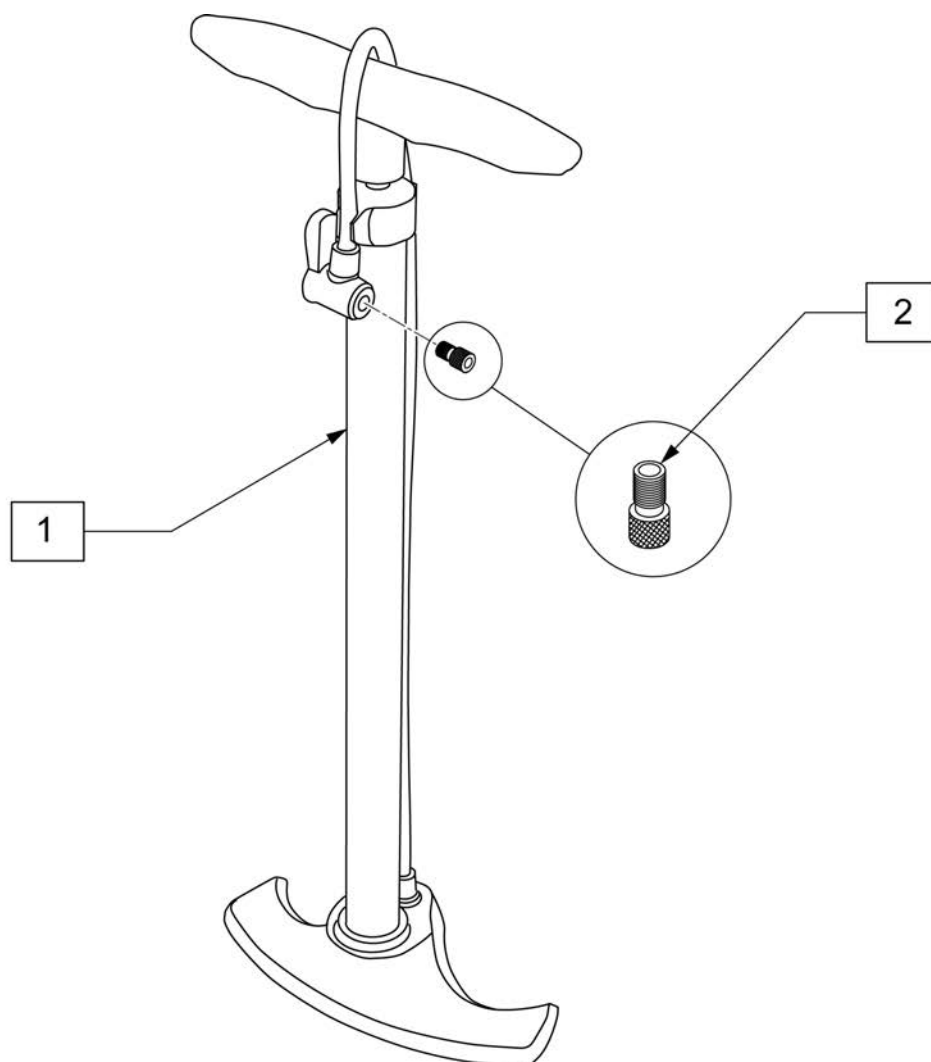


Pos.	Item Number	Description
1	389050	TIRE PUMP
2	381230	VALVE ADAPTER PRESTA




Pos.	Item Number	Description	Remarks
1	ECTC-ARMPAD	TRANSPORT CHAIR NYLON ARMPAD	pair
2	ECTCAR-FTREST	TRANSPORT CHAIR ALUM FTREST	RED (PAIR)
2	ECTCSB-FTREST	TRANSPORT CHAIR STEEL FTREST	BLACK (PAIR)
3	ECTC-RWHEEL	TRANSPORT CHAIR REAR WHEEL	pair
4	ECTC-CASTER	TRANSPORT CHAIR CASTER ASSEM	FORK INCLUDED (PAIR)
5	BZU9903	RPO HANGER PINS ASSEMBLY	INCLUDES 4 PINS
6	ECTC-17UPH	TRANSPORT CHAIR 17" UPH	Use for Steel and Aluminum
6	ECTC-19UPH	TRANSPORT CHAIR 19" UPH	Use for Steel and Aluminum

Note: Replacement Frames for the Transport chair are not available as a replacement part.

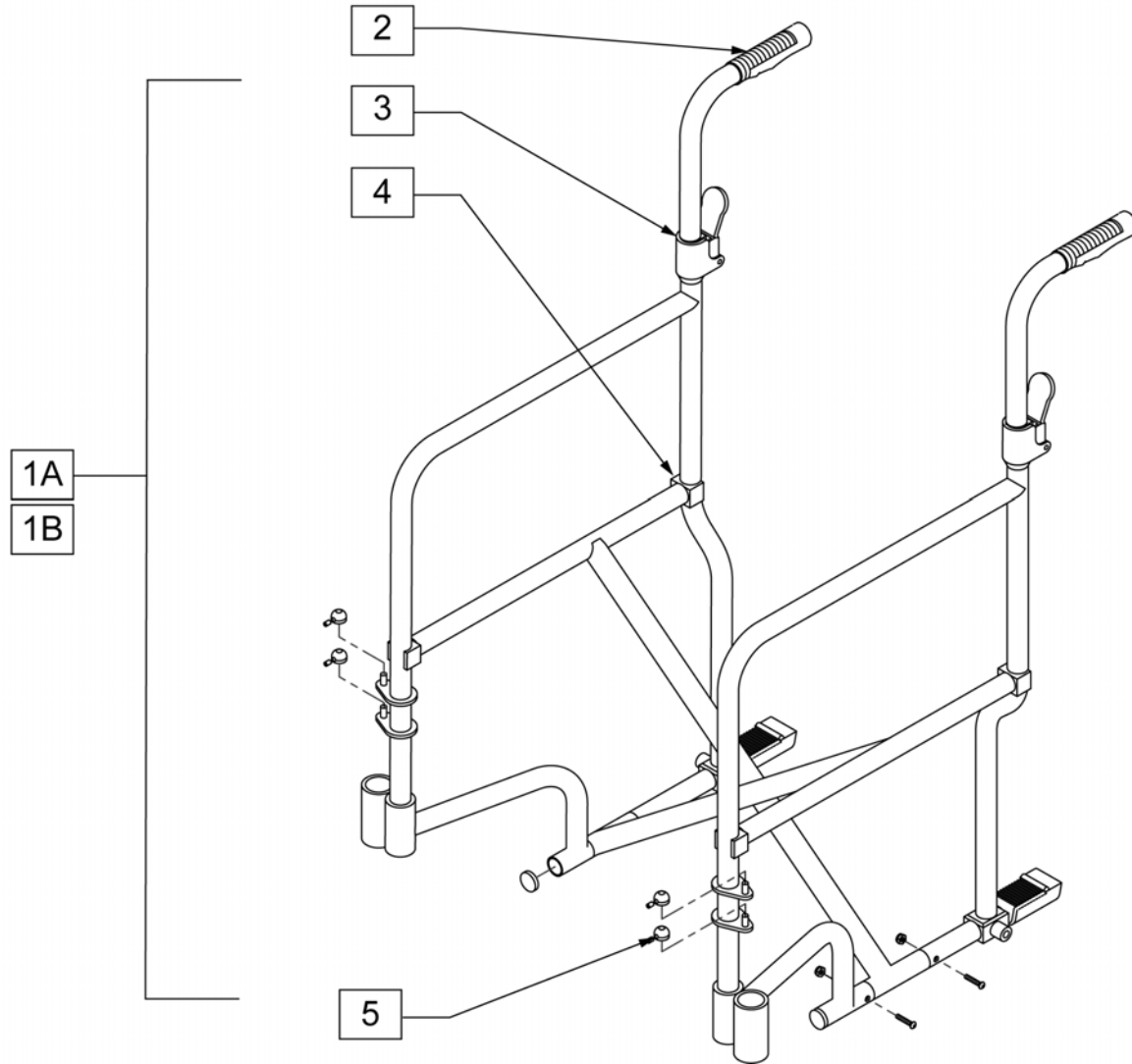


Pos.	Item Number	Description	Remarks
2	GEY-HAND	EC BACK CANE HANDLES	
4	GEY-SRGUIDE	SEAT RAIL GUIDE	
5	GEY-TIN	ONE PAIR TINNERMAN NUTS	PAIR

NOTE: ITEM 2 REQUIRES A 19mm SOCKET TO REMOVE

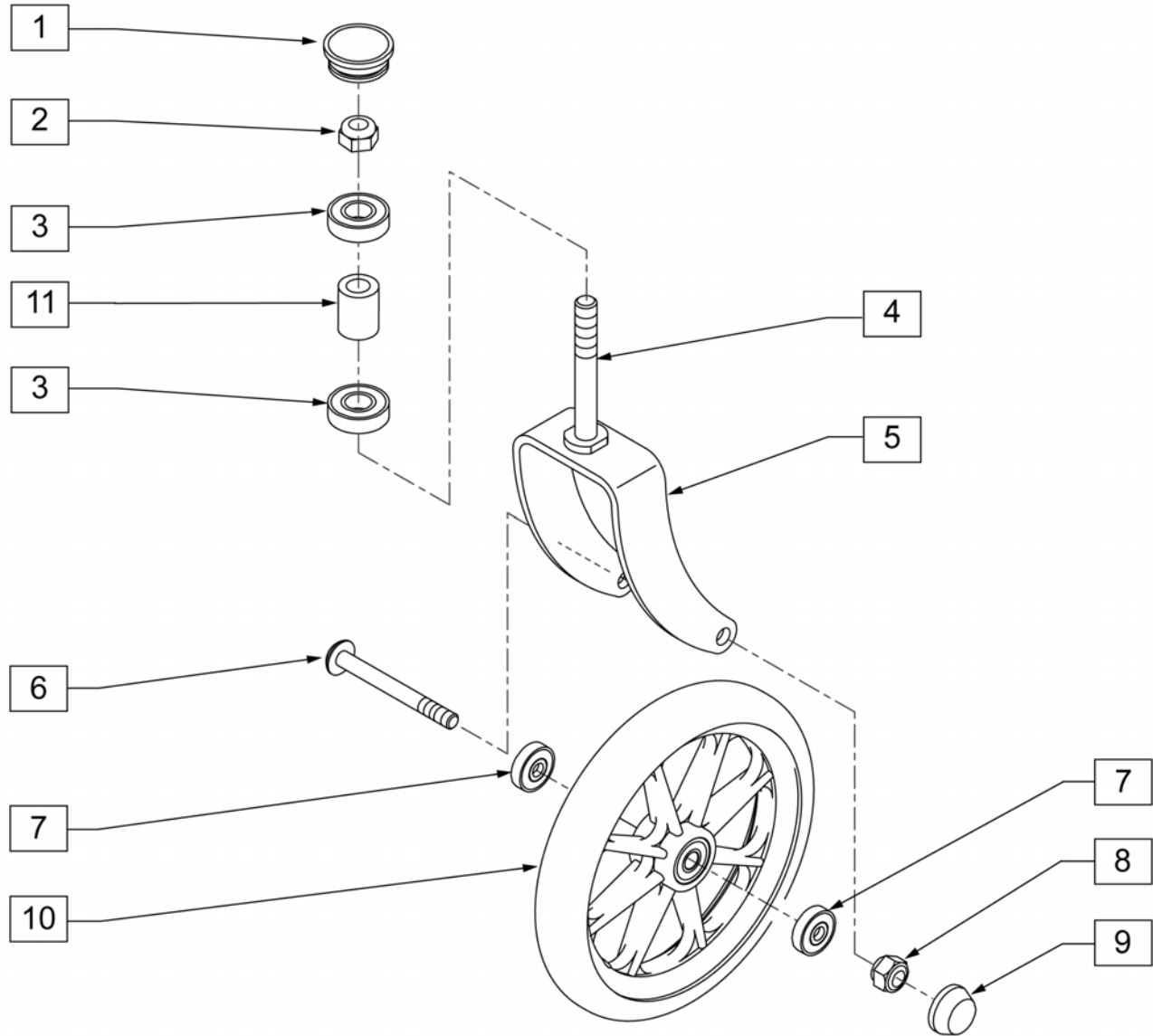


Pos.	Item Number	Description	Remarks
1	GEY-CASC	CASTER HOUSING CAPS	20 Per Bag
3, 11	GEY-CASTJR	CAST JOURNAL BEAR & SLEEVE	PAIR INCLUDES 4 BEARINGS AND 2 SLEEVES - BEARING SIZE = 28mm X 12mm X 36.25mm
6, 7	GEY-CASTBR	CASTER BEARINGS	PAIR - SIZE=22mm X 8mm X 31.5mm
9	GEY-CASTCAP	CASTER TIR BOLT CAP	20 Per Bag
1-11	GEY-1TCA	CASTER ASSM FOR STEEL	EACH


NOTE: ITEM 2 REQUIRES A 19mm SOCKET TO REMOVE

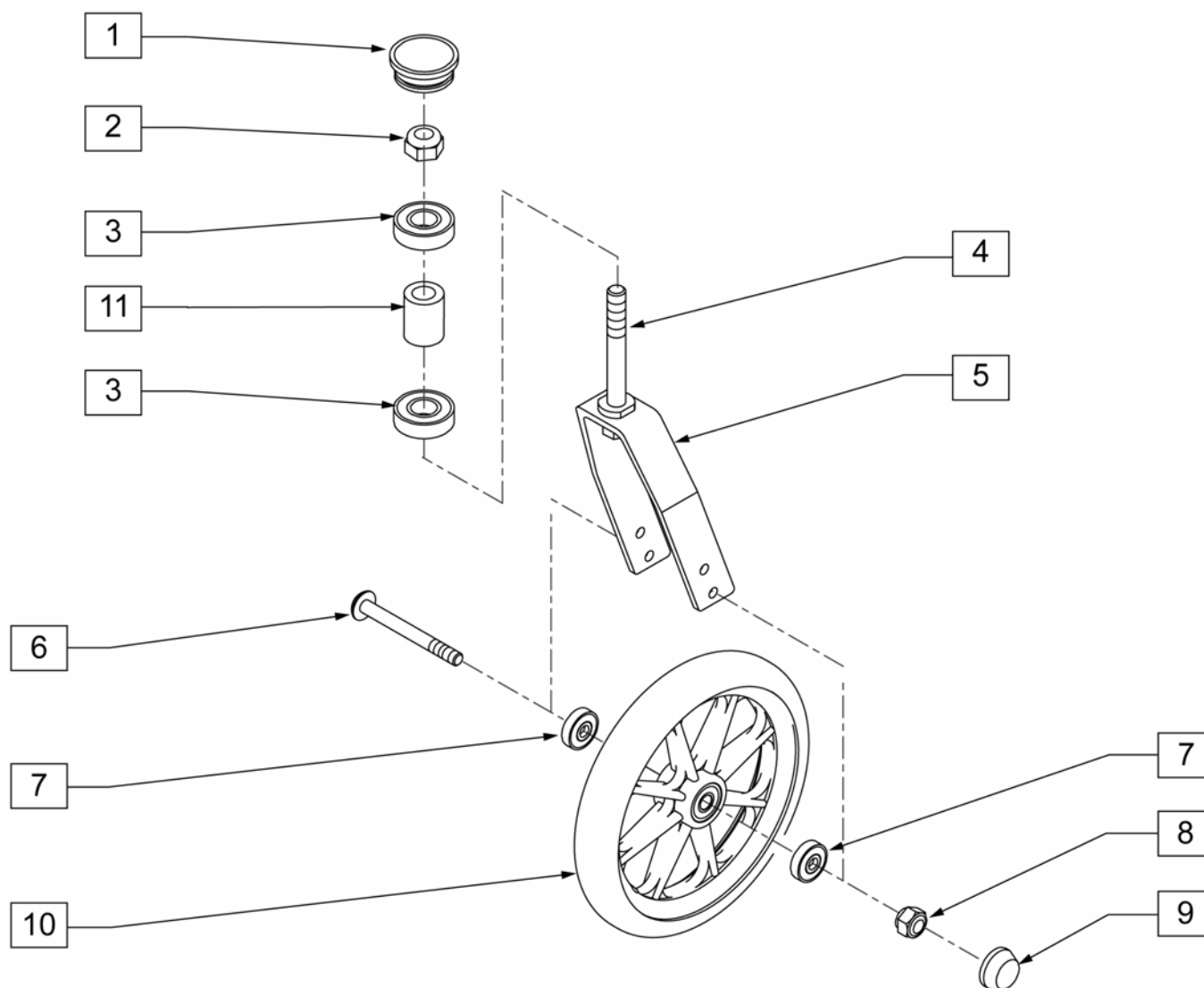


Pos.	Item Number	Description	Remarks
1	GEY-CASC	CASTER HOUSING CAPS	20 Per Bag
3, 11	GEY-CASTJR	CAST JOURNAL BEAR & SLEEVE	PAIR INCLUDES 4 BEARINGS AND 2 SLEEVES - BEARING SIZE = 28mm X 12mm X 36.25mm
7	GEY-CASTBR	CASTER BEARINGS	PAIR - SIZE=22mm X 8mm X 31.5mm
9	GEY-CASTCAP	CASTER TIR BOLT CAP	PAIR
1-10	GEY-1TACA	CASTER ASSM W/ HARDWARE	PAIR

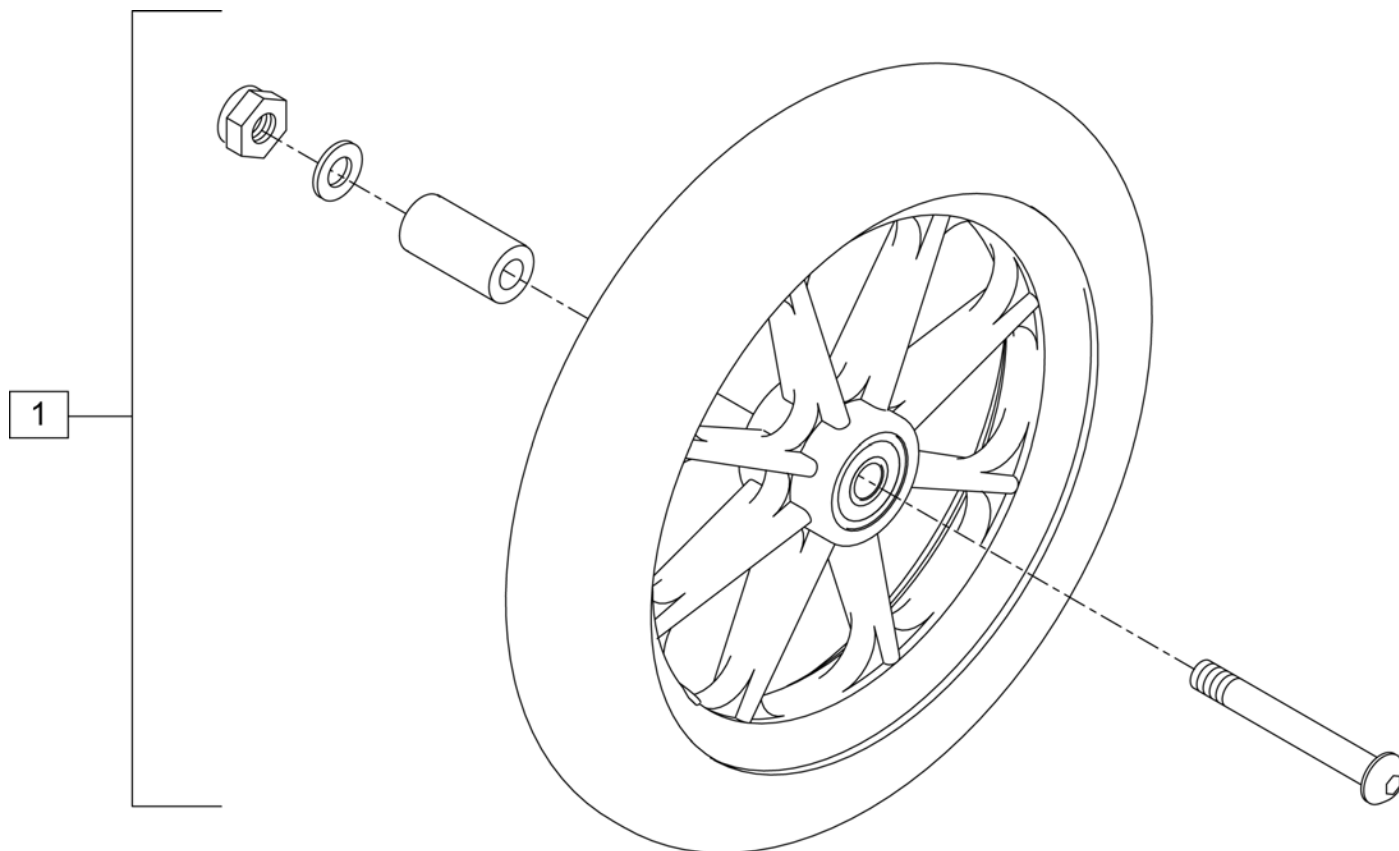


Pos.	Item Number	Description
1	GEY-1TWA	1 x 8 WHEELS W/ HARDWARE

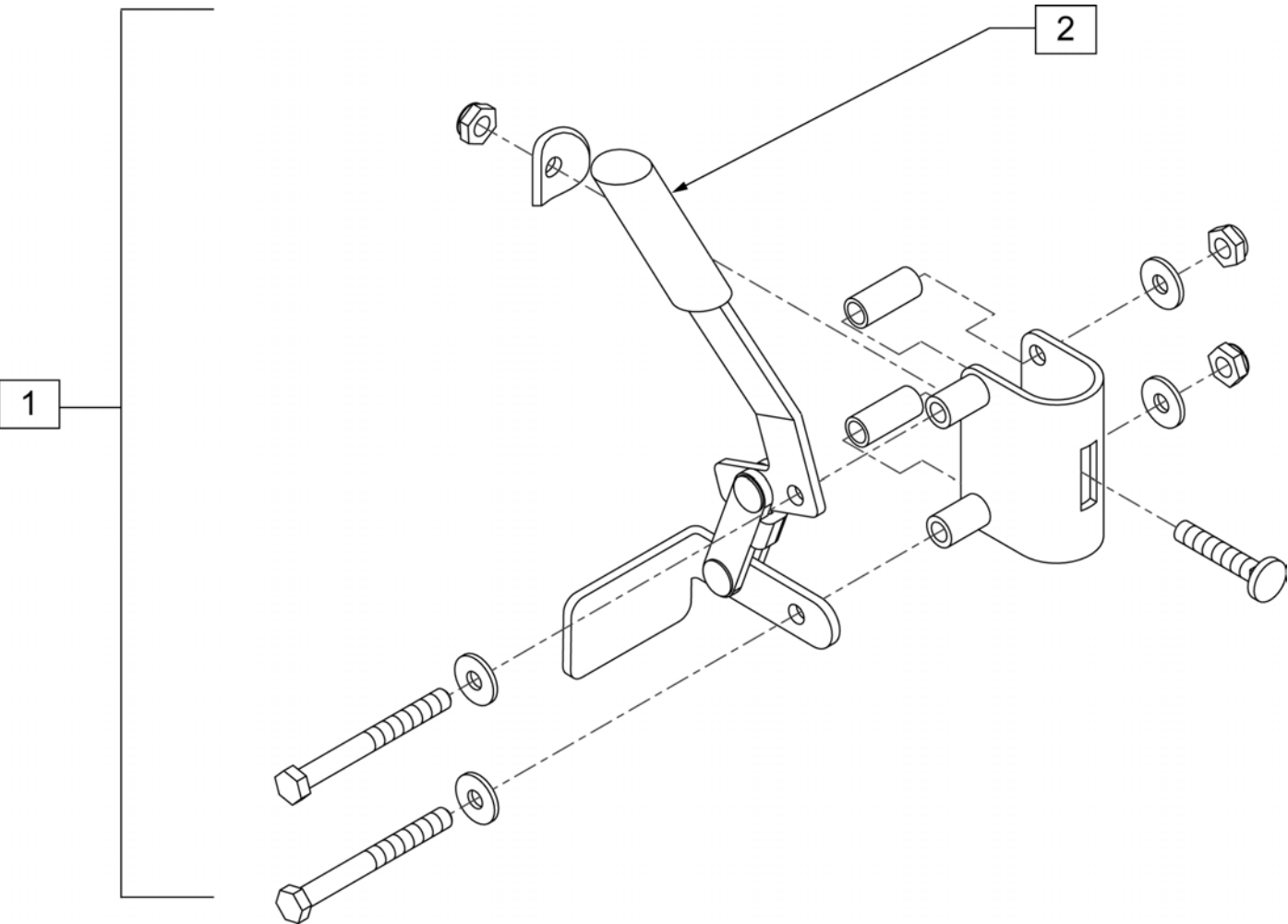


Pos.	Item Number	Description	Remarks
1	GEY-1TWL	WHEEL LOCK	PAIR
2	GEY-BRAKEHD	BRAKE HANDLE	

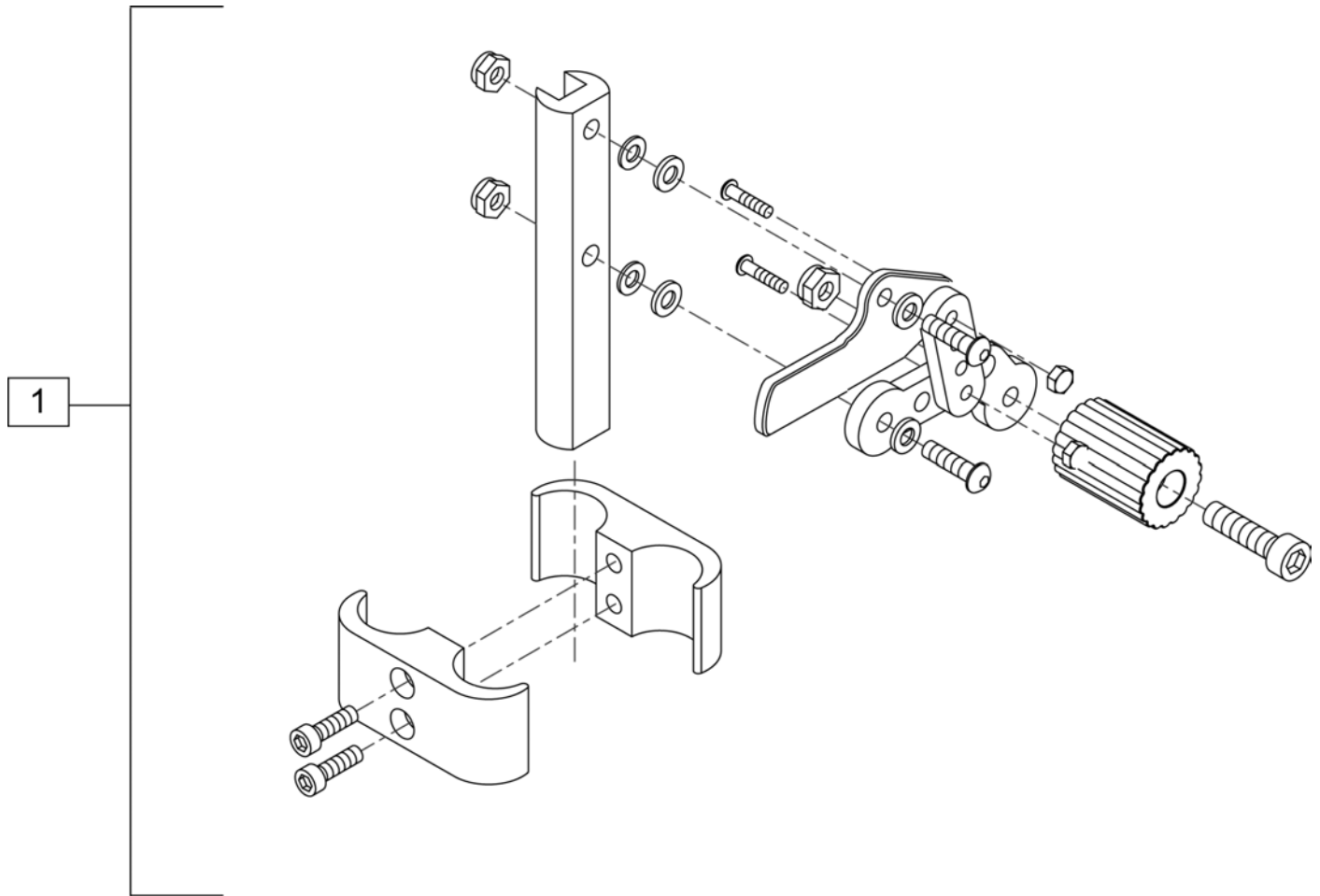


Pos.	Item Number	Description
1	GEY-1TAWL	ALUMINUM TRANSPORT WHEEL LOCK W/ MOUNTING HDWR

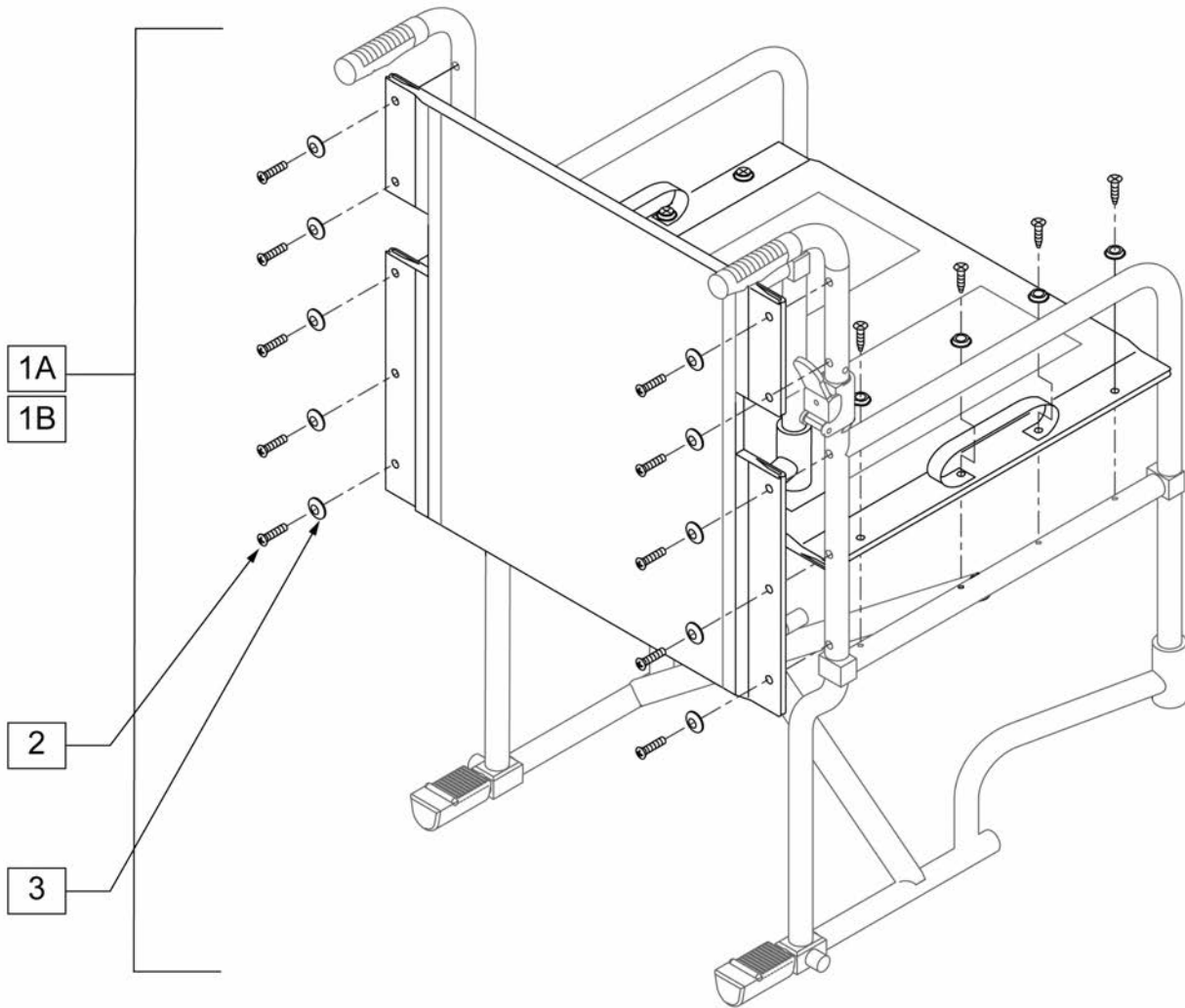

Pos.	Item Number	Description
2, 3	GEY-UPHHW	UPHOLSTERY HARDWARE
1	N/A	NO LONGER AVAILABLE

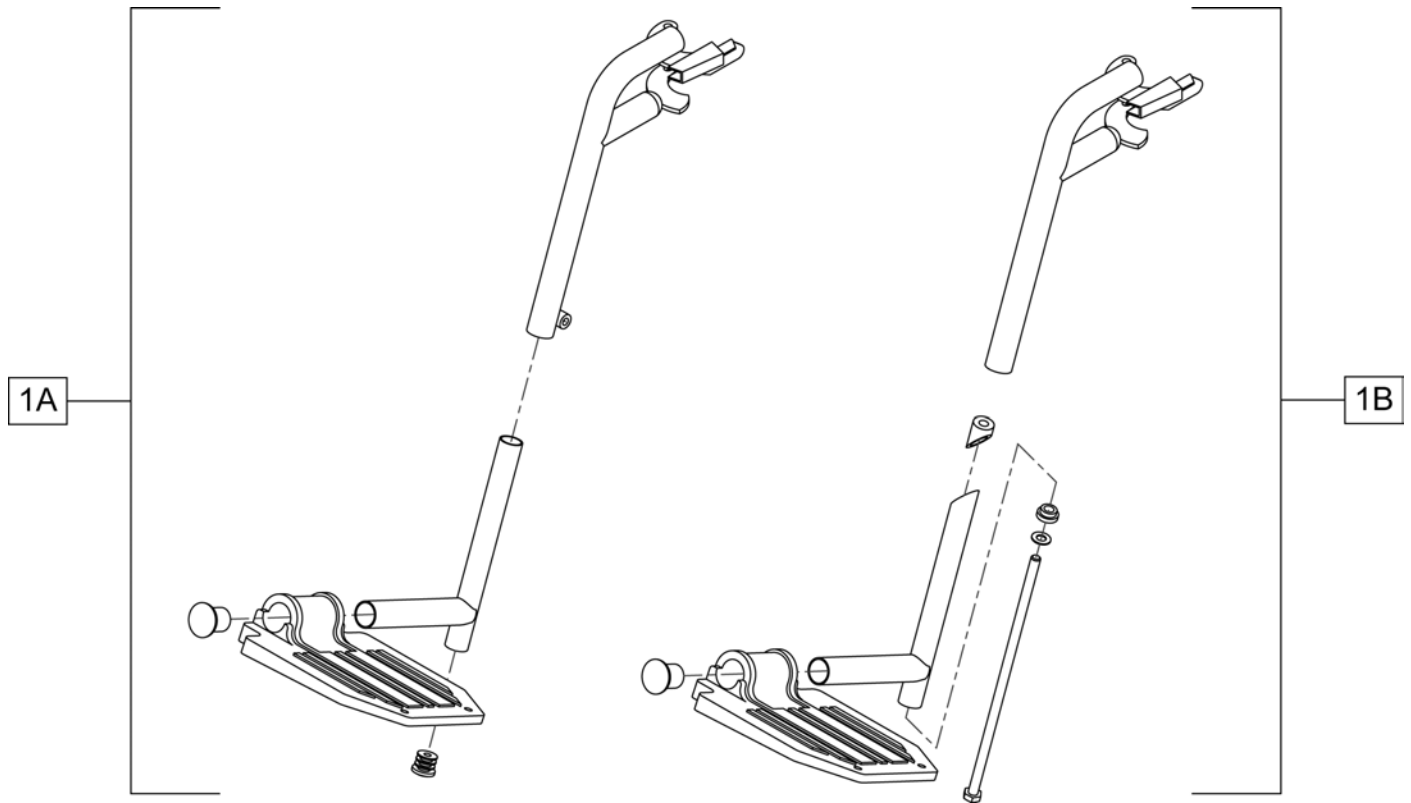

Pos.	Item Number	Description
1A	GEY-1TFR	FOOTREST STEEL TRANSPORT
1B	GEY-1ATFR	Discontinued, parts no longer available

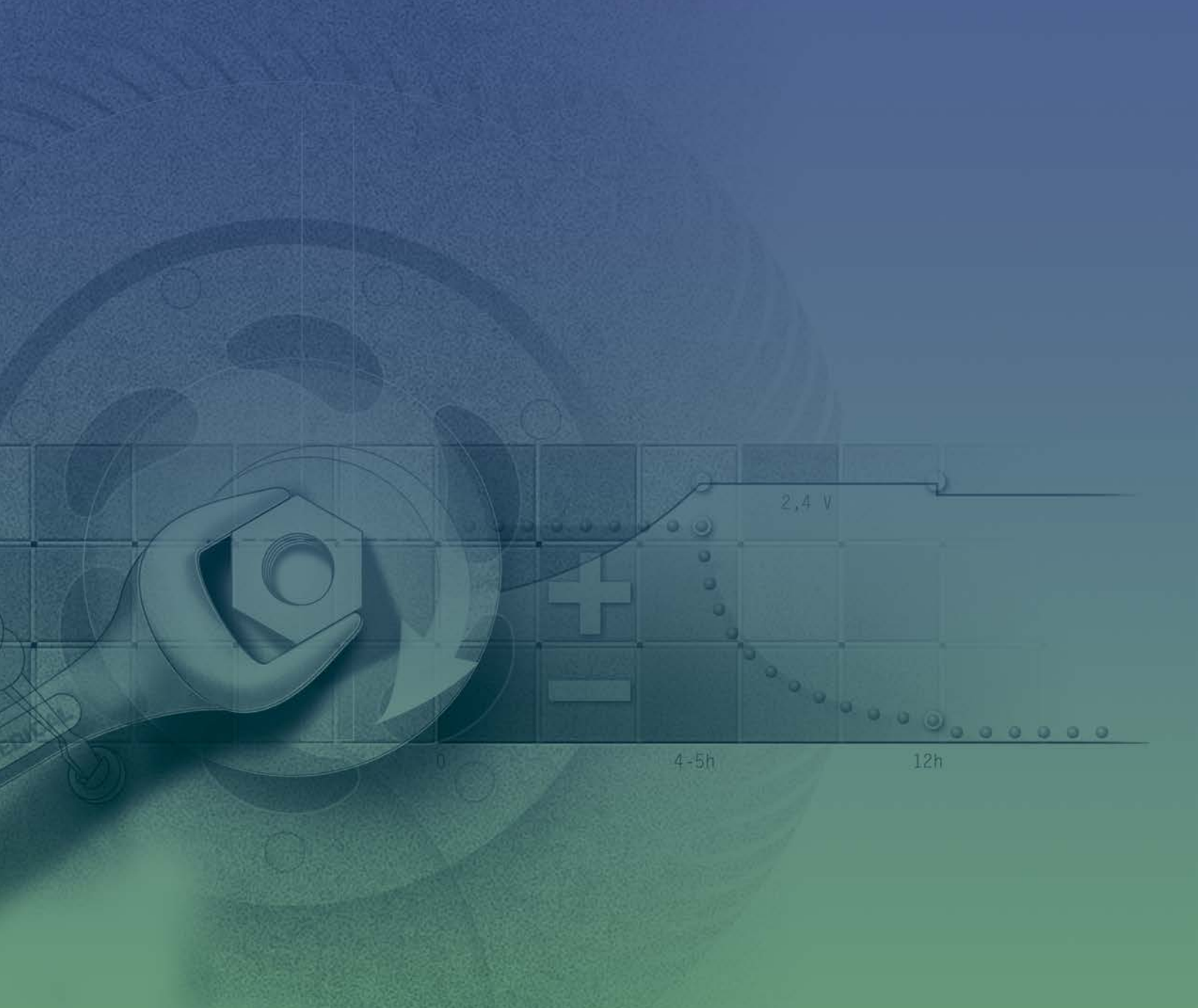

Breezy EC.®